



ResMed

Our softest
mask ever

To find out more visit
[ResMed.com/AirTouchReminders](https://www.resmed.com/AirTouchReminders)



 **RespShop**

AirTouch™ F20



Caring for your AirTouch™ F20 cushion



With ResMed's UltraSoft™ memory foam cushion, you'll experience incredible softness with the same high performance as silicone. But unlike silicone, keeping your memory foam fresh requires an entirely different kind of care. So, think of your memory foam cushion like you would a toothbrush. Replace it when it starts to show wear.



Discard at signs of deterioration



Replace at signs of deterioration



Clean with alcohol-free wipes only



Keep dry and away from liquids



Do not expose to direct sunlight



Do not use domestic cleaner or domestic cleaning devices



Do not use alcohol wipes



Do not submerge in liquids



See ResMed.com for ResMed locations worldwide. AirTouch and UltraSoft are trademarks and/or registered trademarks of the ResMed family of companies. For patent information, see ResMed.com/ip. Product availability may vary across regions. © 2017 ResMed Ltd. 10110072/1 2017-04.