

RESMED



Join Wake Up to Sleep—ResMed's sleep apnea support community—at **WakeUpToSleep/Join**.


RespShop

Fitting

Swift FX Bella Series



1 Hold the pillows at your nose, ensuring the ResMed logo is at the top. Pull one strap around and behind your ear.

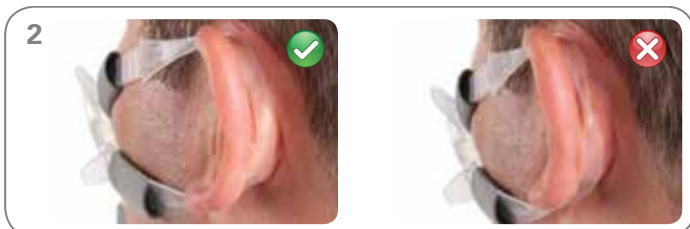

2 Ensure that the strap lies flat and comfortably across the ear. Take care to avoid twisting the strap.



3 Repeat the same process with the other strap.



4 Adjust the mask's fit by pulling both of the upper straps at the same time, along the notches, until you get a comfortable fit.



5 Repeat the process on the mask's lower straps. Do not overtighten.



6 Check that the pillows are upright at the entrance to the nose, then connect the free end of the short tube assembly to the air tubing of your device. Now it's ready for use.

Optional



You also have the option to wear your Swift FX Bella nasal pillows with the included Swift FX or Swift FX for Her headgear. For more information visit ResMed.com.



Combine a Swift FX Bella Series mask with ResMed's S9 AutoSet™ therapy device for a comfortable, quiet and lifestyle-friendly system.



Swift™ FX Bella Series

Nasal PIlIOWs sYsTEM

Choice fit. Complete comfort.

RESMED

Choice fit. Complete comfort.



Swift™ FX Bella Series

Nasal PIlIOWs sYsTEM

ResMed.com

Featuring a minimal design that provides a feeling of freedom, the **Swift™ FX Bella Series** is lighter, softer and more streamlined for maximum comfort.

* Combined responses of those expressing a preference in an independent patient survey in the USA, UK, Germany, France & Australia, from September 2012. Visit www.resmed.com/maskbrand.
ResMed Corp San Diego, CA, USA +1 858 836 5000 or 1 800 424 0737 (toll free). ResMed Ltd Bella Vista, NSW, Australia +61 (2) 8884 1000 or 1 800 658 189 (toll free). See ResMed.com for other ResMed locations worldwide.
AutoSet, S9, S9 AutoSet and Swift are trademarks of ResMed Ltd and are registered in the U.S. Patent and Trademark Office. ©2013 ResMed. Specifications may change without notice. 1015999/1 2013-01


Swift FX Bella Series

Easy, comfortable style With its unobtrusive design and sophisticated cushion technology, Swift FX Bella Series, nasal pillows provide a comfortable fit with a feeling of freedom for men and women.


Pillows (3 sizes) come packaged with the mask system to fit a wide range of users.



Swift FX Bella Series

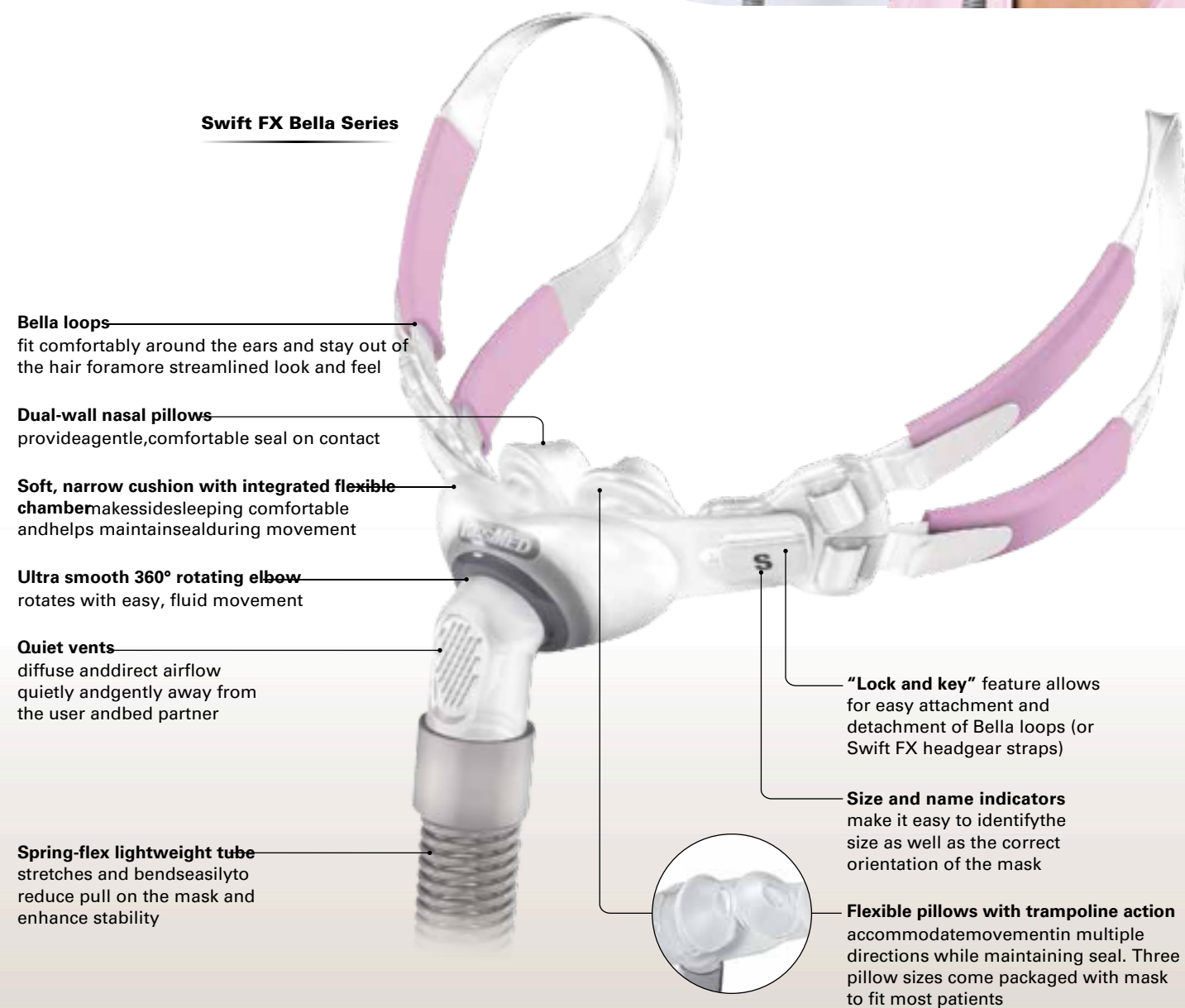

Swift FX Bella components

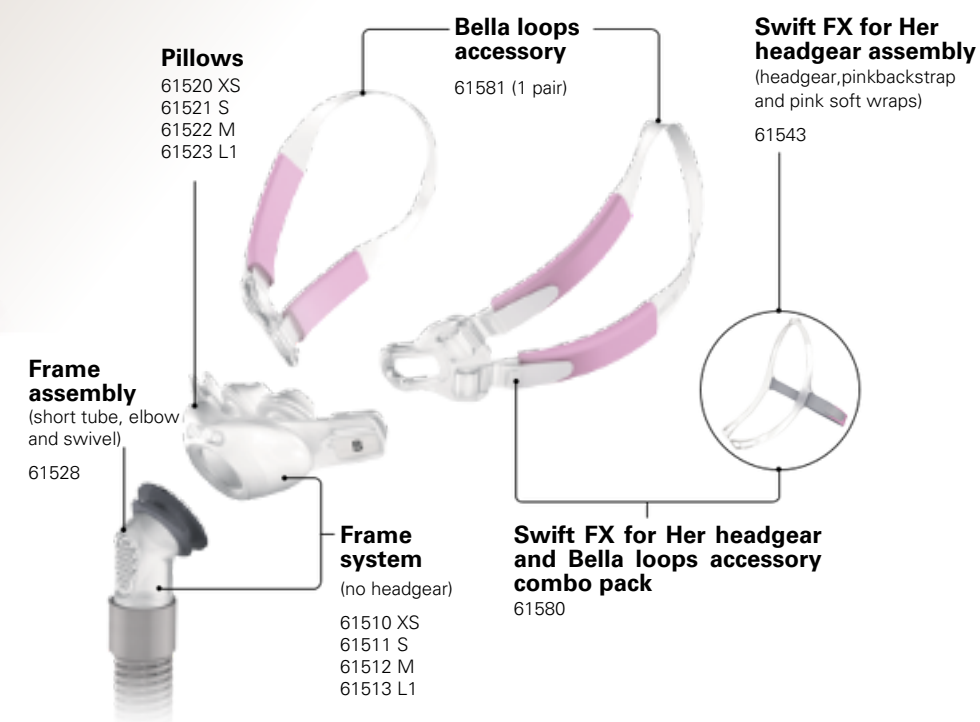

PRODUCT CODES

Complete Systems

Swift FX Bella Nasal Pillows Mask

Style	Part Number
Swift FX Bella	61560


Swift FX Bella complete system includes Bella loops attached to the pillows frame, Swift FX for Her headgear with pink backstrap and pink soft wraps. Also included are Extra Small (XS), Small (S) and Medium (M) pillows.¹

Reimbursement Codes

HCPCS	Code Descriptor
A7034	Mask interface—mask or cannula, 1 per 3 months
A7033	Pillow for use on nasal cannula type interface—replacement, 2 pairs per month
A7035	Headgear used with positive airway pressure device, 1 per 6 months

¹ Large pillows are available separately.

Swift FX Bella Gray components

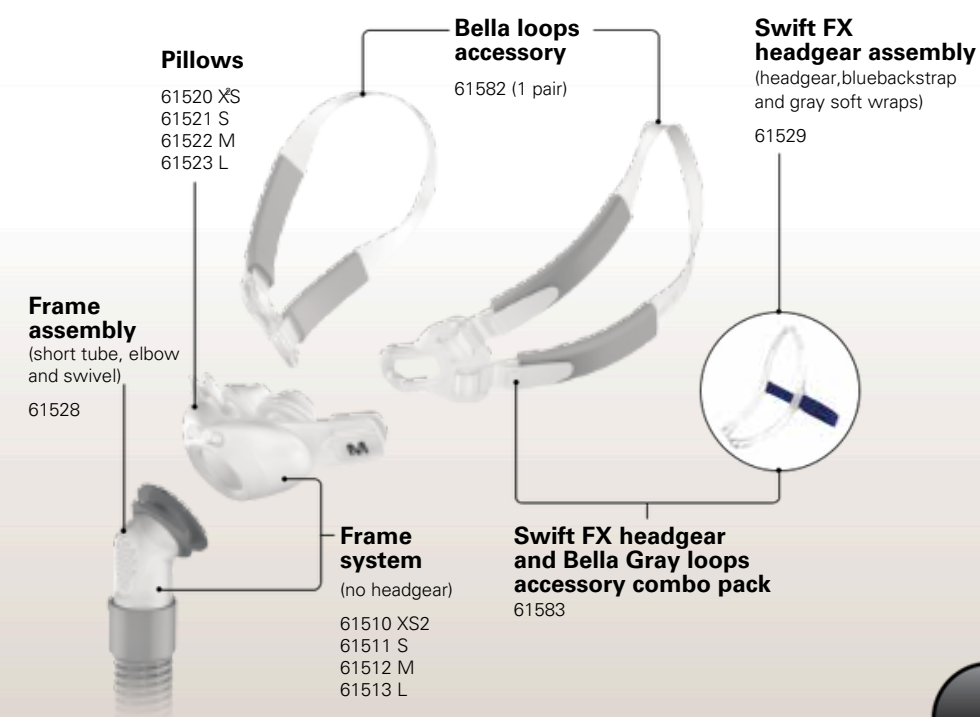

PRODUCT CODES

Complete Systems

Swift FX Bella Gray Nasal Pillows Mask

Style	Part Number
Swift FX Bella Gray	61568

Swift FX Bella Gray complete system includes Bella Gray loops attached to the pillows frame, Swift FX headgear with blue backstrap and gray soft wraps. Also included are Small (S), Medium (M) and Large (L) pillows.²

Reimbursement Codes

HCPCS	Code Descriptor
A7034	Mask interface—mask or cannula, 1 per 3 months
A7033	Pillow for use on nasal cannula type interface—replacement, 2 pairs per month
A7035	Headgear used with positive airway pressure device, 1 per 6 months

² Extra Small (XS) pillows are available separately.