

ResMed

Fitting guide for AirFit X30i



- With one hand holding the bottom of the cushion and the other on the frame, position the mask in front of your face.



- Place the cushion under your nose and ensure it sits against your face. With the ResMed logo on the headgear facing up, pull the headgear and frame over your head.



- Bring the lower headgear bands under your ears and attach the magnetic clips to the frame connectors.



- Undo the fastening tabs on the upper headgear bands and pull evenly.



- Repeat step 4 with the lower headgear bands.



- Connect the air tubing from the device to the mask elbow.

Additional options

For people with long hair, we recommend sliding the hair through the back of the headgear. If tied in a ponytail, the hair should sit high enough to fit through the back of the headgear.

Refer to the User Guide for further instructions and complete labeling information including indications, contraindications, warnings and precautions.

* Reimbursement for AirFit X30i is different than a full face mask. Medicare's allowable cushion replacement is one (1) per month for a full face mask. Because the AirFit X30i is an oral nasal mask, each of the oral and nasal pillows cushions may be replaced up to two (2) times per month. See below for the list of oral nasal HCPCS codes. For more information, refer to your Medicare local coverage determination policy, which you can find online at: <https://www.cms.gov/medicare-coverage-database/>

** The mask contains magnets that may interfere with certain implants or medical devices. Please refer to the User Guide for complete labeling information, including magnet contraindications and warnings.

Oral nasal HCPCS codes:

A7027 Combination oral/nasal mask

A7028 Replacement oral cushion for combination oral/nasal mask

A7029 Replacement nasal pillows for combination oral/nasal mask

A7035 Headgear

Note: Other payer guidelines may vary. It is your responsibility to verify all reimbursement requirements.

The information is being provided on an "as is" basis with no express or implied warranty of any kind and should be used solely for your internal informational purposes only. The information does not constitute professional or legal advice on reimbursement and should be used at your sole liability and discretion. All coding, coverage policies and reimbursement information are subject to change without notice. ResMed does not represent or warrant that any of the information being provided is true or correct, and you agree to hold ResMed harmless in the event of any loss, damage, liabilities or claims arising from the use of the reimbursement information provided to you. Before filing any claims, it is the provider's sole responsibility to verify current requirements and policies with the payer.

Distributed by ResMed Corp, 9001 Spectrum Center Boulevard, San Diego, CA 92123 USA +1 858 836 5000 or 1 800 424 0737 (toll-free). See ResMed.com for other ResMed locations worldwide. AirFit and QuietAir are trademarks and/or

registered trademarks of the ResMed family of companies. Specifications may change without notice. For patent and other intellectual property information, see ResMed.com/ip. © 2025 ResMed. 10112729/1 2025-04

ResMed.com / AirFit X30i



ResMed

AirFit™ X30i



AirFit™ X30i



Give patients a mask designed to fuse freedom with stability


Tube-up meets oral nasal design

Introducing the ResMed AirFit™ X30i tube-up oral nasal mask designed to offer patients the coverage of a full face mask with the added stability of a nasal pillows cushion.

AirFit X30i is an ideal option for patients who breathe through their mouth and want freedom of movement with the stability and seal of an oral nasal mask.



ResMed.com / AirFit X30i

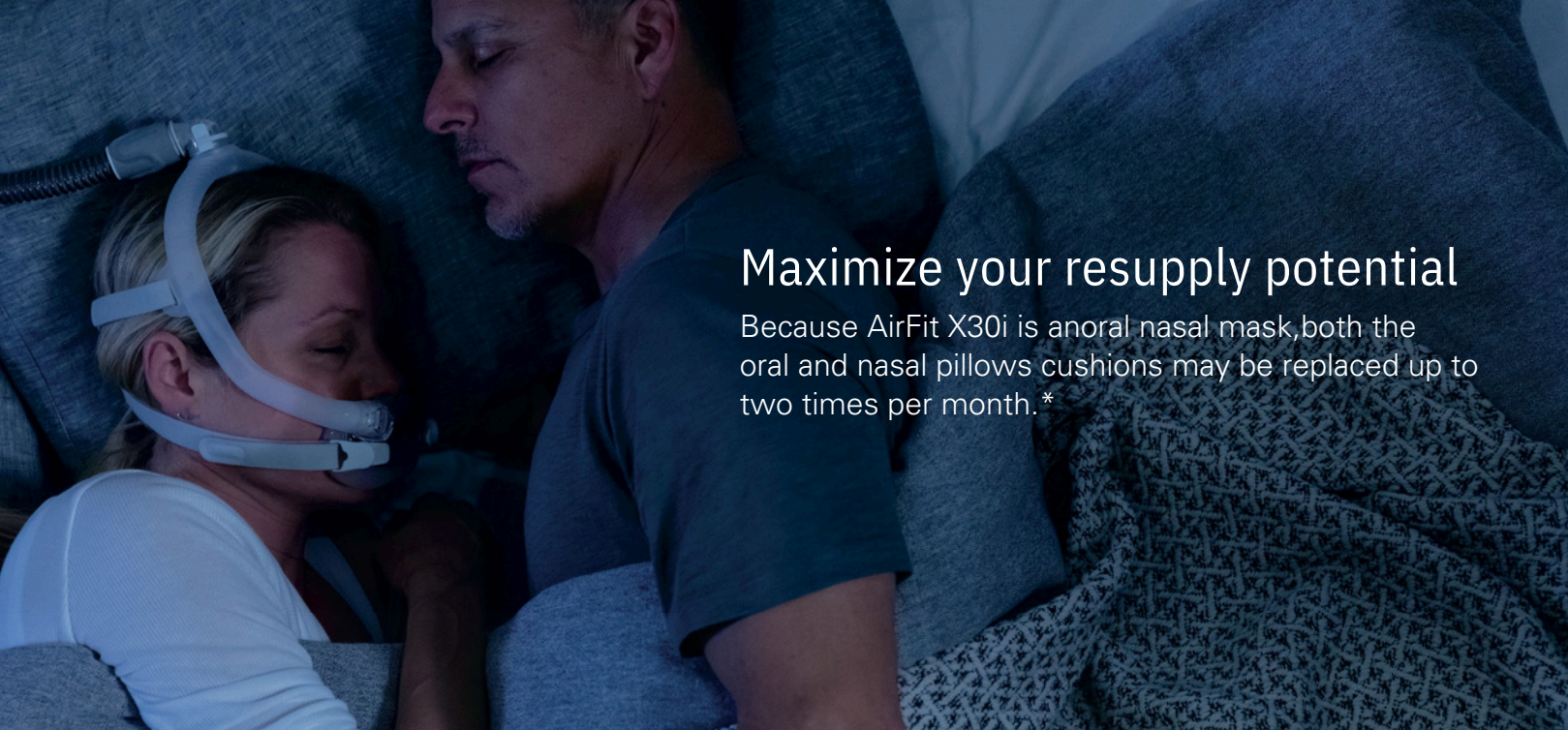

Maximize your resupply potential

Because AirFit X30i is an oral nasal mask, both the oral and nasal pillows cushions may be replaced up to two times per month.*

AirFit X30i highlights

Tube-up mask design
moves air tubing out of the way to give


patients the freedom to sleep in any position.

SpringFit frame
with flexible conduit allows pressurized air to flow through the elbow and into the frame and cushion.

Modular headgear and frame system
is compatible with AirFit F30i mask to allow new and existing patients to determine which is best tailored to their needs. For patients who have trouble with the seal, or their mask is too loud, the AirFit X30i may be a better option.

Magnetic clips**
snap on and off for easy adjustment and removal.

Oral nasal mask
combines the coverage of a full face cushion with the seal and stability of nasal pillows cushion, using dual cushions for comfort and performance.

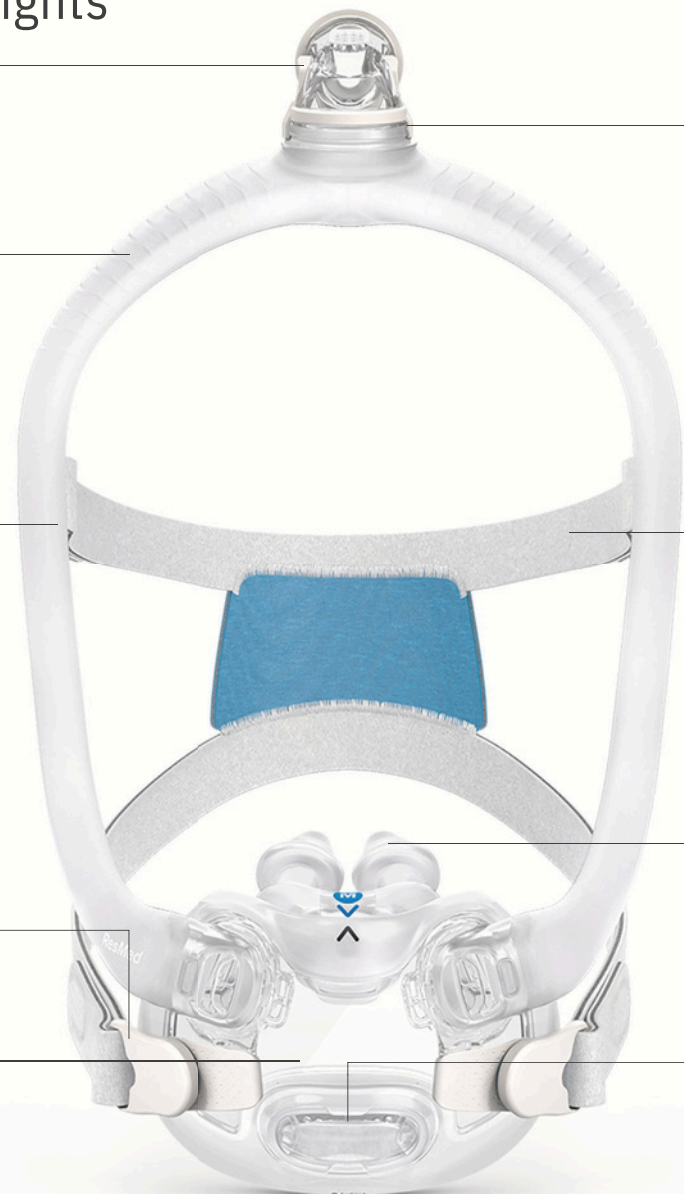

Quick-release elbow
allows patients to easily disconnect and reconnect mask from tubing.

Plush headgear
is specifically designed for softness and stability by including multiple connection points to the mask system.

Pillows cushion
rests on the edge of the nares to seal on contact. The pillows cushion connects to the oral cushion and disperses therapy pressure gently to the patient through the nose.

QuietAir™ diffused venting
is designed to be quieter than a whisper while minimizing breeze for both patients and bed partners.

Complete mask system

64300
64301
64350

Oral cushion (STD)
64308

Pillows cushion (S, M, L)
64309
6 4 310
6 4 311

Frame (SML, STD, LGE)

Headgear
63372

Frame system (64302, 64303, 64304, 64305, 64306, 64307, 64351, 64352, 64353)

AirFit™ X30i

AirFit X30i product codes

Complete mask system
(includes frame, oral cushion, pillows cushion and elbow)

Product name	Product code
AirFit X30i SML STPK	64300
AirFit X30i STD STPK	64301
AirFit X30i LGE STPK	64350

Nasal pillows cushion

Product name	Product code
AirFit X30i PILLOWS - S	64309
AirFit X30i PILLOWS - M	6 4 310
AirFit X30i PILLOWS - L	6 4 311

Oral cushion

Product name	Product code
AirFit X30i ORO CUSHION STD	64308

Headgear (includes headgear and magnetic clips)

Product name	Product code
AirFit F30i / X30i HEADGEAR	63372

Frame system

Product name	Product code
AirFit X30i SML FRAME SYS - S	64302
AirFit X30i SML FRAME SYS - M	64303
AirFit X30i SML FRAME SYS - L	64304
AirFit X30i STD FRAME SYS - S	64305
AirFit X30i STD FRAME SYS - M	64306
AirFit X30i STD FRAME SYS - L	64307
AirFit X30i LGE FRAME SYS - S	64351
AirFit X30i LGE FRAME SYS - M	64352
AirFit X30i LGE FRAME SYS - L	64353