


AirFit™F30i

Full face mask

The ResMed AirFit F30i* is a full face mask with a top-of-head tube connection designed to deliver a reliable seal with the freedom to sleep in any position.

Systems and Components

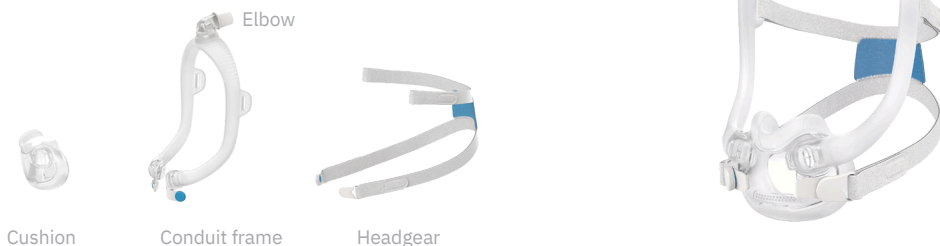

How to fit the AirFit F30i



Place cushion under the nose and against the face. With the ResMed logo facing up, pull headgear over head.



Bring the lower straps under the ears and reattach to the frame with the clips.* Feed longer hair through the back of the frame, if needed.



Adjust the upper straps and then the lower straps for a snug fit.

Ordering information

Product	Part number	HCPC	Recommended replacement schedule†
Complete mask system (includes frame, cushion and headgear) <ul style="list-style-type: none"> • Small frame—small cushion • Standard frame—small/medium/wide cushion • Large frame—medium cushion • Small non-magnetic frame—small cushion • Standard non-magnetic frame—small/medium/wide cushion • Large non-magnetic frame—medium cushion 	63330 63331/63332/63333 63334 63303 63304/63309/63315 63316	A7030 + A7035	—
Frame system (includes frame and cushion) <ul style="list-style-type: none"> • Small frame—small/medium/small wide/wide • Standard frame—small/medium/small wide/wide • Large frame—small/medium/small wide/wide 	63355/63356/63357/63358 63359/63360/63361/63362 63363/63364/63365/63366	A7030	Every 3 months
Conduit frame—small/standard/large	63368/63369/63370	—	—
Cushion—small/medium/small wide/wide	63350/63351/63352/63353	A7031	Monthly
Headgear <ul style="list-style-type: none"> • Standard • Non-magnetic, Standard Headgear 	63372 63317	A7035	Every 6 months

*The mask contains magnets that may interfere with certain implants or medical devices. Please refer to the User Guide for complete information, including magnet contraindications and warnings.

†Replacement cycle may vary depending on patient's insurance coverage.