


AirFit™ N30i

The ResMed AirFit N30i features a top-of-head tube design and nasal cradle cushion that allows patients the freedom to sleep in any position. The SpringFit frame is designed to adapt to each patient's face.

Nasal mask

Systems and Components

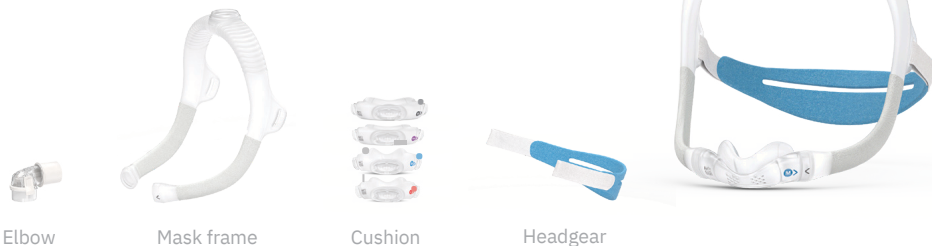

How to fit the AirFit N30i



Pull the frame and headgear over the head. Tighten or loosen the headgear until the cushion fits comfortably. Feed longer hair through the back of the frame and over the top of the headgear, if needed.



Tighten or loosen the headgear until the cushion fits comfortably. Feed longer hair through the back of the frame and over the top of the headgear, if needed.



Attach the elbow to the frame. Connect the airtube from the device to the elbow, then reattach the elbow to the frame.

Ordering information

Product	Part number	HCPC	Recommended replacement schedule*
Complete mask system (includes frame, cushion and headgear) <ul style="list-style-type: none"> Starter pack, small—small/medium/small wide cushions Starter pack, standard—small/medium/small wide cushions 	63801 63800	A7034 + A7035	—
Frame system (includes frame, cushion and elbow) <ul style="list-style-type: none"> Small frame—small/medium/wide/small wide Standard frame—small/medium/wide/small wide 	63803/63805/63807/63809 63802/63804/63806/63808	A7034	Every 3 months
Cushion <ul style="list-style-type: none"> Small Small wide Medium Wide 	63813 63810 63811 63812	A7032	Every 2 weeks
Headgear—standard	63814	A7035	Every 6 months

*Replacement cycle may vary depending on patient's insurance coverage.