


AirFit™P30i

Nasal pillows mask

The ResMed AirFit P30i mask provides a nasal pillows option for people who prefer a top-of-head tube design. The SpringFit frame is designed to adapt to each patient's face.

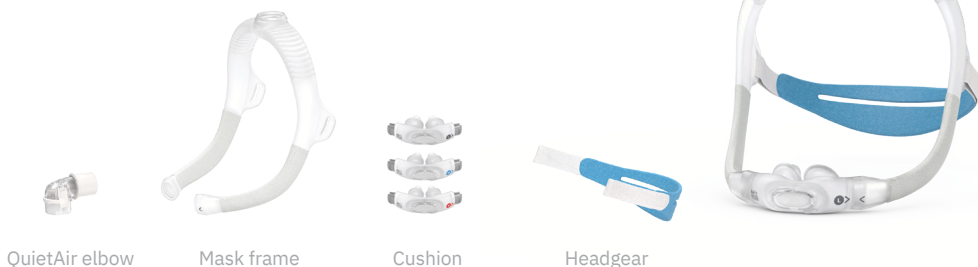

How to fit the AirFit P30i



Pull the frame and headgear over the head so the elbow sits on top and the headgear is positioned behind the head. Place the cushion under the nose and align the pillows into the nostrils.



Tighten or loosen the headgear until the cushion fits comfortably. Feed longer hair through the back of the frame and over the top of the headgear, if needed.



Squeeze the side buttons on the elbow and detach it from the frame. Connect the air tubing from the device to the elbow, then reattach the elbow to the frame.

Ordering information

Product	Part number	HCPC	Recommended replacement schedule*
Complete mask system (includes frame system, headgear and additional cushion sizes) <ul style="list-style-type: none"> • Starter pack, small—small/medium/large pillows • Starter pack, standard—small/medium/large pillows 	63851 63850	A7034 + A7035	—
Frame system (includes frame, cushion and elbow) <ul style="list-style-type: none"> • Small frame—small/medium/large • Standard frame—small/medium/large 	63857/63855/63853 63856/63854/63852	A7034	Every 3 months
Cushion <ul style="list-style-type: none"> • Small • Medium • Large 	63861 63862 63863	A7033	Every 2 weeks
Headgear—standard Replacement cycle may vary depending on patient's insurance coverage.	63814	A7035	Every 6 months