

# AirTouch™ F20 | AirTouch F20 for Her Full face mask

The ResMed AirTouch F20\* features the ResMed UltraSoft™ memory foam cushion that conforms to the unique contours of each face and is designed to deliver a light, breathable seal across a range of therapy pressures.

## Systems and Components



QuietAir elbow

Mask frame

Cushion

Headgear

### How to fit the AirTouch F20



Place cushion over the nose and against the face. With the ResMed logo facing up, pull headgear over head.



Bring the lower straps under the ears and reattach to the frame with the magnetic clips.\* Feed longer hair through the back of the frame, if needed.



Undo the fastening tabs on the upper headgear straps and pull evenly for a snug fit. Repeat with lower headgear straps.

### Ordering information

Product	Part number	HCPC	Recommended replacement schedule†
Complete mask system (includes frame, cushion, elbow, soft sleeves and headgear) • AirTouch F20—small/medium/large • AirTouch F20 for Her—small/medium	63000/63001/63002 63003/63004	A7030 + A7035	—
Frame system (includes frame, soft sleeves, cushion and elbow) • AirTouch F20—small/medium/large • AirTouch F20 for Her—small/medium	63021/63022/63023 63024/63025	A7030	Every 3 months
Cushion • Small • Medium • Large	63028 63029 63030	A7031	Monthly
Headgear • AirTouch F20—small/standard/large • AirTouch F20 for Her—small	63470/63471/63472 63473	A7035	Every 6 months

\*The mask contains magnets that may interfere with certain implants or medical devices. Please refer to the User Guide for complete information, including magnet contraindications and warnings.

†Replacement cycle may vary depending on patient's insurance coverage.