



Printing instructions

- 1 Print the sizing guide
- 2 Measure your printout

- > PRINT ON U.S. LETTER OR A4-SIZED PAPER. PLACE YOUR RULER ON THE SCALE BELOW.
- > ENSURE YOU SELECT "ACTUAL SIZE" AND THEY MEASURE THE SAME. NOT "FIT" IN THE PRINTER SETTINGS.
- > FOR BEST RESULTS, PRINT IN COLOR.

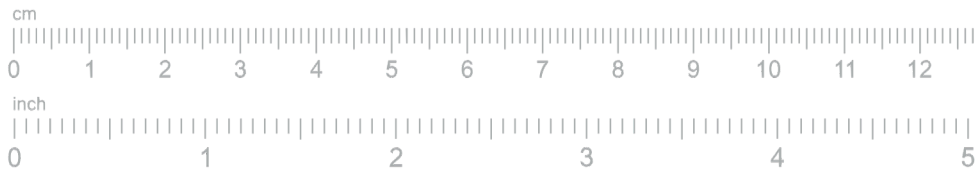

**TIP**  
Use a mirror or ask someone at home to assist you with sizing.

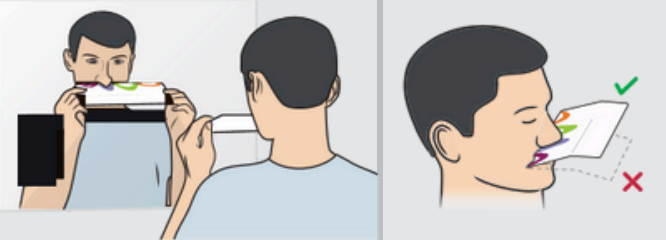


HOLD THE GUIDE SO THAT IT PRESSES THE SIZE THAT AGAINST THE UNDERSIDE OF YOUR NOSE.

Sizing guide instructions

Carefully cut along the dotted lines.

900821 REV E © 2021 Fisher & Paykel Healthcare Limited  
F&P, Evora and Solo are trademarks of Fisher & Paykel Healthcare Limited.

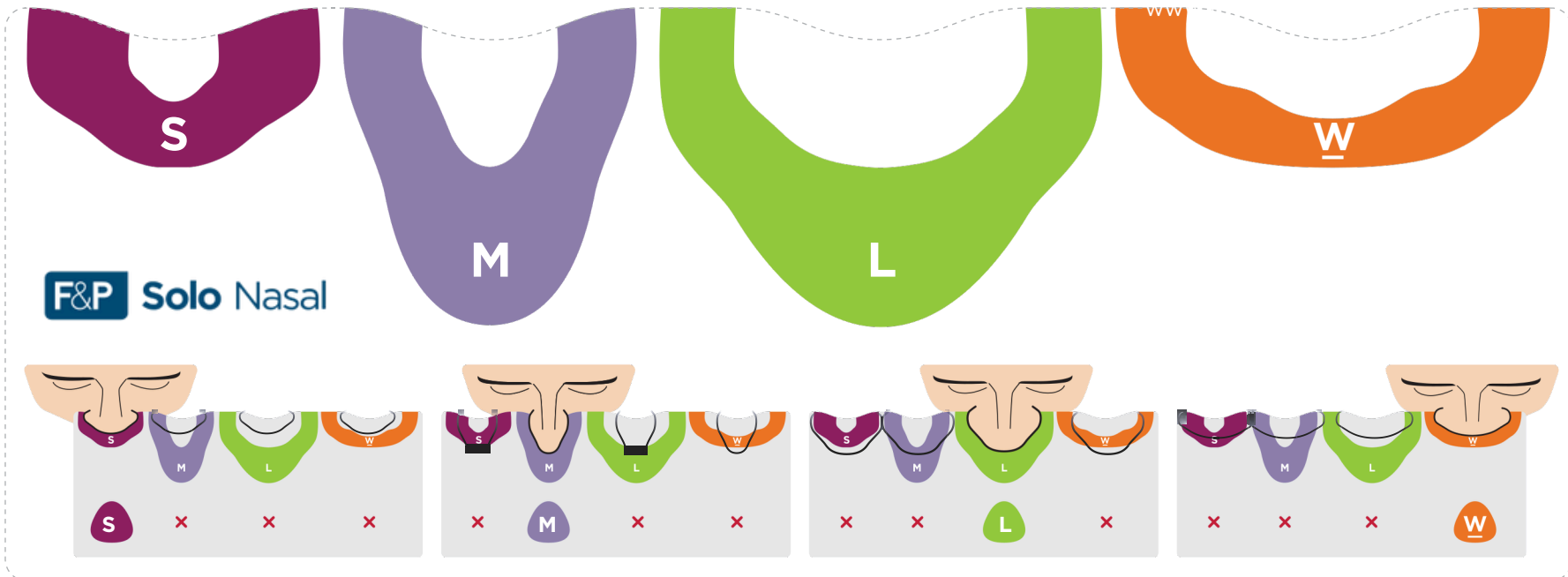

THIS PRODUCT MAY NOT BE RIGHT FOR YOU. READ THE WARNINGS AND INSTRUCTIONS FOR USE BEFORE PURCHASE. F&P SOLO™ NASAL MASK MAY NOT BE AVAILABLE IN ALL MARKETS.

F&P AND F&P SOLO ARE TRADEMARKS OF FISHER & PAYKEL HEALTHCARE LIMITED. FOR PATENT INFORMATION, SEE WWW.FPHCARE.COM/IP. REF 626976 REV A 2022-02 © 2022 FISHER & PAYKEL HEALTHCARE LIMITED.