

Consumable parts

Please refer to *Section 4, "Cleaning and maintenance"*, of your myAIRVO 2 User Manual for information on consumable parts. myAIRVO 2 has four parts which you will need to replace regularly to keep your device as clean as possible. The times recommended below assume your myAIRVO 2 is only being used by one person, and rely on the correct cleaning procedures being followed.



<div>30</div> <div>Optiflow™+ tracheostomy interface</div> <div>Replace every 30 days. Product code: OPT970E</div>	<div>Reorder code:</div> <div></div>	
<div>60</div> <div>Heated breathing tube</div> <div>Replace every 60 days. Product code: 900PT560E (top right) 900PT500E (bottom right)</div>	<div>Reorder code:</div> <div></div>	
<div>90</div> <div>Air filter</div> <div>Replace every 90 days or 1000 hours. Product code: 900PT913 (2-pack)</div>	<div>Reorder code:</div> <div></div>	
<div>N/A</div> <div>HC360 reusable water chamber</div> <div>Product code: HC360</div>	<div>Reorder code:</div> <div></div>	

Some products may not be available in your country. Please contact your local Fisher and Paykel Healthcare representative.

Contact details for support and ordering

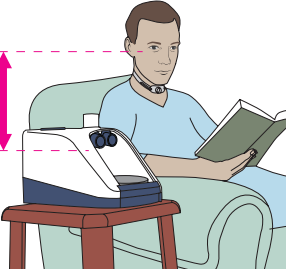
www.fphcare.com/myairvo

Important note: This is a Quick Start Guide only and should not be considered in any way a replacement for your myAIRVO 2 User Manual. Before using your myAIRVO 2 device you must read your myAIRVO 2 User Manual to understand the device's intended use, warnings and cautions. myAIRVO 2 is not intended for life support.

Setup steps for your myAIRVO 2 with the HC360 reusable water chamber


Please refer to *Section 2, "Setting up myAIRVO 2"*, and *Section 3, "Using myAIRVO 2"*, of your myAIRVO 2 User Manual for information on setup.

1




Place your myAIRVO 2 on a low shelf or near the floor beside your bed. Make sure it is placed below your head height and flat.

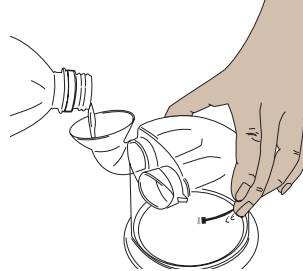2




With the aid of the supplied funnel, fill the chamber with enough distilled water for the period of use, but never above the 560 mL maximum water level line.

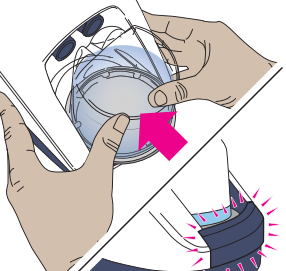3




Fit the water chamber onto your device by depressing the finger guard, pushing forward until the finger guard clicks into place.

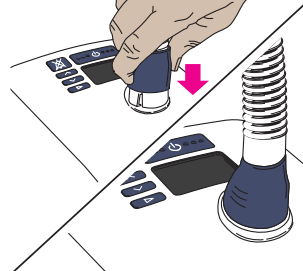4




Attach the heated breathing tube to your myAIRVO 2.

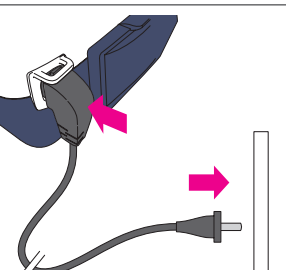5



Plug myAIRVO 2's power cord into the mains/utility power socket and into the rear of the unit.

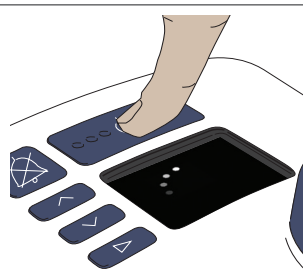
6



Switch on myAIRVO 2 by pressing and holding the On/Off button for 5 seconds.

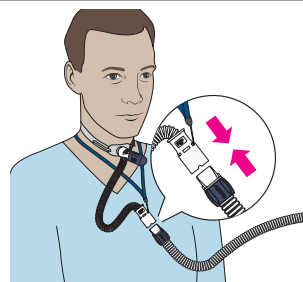
myAIRVO 2 will begin to warm up, displaying a warm-up symbol on the screen.

7



When myAIRVO 2 is ready for you to use, a tick symbol will be displayed.

8







Connect your interface and begin therapy. During use, if excess condensate forms in the tube, refer to Condensate Management in *Section 2, "Setting up myAIRVO 2"*.

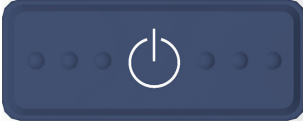REF 614389 REV A November 2017 © Fisher & Paykel Healthcare Limited

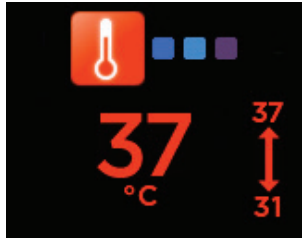
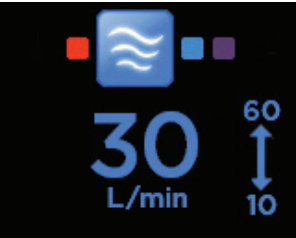

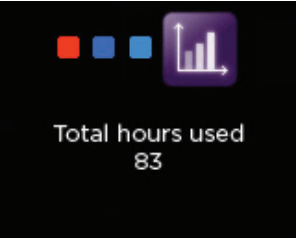
Understanding myAIRVO 2 buttons

Please refer to *Section 3, "Using myAIRVO 2"*, of your myAIRVO 2 User Manual for information on how to operate your myAIRVO 2.

 <p>On/Off button The On/Off button turns myAIRVO 2 on when held for 5 seconds. After use, press for 3 seconds to begin Drying Mode. Your device will turn off automatically after 99 minutes.</p>	 <p>Audio Pause button The Audio Pause button allows you to temporarily silence audio alarms.</p>
 <p>Mode button The Mode button allows you to cycle between modes:<ul style="list-style-type: none">• Temperature• Flow Rate• Day/Night• Compliance</p>	 <p>Up and Down buttons The Up and Down buttons allow you to change settings within the following modes:<ul style="list-style-type: none">• Temperature• Flow Rate• Day/Night</p>




Understanding myAIRVO 2 modes and settings in default mode

Please refer to *Section 3, "Using myAIRVO 2"*, of your myAIRVO 2 User Manual for information on how to operate your myAIRVO 2. Your myAIRVO 2 will remember your settings when switched off.

 <p>Temperature Mode Please refer to Target Dew-Point Temperature in <i>Section 3, "Using myAIRVO 2"</i>.</p>	 <p>Flow Rate Mode Use the Up and Down buttons to select your flow rate from 10-60 L/min. Please see your myAIRVO 2 User Manual for information about flow ranges for your interface.</p>
 <p>Day/Night Mode Use the Up and Down buttons to cycle between Day Mode and Night Mode. In Night Mode some audio signals will become quieter and your display screen will dim.</p>	 <p>Compliance Mode Compliance Mode automatically cycles between device usage statistics:<ul style="list-style-type: none">• Total hours used• Hours per day• Checksum (for your clinician)</p>

myAIRVO 2 visual and auditory alarms


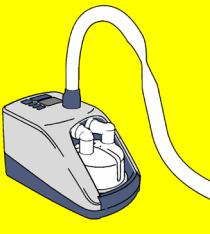

Please refer to *Section 3, "Using myAIRVO 2"*, of your myAIRVO 2 User Manual for information on alarms. Your myAIRVO 2 has alarms to warn you about interruptions to your treatment. You can use the Audio Pause button to silence the alarm for 115 seconds.

<p>Audio Pause button</p> 	 <p>Check for blockages</p>	
--	---	---

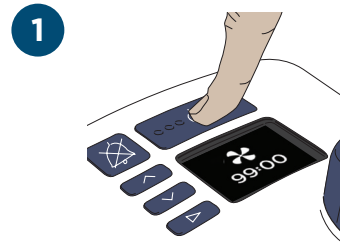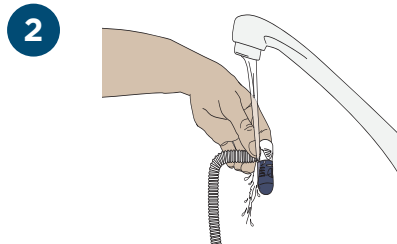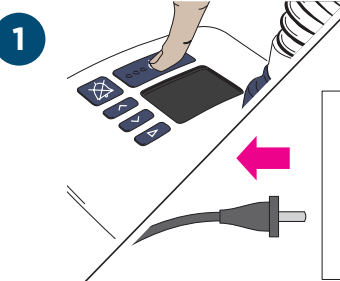
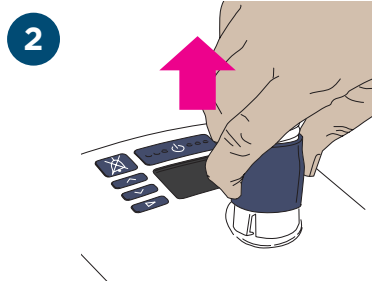
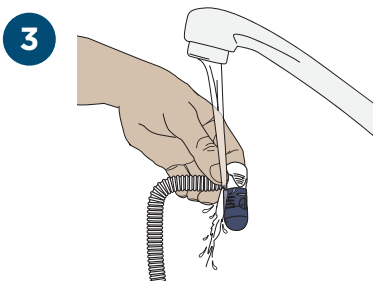
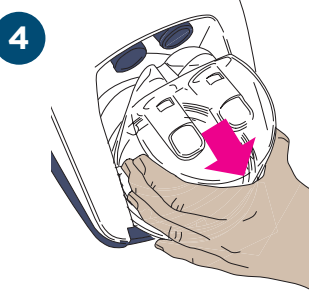

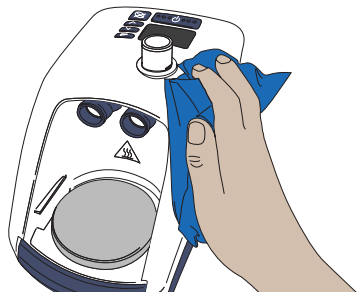
Cleaning myAIRVO 2

Please refer to *Section 3, "Using myAIRVO 2"*, and *Section 4, "Cleaning and maintenance"*, of your myAIRVO 2 User Manual for information on how to clean myAIRVO 2.

Daily cleaning

 <p>1 After use, remove your interface. To begin the 99-minute Drying Mode, press and hold the On/Off button for 3 seconds. To avoid burns, do not wear the interface during Drying Mode and do not remove the water chamber until Drying Mode is complete.</p>	 <p>2 Rinse the interface in drinking-quality water then reconnect to the heated breathing tube whilst still in Drying Mode to dry the interface.</p>	 <p>3 After Drying Mode is complete, remove the water chamber by pushing down the finger guard and pulling out the chamber. Wash and rinse the chamber and then refill as per set up steps on previous page.</p>
--	--	---

Weekly cleaning

 <p>1 Switch off the unit and unplug from the mains/utility power socket.</p>	 <p>2 Remove the heated breathing tube and drain any excess condensate.</p>	 <p>3 Remove the interface from the heated breathing tube, wash it in warm water with mild dishwashing detergent added, rinse it in drinking-quality water, then reconnect it to the heated breathing tube.</p>
 <p>4 Remove water chamber. Pour out and discard the remaining water. Remove chamber base. Wash the chamber top and base in mild dishwashing detergent then rinse. Soak the chamber in a solution of vinegar (1 part) and water (2 parts) for 10 minutes. Rinse and dry.</p>		 <p>5 Thoroughly wipe the inside of the heated breathing tube connection port with a clean, low-lint cloth dipped in warm water with mild dishwashing detergent added. Wipe the exterior of the unit with a clean damp (not wet) cloth dipped in warm water with mild dishwashing detergent added.</p>

When cleaning is complete, set up your myAIRVO 2 using the setup steps previously described.