



# Sizing guide

## Instructions:


- 1 Print out this Adobe PDF File at 100% on your printer. Use the print accuracy check at right to ensure printing at the proper size. Fold the bottom part of this page on the dashed line. Cut along the dotted line around the fitting gauge.
- 2 Place the fitting gauge on your nose as shown in the picture below. Determine the appropriate size by which area your nose reaches. (S - Small, M - Medium or L - Large)
- 3
- 4
- 5

## Print accuracy check



Philips Respironics always recommends consulting a physician or registered Sleep Therapist in fitting a mask to ensure a proper sizing and fit.

## Amara View

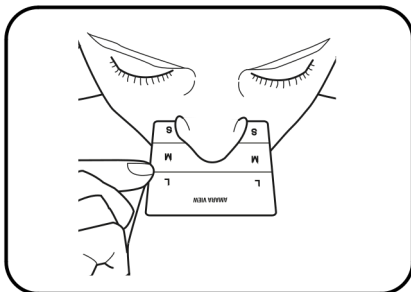

FOLD along dashed line

## Fitting gauge

CUT along dotted line

