




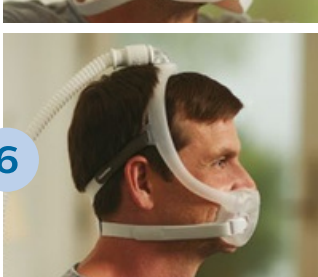


DreamWearpatient fitting guide

Follow the instructions to get started fitting your full face, nasal or gel pillows mask

Full face



Using the full face sizing gauge, hold horizontally under the nose and select the cushion size based on where the outer edge of your nostrils and tip of your nose contact the gauge. Note: Four cushion sizes (small, medium, large, medium-wide) are available.

	<p>1 The cushion opening will be directly under the nostrils. The cushion will hug the nose and mouth, and leak will be minimal. Note: At no time should the nose be inside of the cushion opening. If any part of the cushion overlaps the nose, the cushion is being worn incorrectly. A different size cushion may be needed.</p>
	<p>2 With the mask assembled, place the cushion under the nose.</p>
	<p>3 Position the frame on the top of the head. Pull the headgear over the back of the head.</p>
	<p>4 Attach the magnetic headgear clips to the mask cushion.</p>
	<p>5 To adjust the mask, peel the headgear tabs away from the fabric. Adjust the length of the top straps. Press the tabs back against the fabric to reattach. Repeat this step with the bottom straps. Note: Do not overtighten the headgear.</p>
	<p>6 Position the mask until it fits comfortably. When finished, the elbow should rest at the top of the head. Note: Please read the Instructions for use for further fitting information.</p>

Nasal

Using the nasal cushion sizing gauge (included with the mask), hold it horizontally under the nose. Select the cushion size based on where the outer edge of the nostrils and tip of the nose contact the gauge.

Note: Four cushion sizes (small, medium, large, medium-wide) are available.

	<p>1 The cushion opening will be directly under the nostrils. The cushion will hug the nose and leak will be minimal. Note: At no time should the nose be inside of the cushion opening. If any part of the cushion overlaps the nose, the cushion is being worn incorrectly.</p>	
	<p>2 With the mask assembled, place the cushion under the nose.</p>	
	<p>3 Position the frame on the top of the head. Pull the headgear over the back of the head.</p>	
	<p>4 To adjust the mask, peel the headgear tabs away from the fabric. Adjust the length of the straps. Press the tabs back against the fabric to reattach. Note: Do not overtighten the headgear.</p>	
	<p>5 Position the mask until it fits comfortably.</p>	
	<p>6 When finished, the elbow should rest at the top of the head. Note: Please read the Instructions for Use for further fitting information.</p>	

Gel pillows

Using the gel pillows sizing gauge, place the pillows cushion tips into the nostrils. Choose the cushion size that allows the blue gel bases to sit comfortably under the nose without any gaps.

Note: Three cushion sizes (small, medium, large) are available.

Making it simple to find the right fit

Achieving the right fit with DreamWear is easy

Medium frame sizing

The medium mask frame will comfortably fit most faces. If the medium frame does not fit a patient's face, a small or large mask frame might better suit your patient's needs.

Medium correct frame → Proper fit

Large frame sizing

Mask too small → Large correct frame → Proper fit

Small frame sizing

Mask too large → Small correct frame → Proper fit

Ordering information

DreamWear standard and fitpak configurations	Nasal part number	Gel pillows part number	Full face part number
DreamWear with headgear, sm and med frames, small cushion	N/A	N/A	1133375
DreamWear with headgear, sm and med frames, medium cushion	N/A	N/A	1133376
DreamWear with headgear, sm and med frames, large cushion	N/A	N/A	1133377
DreamWear with headgear, sm and med frames, med-wide cushion	N/A	N/A	1133378
DreamWear with headgear, med frame, all cushion sizes	111 6 70 0	1124984	1133 4 0 0
DreamWear without headgear, med frame, all cushion sizes	111 6 701	1124985	N/A
DreamWear mask without headgear, medium frame	Nasal part number	Gel pillows part number	Full face part number
Small cushion	111 6 705	1124986	1133 4 05
Medium cushion	111 6 70 6	1124987	1133 4 0 6
Large cushion	111 6 707	1125008	1133 4 07
Medium-wide cushion*	111 6 70 8	N/A	1133 4 0 8
DreamWear mask without headgear, small frame	Nasal part number	Gel pillows part number	Full face part number
Small cushion	1116710	1125009	1133 410
Medium cushion	111 6 7 11	112 5 01 0	1133 411
Large cushion	111 6 7 12	112 5 011	1133 412
Medium-wide cushion*	111 6 7 13	N/A	1133 413
DreamWear mask without headgear, large frame	Nasal part number	Gel pillows part number	Full face part number
Small cushion	1116716	112 5 013	1133 416
Medium cushion	1116717	112 5 014	1133 417
Large cushion	1116718	N/A	1133 41 8
Medium-wide cushion*	Nasal part number	Gel pillows part number	Full face part number
DreamWear mask with headgear, small frame	1116685	112 5 01 8	1133385
Small cushion	1116686	112 5 019	11333 8 6
Medium cushion	1116687	112 5 02 0	1133387
Large cushion	1116688	N/A	11333 8 8
Medium-wide cushion*	Nasal part number	Gel pillows part number	Full face part number
DreamWear mask with headgear, medium frame	1116680	112 5 015	11333 8 0
Small cushion	1116681	112 5 01 6	1133381
Medium cushion	1116682	112 5 017	1133382
Large cushion	1116683	N/A	1133383
Medium-wide cushion*	Nasal part number	Gel pillows part number	Full face part number
DreamWear mask with headgear, large frame	111 6 69 0	112 5 02 1	1133390
Small cushion	111 6 691	112 5 02 2	1133391
Medium cushion	111 6 692	112 5 02 3	1133392
Large cushion	111 6 693	N/A	1133393
Medium-wide cushion*	Nasal part number	Gel pillows part number	Full face part number
DreamWear mask with headgear, large frame	111 6 692	112 5 02 3	1133392
Small cushion	111 6 693	N/A	1133393
Medium cushion		Gel pillows part number	Full face part number
Large cushion		112 5 03 5	1133 43 0
Medium-wide cushion*		112 5 03 6	1133 431
		112 5 03 7	1133 432
Replacement parts	Nasal part number		
Small cushion Medium	111 6 74 0	N/A	1133 433
cushion Large cushion	111 6 741	1116745	1116745
Medium-wide cushion*	111 6 742	111 6 74 6	111 6 74 6
Small frame Medium	111 6 743	111 6 747	111 6 747
frame Large frame	1116745	1116750	1133 45 0
Headgear Headgear	111 6 74 6	N/A	1133 45 4
magnetic clips Fabric	111 6 747	1116754	1116754
wraps Sizing gauge	1116750	112 5 03 8	1116752
	N/A		
	1116754		
	1116752		

*A medium-wide cushion isn't available for gel pillows.