

## AirFit™ F40

Full face mask

The ResMed AirFit F40\* is an ultra-compact, full face mask that combines high seal performance with comfort in a flexible, easy-to-fit design.

### Systems and Components



Quick release elbow

Mask frame



Cushion




Headgear



### How to fit the AirFit F40



Unclip lower straps away from frame and place cushion under nose to fit against the face. With the ResMed logo facing up, pull headgear over head.

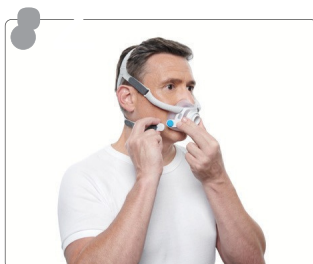

Bring the lower straps under the ears and reattach to the frame with the magnetic clips.\* Feed longer hair through the back of the frame, if needed.



Adjust the upper straps and then the lower straps for a snug fit.

### Ordering information

Product	Part number	HCPC	Recommended replacement schedule†
Complete mask system (includes frame, cushion and headgear) <ul style="list-style-type: none"> <li>• Small frame—medium cushion</li> <li>• Standard frame—small wide/medium/large cushion</li> <li>• Large frame—medium cushion</li> </ul>	64603 64600/64601/64602 64604	A7030 + A7035	—
Frame system (includes standard frame and cushion) <ul style="list-style-type: none"> <li>• Small wide</li> <li>• Medium</li> <li>• Large</li> </ul>	64607 64608 64609	A7030	Every 3 months
Cushion <ul style="list-style-type: none"> <li>• Small wide</li> <li>• Medium</li> <li>• Large</li> </ul>	64627 64628 64629	A7031	Monthly
Headgear <ul style="list-style-type: none"> <li>• Small</li> <li>• Standard</li> <li>• Large</li> </ul>	64633 64634 64635	A7035	Every 6 months

\*The mask contains magnets that may interfere with certain implants or medical devices. Please refer to the User Guide for complete information, including magnet contraindications and warnings.

†Replacement cycle may vary depending on patient's insurance coverage.