


AirTouch™ N20 | AirTouch N20 for Her

Nasal mask

UltraSoft™ memory foam cushion to provide patients with a unique comfort experience.

Systems and Components

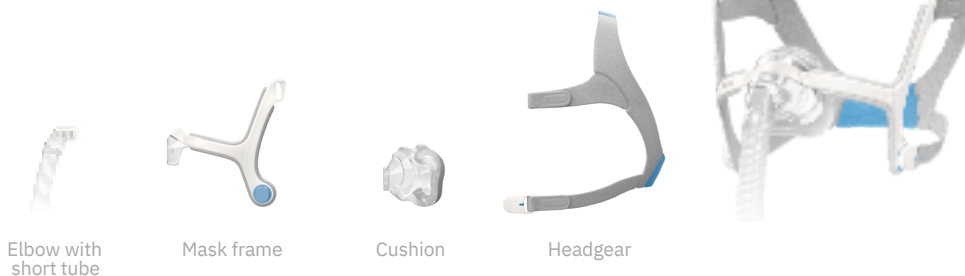

How to fit the AirTouch N20



Place cushion over the nose and against the face. With the ResMed logo facing up, pull headgear over head.



Bring the lower straps under the ears and reattach to the frame with the magnetic clips.* Feed longer hair through the back of the frame, if needed.



Undo the fastening tabs on the upper headgear straps and pull evenly for a snug fit. Repeat with lower headgear straps.

Ordering information

Product	Part number	HCPC	Recommended replacement schedule†
Complete mask system (includes frame, cushion, elbow with short tube and headgear with clips) <ul style="list-style-type: none"> AirTouch N20—small/medium/large AirTouch N20 for Her—small 	63903/63901/63902 63900	A7034 + A7035	—
Frame system (includes frame, cushion and elbow with short tube) <ul style="list-style-type: none"> AirTouch N20—small/medium/large AirTouch N20 for Her—small 	63953/63955/63956 63954	A7034	Every 3 months
Cushion <ul style="list-style-type: none"> Small Medium Large 	63950 63951 63952	A7032	Every 2 weeks
Headgear <ul style="list-style-type: none"> AirTouch N20—small/standard/large AirTouch N20 for Her—small 	63560/63561/63562 63558	A7035	Every 6 months

*The mask contains magnets that may interfere with certain implants or medical devices. Please refer to the User Guide for complete information, including magnet contraindications and warnings.

†Replacement cycle may vary depending on patient's insurance coverage.