

Fitting Guide

Note: Prior to fitting your mask, remove the **orange Spring frame support** from the headgear and discard.



- With both lower headgear clips released, position the cushion at an angle on the nasal bridge.



- Keeping the upper position of the mask in place, swing the lower half of the mask down against the face.



- Hold the mask firmly against the face and pull the headgear over the head.



- Bringing the lower straps below the ears, clip the headgear into the lower part of the spring frame.



- Unfasten the Velcro®, pull the upper headgear straps evenly until they are comfortable and firm, and reattach the Velcro.



- Repeat step 5 with the lower headgear straps.



- Connect one end of the air tubing to the swivel and the other end to the airflow device.
- Turn on the airflow device.

Tip One



- Ensure the headgear is correctly positioned on the head.

Tip Two




- Ensure the mask sits mid-bridge and does not encroach on the eyes.

Quattro™ FX
FULL FACE MASK

www.quattrofx.com

ResMed Corp San Diego, CA, USA +1 858 836 5000 or 1 800 424 0737 (toll free). ResMed Ltd Bella Vista, NSW, Australia +61 (2) 8884 1000 or 1 800 658 189 (toll free). See ResMed.com for other ResMed locations worldwide. Spring Air is a trademark of ResMed Ltd. Quattro is a trademark of ResMed Ltd and is registered in the U.S. Patent and Trademark Office. ©2010 ResMed. Specifications may change without notice. 1014322/1 10 11

Less mask. **More you.**

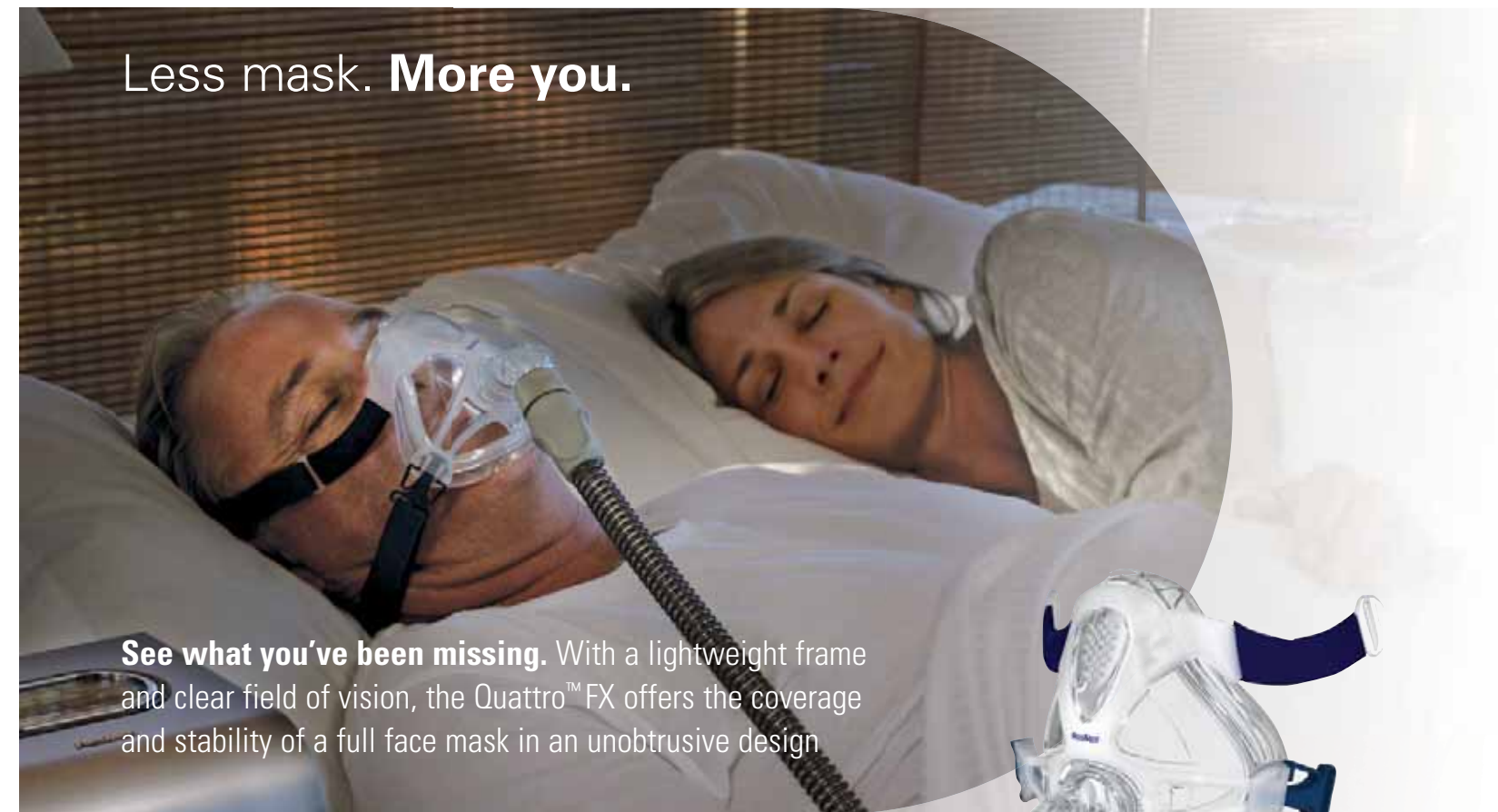

See what you've been missing. With a lightweight frame and clear field of vision, the Quattro™ FX offers the coverage and stability of a full face mask in an unobtrusive design



With Quattro FX, therapy has never been easier

Benefits for you ...

Time and cost savings

Quattro FX gives you the tools to achieve the best fit for your patients—the first time, and every time.

New, intuitive features make fitting the Quattro FX a quick and straightforward process, saving you time, money and additional patient visits.

It's as easy as size, swing and seal

The custom-made sizing tool and Swing Seal fitting technique make it easy to achieve a comfortable, effective seal. The suspension-like combination of the Spring frame and new headgear design eliminates the need for forehead support.

With a strong seal and fewer points to adjust, fitting and wearing a full face mask has never been easier.

Therapy acceptance

The comfort, performance and feeling of openness that Quattro FX provides, help users accept therapy early on and stay compliant in the long run.

... and your patients

Stable performance

New technologies in the Spring Air™ cushion and Spring frame give the stability and adjustment provided by a conventional forehead support.

Freedom to be

Fewer points of contact with the face reduce the chances of skin irritation; a clear line of sight helps break down the physical and emotional barriers associated with full face therapy.

A good night's sleep

The high-tech Spring Air cushion distributes pressure evenly and absorbs even the slightest user movements, so users can enjoy a good night's sleep, confident that their seal is safe.

ResMed quality

The Quattro FX offers ResMed's superior comfort and quality you know you can trust.

Quattro™ FX
FULL FACE MASK

www.quattrofx.com

Technology highlight

Size. Swing. Seal.
Sizing, fitting and sealing Quattro FX is fast and straightforward.

1 Size using custom-made sizing tool

2 Position the cushion at an angle on the nasal bridge ...

3 ... then **swing** the lower half of the mask down against the face to **seal**.

Quattro FX: fits right, seals securely — first time, every time.

The **Quattro FX** provides ResMed quality and full face comfort in a lightweight, unobtrusive mask.

Clear line of sight
Creates a feeling of openness; a smaller frame also makes side sleeping more comfortable

Built-in vent design
Disperses air gently and quietly

Flexible, durable Spring frame
Strong enough to stabilize the mask on the face, soft enough to accommodate user movement during sleep

360° elbow rotation
Positions the tubing for ease of use

Swivel
Offers a convenient way to connect and disconnect air tubing



New, tension-reducing headgear design
Features a contoured crown strap that sits snugly at the top of the head and avoids the sensitive area around the back of the neck



Contoured cushion membrane
gently rests against sensitive nasal bridge area and inflates to gently meet the nose rather than stretch over it

Dual-wall Spring Air cushion
Provides soft, even pressure distribution and inner cushion wall cut-out maintains seal during jaw drop

Set-and-forget headgear clips
Easy to grip for attaching and detaching; retain desired headgear settings



PRODUCT CODES

Quattro FX Full Face Mask Complete System

Size	Part Number
Small	61700
Medium	61701
Large	61702

*Medium headgear included with all mask sizes

REIMBURSEMENT CODES

HCPCS	Code Descriptor
A7030	Full face mask used with positive airway pressure device, 1 per 3 months
A7031	Replacement cushion for full face mask, 1 per month
A7035	Headgear used with positive airway pressure device, 1 per 6 months

For details, refer to the **Quattro FX Full Face Mask User Guide**.

