

# Provider setup

Philips Respironics Sleep Therapy System

## Step 1: Set up device

To access provider screen:

1. In patient screen, highlight **Setup**.
2. Press and hold ramp button and control wheel simultaneously for 4-5 seconds until beep.
3. Provider screen then appears. Highlight **Setup** again and press control wheel.

To program device:


1. Turn control wheel to specific setting and press.
2. Turn control wheel to desired option and press control wheel to lock.
3. Turn control wheel to next setting and repeat.
4. To exit, turn control wheel to **Back** and press.

Menu options will vary based on device model being used. Refer to provider manual for complete description of each setting and option.

Setting	Option	
<b>Back</b>	<b>Auto</b>	<b>CPAP</b>
hPa/cmH2O	hPa	cmH2O
<b>Auto max</b>	(auto min)-20.0	
<b>Auto min</b>	4.0-(auto max)	
<b>Flex type</b>	None	C-Flex A-Flex
<b>Flex</b>	1	2 3
<b>Flex lock</b>	on	off
<b>SYSTEM ONE resistance</b>	0	X1 X2 X3 X4 X5
<b>Lock SYSTEM ONE</b>	on	off
<b>Ramp time</b>	0:00-0:45	
<b>Ramp start</b>	4.0-(auto min)	
<b>Humidifier</b>	0	1 2 3 4 5
<b>Auto on</b>	on	off
<b>Auto off</b>	on	off
<b>Mask alert</b>	on	off
<b>LED backlight</b>	on	off
<b>Show AHI/leak/PB</b>	on	off
<b>Split night</b>	on	off
<b>Split night start</b>		
<b>GMT offset</b>	-12	-12
<b>ENGLISH</b>	1	0
<b>Back</b>		

Note: the screen will automatically reset to patient mode after 60 seconds of inactivity.

Provider screen

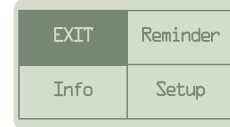


Patient screen



Provider screen




Provider screen

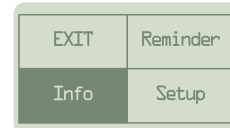

## Step 2: Verify setup

1. Turn control wheel to highlight **Exit** and press. You are now in patient screen.
2. Turn control wheel to **Therapy** and press.
3. Pressure will be displayed and device will start. (Auto CPAP will display min pressure)

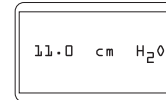

## Step 3: Set reminders

Return device to provider screen, then:

1. Turn control wheel to **Reminder** and press.
2. Turn control wheel to desired option and press.

<b>Back</b>			
<b>Reminder off</b>	30	90	
	180	270	365

3. Turn control wheel to **Back** and press.


## Step 4: View information

In provider screen:


1. Turn control wheel to **Info** and press.
2. Turn control wheel to desired option and press to view information as identified.
3. To exit out of any option, turn control wheel to **Back** and press.

Option	Data display	
<b>Back</b>		
<b>Phone-in</b>		
<b>Compliance VIC</b>		
<b>Therapy hours</b>	Therapy hours	AHI
	7 day avg: 0:00	7 day avg: 0:0
	30 day avg: 0:00	30 day avg: 0:0
	Back	Back
<b>Blower hours</b>	Blower hours	Periodic breathing
	Total hours: 0:0	7 day avg: 0%
	Back	30 day avg: 0%
	Back	Back
<b>System leak</b>	Days > 4	90% pressure
	Days > 4	7 day avg: 0:0
	7 days: 0	30 day avg: 0:0
	30 days: 0	Back
	Back	Back
<b>Periodic breathing</b>		
<b>Reset data</b>		
<b>Machine hours</b>	Leak	Machine hours
	System leak	Total hours: 0:0
	7 day avg: 0%	Back
	30 day avg: 0%	Back
	Back	

Patient screen

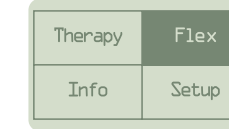

Provider screen

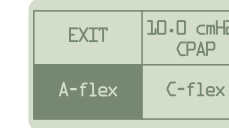

Waveform




Note: Flex options vary among models.

## Step 5: Flex demo

1. From the patient screen, highlight **Flex**.
2. Press and hold ramp button and control wheel simultaneously for 4 to 5 seconds until beep. You are now in provider screen.
3. New screen appears with **CPAP** and **Flex** modes in each quadrant.
4. Turn control wheel to the specific mode you want to demo and press.
5. Device will auto start blower and deliver the selected mode. The screen will automatically cycle to display the waveform.
6. After 10 minutes of no action, device resets to patient screen and blower turns off.

## Step 6: Undocking

Refer to diagram below. Hold entire unit at each end and slide release latch to the right.

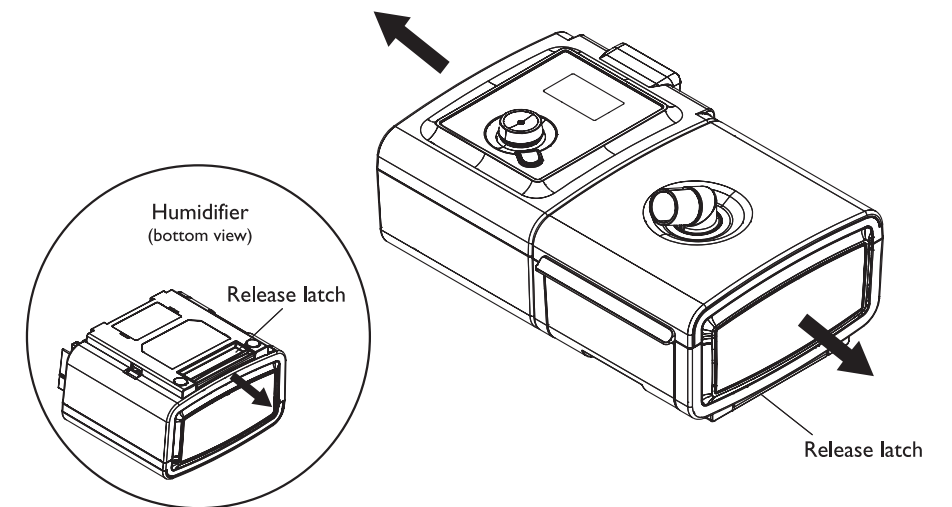


# Patient setup

Philips Respironics Sleep Therapy System

## Step 1: Humidification

With device turned off, remove and fill chamber with water.

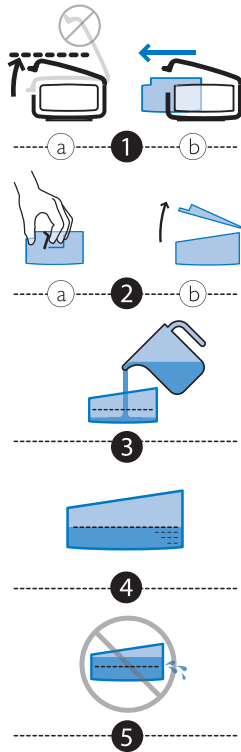


- (a) Lift humidifier door up until it locks in place. Do not force door higher than the locked position. (b) Grasp front of chamber and slide toward you.
- (a) Gently push tab located inside round opening on top of chamber. (b) Separate top and bottom of chamber.
- Fill bottom chamber with **distilled** water.
- Observe maximum fill lines located on sides of chamber.
- Do not overfill.**

Reassemble the chamber by placing the hinges on lid over the two tabs on back of chamber. Close lid until it locks in place with an audible click. Insert filled chamber into humidifier.


### Humidifier settings

- When blower is activated, the humidification icon and numerical setting will light up on the device.
- Turn control wheel to adjust humidifier number to the desired setting (0 thru 5) and press. The settings represent relative humidity levels that will remain consistent throughout the night even if room temperature changes.

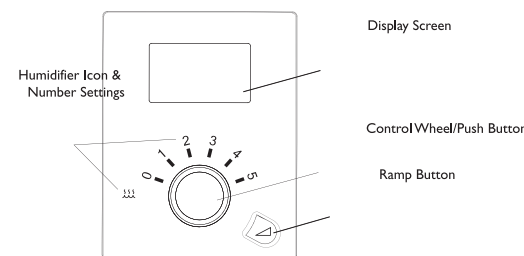


Note: refer to step 3 of this patient setup guide to set the humidifier icon and setting lights to dim or turn off after 30 seconds.


## Step 2: Activate device

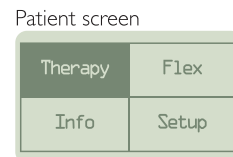

- Connect power source into power outlet on back of device. Plug power cord into power source and wall outlet.
- Turn control wheel to **Therapy** and press to start blower.
- Screen will automatically change to pressure setting.

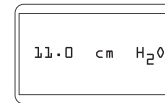

## Step 3: Set up/change device settings

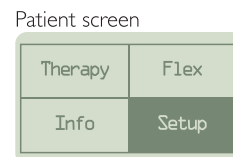


- Turn control wheel to **Setup** and press.
- A menu of settings with associated options is displayed.
- Turn control wheel to desired setting and press. The option can now be changed by turning the control wheel. Press again to lock option.
- Turn control wheel to next setting and repeat previous actions.
- Turn control wheel to **Back** and press to exit.

Setting	Option
Back	
Flex	0 1 2 3
Humidifier	0 1 2 3 4 5
Ramp start	4.0-(auto min)
Mask resistance	1 2 3 4 5
Auto on	on off
Auto off	on off
Mask alert	on off
LED backlight	on off
Back	

Note: the setup menu on your device may look slightly different depending on the device model you are using.

This menu identifies settings and options available to change. Refer to your user manual for specific information on each.

## Step 4: View information

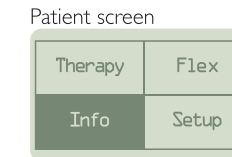


- Turn control wheel to **Info** and press. A list of information topics is displayed.
- Turn control wheel to desired topic and press. A new screen will display specific information. Your homecare provider may want you to read this information to them.

Note: this menu identifies information that may be provided by your device.



Option	Data display
Back	Status
Status	Wireless: good
Phone-in	Back
Compliance VIC	Therapy hours
Therapy hours	7 day avg: 0:00
Days > 4	30 day avg: 0:00
Back	Back
Back	Days > 4
	7 days: 0
	30 days: 0
	Back

- Turn control wheel to **Back** and press to exit.

## Step 5: Flex settings

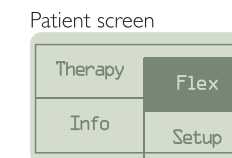

- Turn control wheel to **Flex** and press.
- Turn control wheel to **Flex demo** and press.
- The blower will automatically activate and provide pressure while you cycle through the three different settings.
- Press control wheel again and it will lock in setting and blower will shut off.

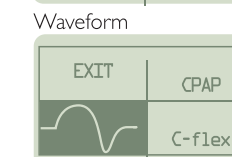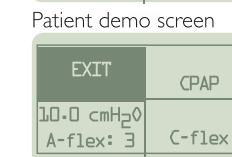

Note: Flex options may look different depending on the device model you are using.

## Step 6: Undocking

Hold entire unit at each end and slide release latch to the right. Refer to step 6 diagram on the reverse side of this setup guide.