

1 Replacing the Filter


Step 1: Remove old filter

Open the SoClean so that you can see inside the chamber. Remove the old filter inside the SoClean by pulling the filter up and out. You can discard the old filter in the garbage



Step 2: Remove protective plastic and tape

Fully remove all protective plastic and tape from the new filter. You may see some black particles on the filter or on your hands. This is normal from settling during shipping and can be wiped off.

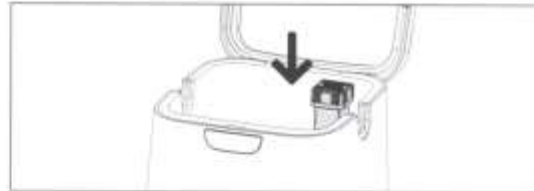


Step 3: Install new filter

Place the new filter into the SoClean.

Now see [Replacing the Check Valve](#)

2 Replacing the Check Valve

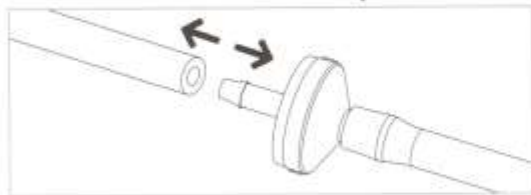


Step 1: Removal of your current check valve

Grasp the barrel of your current check valve firmly and pull the tubing away from the barrel system. It may be necessary to use a twisting motion to free the tubing from the stem. After one side of the tubing has been disconnected from the check valve, repeat to disconnect the tubing from the other side.



Step 2: Positioning your new check valve

The direction of your new check valve is critical. Your SoClean will not work if it is installed facing the wrong direction. The end of your new check valve has a label with 2 arrows that should be positioned away from your SoClean and closest to your CPAP machine.

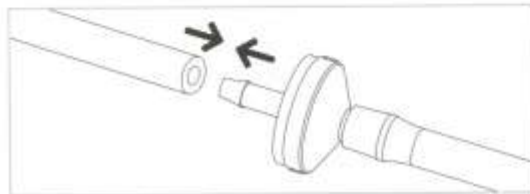

Step 3: Installing your new check valve

With your check valve facing the correct direction, grasp the barrel and insert the stem as far as possible into the end of the tubing. The tubing should be nearly touching the barrel of the check valve. A thin film of dish soap on the stem can be used to help it slide on easier.

Repeat for the remaining side of the check valve.

3 Resetting the SoClean



Perform only after replacing the Cartridge Filter and Check Valve.

Press and hold the **MANUAL** and **HOURGLASS** button simultaneously until you see a smiley face on the display. This means the reset is successful.