

AirFit™ F20 | AirFit F20 for Her

The ResMed AirFit F20* and AirFit F20 for Her* full face masks are designed to fit all patients regardless of facial profile.

Full face mask



Systems and Components



Elbow



Mask frame



Cushion



Headgear



How to fit the AirFit F20



Place cushion over the nose and against the face. With the ResMed logo facing up, pull headgear over head.



Bring the lower straps under the ears and reattach to the frame with the clips.* Feed longer hair through the back of the frame, if needed.



Undo the fastening tabs on the upper headgear straps and pull evenly for a snug fit. Repeat with lower headgear straps.

Ordering information

Product	Part number	HCPC	Recommended replacement schedule†
Complete mask system (includes frame, cushion, elbow, soft sleeves and headgear) <ul style="list-style-type: none"> AirFit F20—small/medium/large AirFit F20 for Her—small/medium AirFit F20 Non Magnetic—small/medium/large 	63400/63401/63402 63403/63404 64028/64029/64030	A7030 + A7035	—
Frame system (includes frame, soft sleeves, cushion, elbow) <ul style="list-style-type: none"> AirFit F20—small/medium/large AirFit F20 for Her—small/medium AirFit F20 Non Magnetic—small/medium/large 	63460/63461/63462 63463/63464 64045/64046/64047	A7030	Every 3 months
Cushion—small/medium/large	63467/63468/63469	A7031	Monthly
Headgear <ul style="list-style-type: none"> AirFit F20—small/standard/large AirFit F20 for Her—small AirFit F20 Non Magnetic—small/medium/large 	63470/63471/63472 63473 64040/64041/64042	A7035	Every 6 months

*The mask contains magnets that may interfere with certain implants or medical devices. Please refer to the User Guide for complete information, including magnet contraindications and warnings.

†Replacement cycle may vary depending on patient's insurance coverage.