



## Easily fits them and everyone in between

Everyone's face is different. But with our new AirFit™ 20 series, fitting a broad range of patients has never been easier. That's because each mask has been specially-designed to fit all patients regardless of facial profile, taking the constant guesswork out of mask selection. And with a recent international study showing the AirFit N20 fit 99% of patients,<sup>1</sup> you can trust that it will perform for your diverse group of patients.



[ResMed.com/AirFitN20](https://www.ResMed.com/AirFitN20)



## AirFit N20 highlights


**Plush headgear** is specifically designed for extra softness and stability.

**InfinitySeal™ cushion** features specially-designed loops and varying thickness levels designed to prevent blowout.

**Soft and flexible frame** sits underneath the eyes and adapts to different facial types, while its integrated padding provides added softness.

**Quick-release elbow** makes it simple to disconnect from the mask tubing without removing the mask.


**Magnetic clips** guide the headgear to the frame quickly and easily.

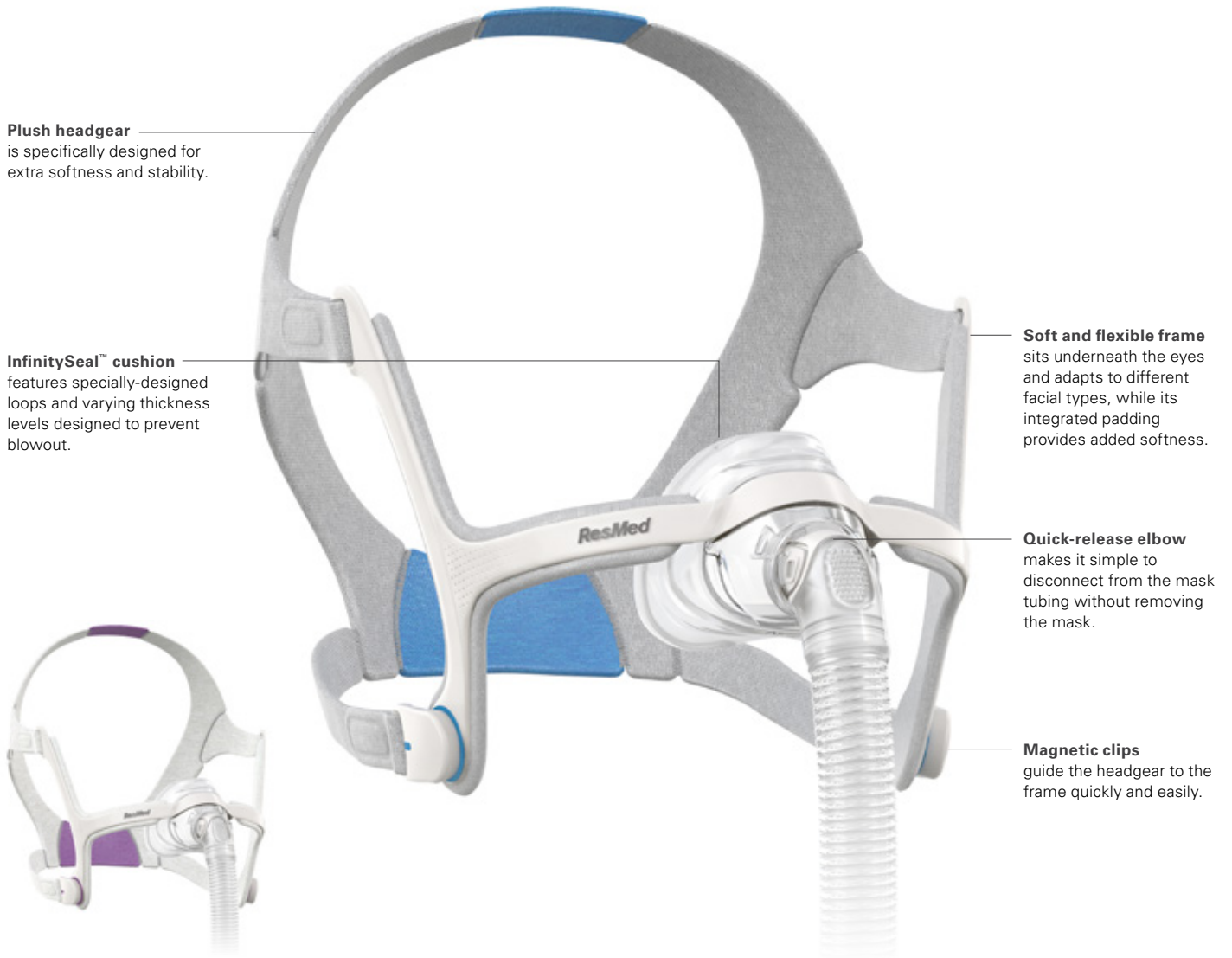

**AirFit N20 for Her**  
Designed specifically for women.

The AirFit N20 fit a remarkable 99% of patients in a recent international fitting study.<sup>1</sup>

## AirFit N20 components

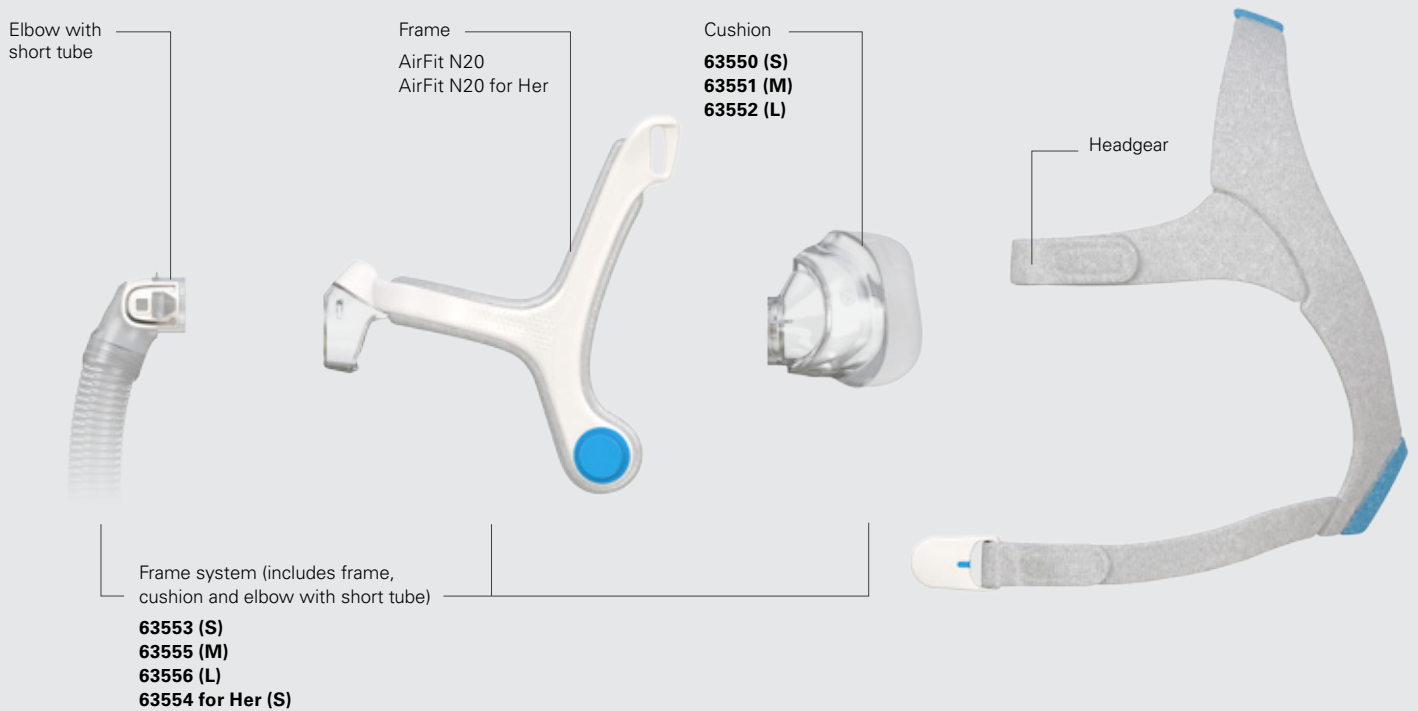

## Product codes

### Complete mask systems (Includes frame system + headgear)

AirFit N20: Small	<b>63503</b>
AirFit N20: Medium	<b>63501</b>
AirFit N20: Large	<b>63502</b>
AirFit N20 for Her	<b>63500</b>



AirFit N20 headgear with headgear clips  
**63560 (S)**  
**63561 (Std)**  
**63562 (L)**



AirFit N20 for Her headgear with headgear clips  
**63558 (S)**



## Fitting guide for AirFit N20



- Start by ensuring the magnets on the lower straps are unclipped.
- If they are clipped, detach them by twisting and pulling them away from the frame.



- Pull the headgear over the head, making sure the logo is facing outward.



- Position the lower straps so they both sit below the ears.
- Bring each of the magnets up to meet their corresponding clips on the frame.



- Adjust the fastening tabs on the upper straps until the mask feels even and stable.
- Reattach the fastening tabs.



- Repeat step 4 with the lower straps.
- Reattach the fastening tabs.



- Connect the device's air tube to the elbow, then click the elbow into the front of the mask.
- Start the device.



Refer to the user guide for further instructions and complete labeling information including indications, contraindications, warnings and precautions.

<sup>1</sup> ResMed AirFit N20 internal international fitting study of new and existing CPAP patients, conducted Nov 2015.

Distributed by ResMed Corp, 9001 Spectrum Center Boulevard, San Diego, CA 92123 USA +1 858 836 5000 or 1 800 424 0737 (toll free). See ResMed.com for other ResMed locations worldwide. AirFit and InfinitySeal are trademarks and/or registered trademarks of the ResMed family of companies. Specifications may change without notice. © 2016 ResMed Ltd. 1019410/2 2016-10