

AirFit™ N30

The ResMed AirFit N30 features a nasal cradle cushion with front-end tube connection and unique features designed to minimize facial contact.

Nasal mask




Systems and Components



Mask



Cushion

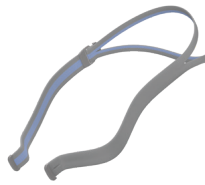


Headgear



How to fit the AirFit N30



Place cushion under the nose with size indicator toward the face.

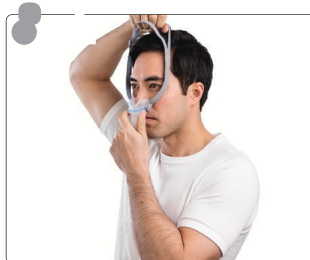

Hold bottom band and pull headgear over head so top band sits comfortably on crown of head. Feed longer hair through back of frame.



Pull cushion away from nose and place it back to ensure a good fit. Spread split bands apart to loosen or draw them closer together to tighten.

Ordering information

Product	Part number	HCPC	Recommended replacement schedule*
Complete mask system (includes frame, cushion and headgear) <ul style="list-style-type: none"> • Small • Medium • Small wide 	64222 64223 64224	A7034 + A7035	-
Frame system (includes frame and cushion) <ul style="list-style-type: none"> • Small • Medium • Small wide 	64210 64211 64212	A7034	Every 3 months
Cushion <ul style="list-style-type: none"> • Small • Medium • Small wide 	64213 64214 64215	A7032	Every 2 weeks
Headgear—standard Replacement cycle may vary depending on patient's insurance coverage.	64216	A7035	Every 6 months