



HumidX™  
HumidX™ Plus  
HumidX™ F20

# A breakthrough in waterless humidification

Introducing HumidX for AirMini, ResMed's first waterless humidification system that's been designed to deliver a comfortable therapy experience for your patients at home, or while they're away.

## Humidification, but not as you know it

As simple as it is impressive, HumidX technology requires no power, water or bulky humidifier. Instead, it uses unique Heat and Moisture Exchange (HME) technology to turn each breath out into a warmer and more natural breath in.

HumidX and ActiveAir technology for nasal and nasal pillows masks displayed.



## Technology that serves you and your patients


The HumidX\* is inserted in front of the vent of the patient's mask, allowing it to capture the air's warmth and humidity before it escapes through the vent.\*

It is ResMed's ground-breaking ActiveAir™ technology that makes waterless humidification in AirMini possible. This technology allows the vent to be built directly into the AirMini mask system and located downstream of the HumidX without compromising on comfort or seal.

HumidX technology is currently available for AirFit™ P10 for AirMini nasal pillows mask, AirFit N20, AirFit N20 Classic, AirTouch™ N20 and AirFit N30 for AirMini nasal masks, and AirFit F20 and AirTouch F20 full face masks.

Learn more about HumidX and AirMini today.  
[ResMed.com/AirMini](https://www.ResMed.com/AirMini)

## ActiveAir vent system

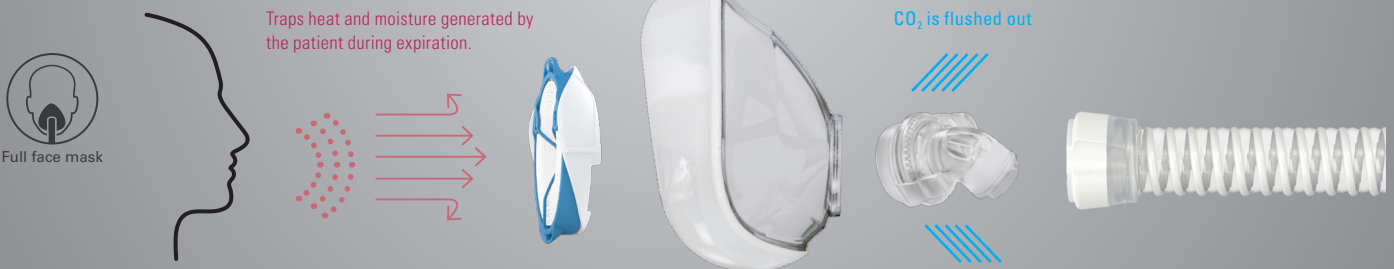

Replace every 30 days for best results. Images are not to scale.

## How it all works

As patients breathe, heat and moisture from every exhale is captured and stored within the HME's tiny paper ridges.

When they inhale, this heat and moisture is released to provide warm and comfortable air throughout the night.



### HumidX for nasal and nasal pillows masks

ResMed's standard waterless humidification system designed for most climates and sleep environments.



### HumidX Plus for nasal and nasal pillows masks

Optimised for dry climates and high altitude environments where humidity levels are very low, e.g., aircraft.

## Technology reinvented

Some hospital devices have been using waterless humidification for over 50 years.

What sets ResMed's HumidX apart from other CPAP waterless humidification solutions is that it passes air through an exchange before it reaches the mask's vent. In doing so, heat and moisture loss is minimised.



### HumidX F20 for full face masks\*

ResMed's standard waterless humidification system designed for most climates and sleep environments.

## Product codes

### For nasal and nasal pillows masks

HumidX 3PK **38809**

HumidX 6PK **38810**

HumidX Plus 3PK **38812**

HumidX Plus 6PK **38813**

### For full face masks\*

HumidX F20 3PK **38012**

HumidX F20 6PK **38013**

Learn more about HumidX at  
[ResMed.com/HumidX](https://www.resmed.com/humidx)