



equipment, always keep accessories (power supplies, battery packs, etc.) and other electrical devices (telephones, radios, etc.) away from your CPAP when used with the cellular modem.

Symbols

Symbol	Definition
	Consult accompanying instructions
	This device contains an RF transmitter
IP22	Drip proof equipment

SpO₂
(Optional Oximeter
Connection)



 **Respshop**
better sleep, better life.
[Click Here To Buy](#)

Installing the Modem

In order for the modem to work properly, it must be installed in a therapy device.

1. Ensure the therapy device is turned off.
2. Open the accessory door on the therapy device.




Open Accessory Door

1. Slide the modem into the accessory slot until it clicks into place.



Slide Modem into Place

4. Close the accessory door on the therapy device.

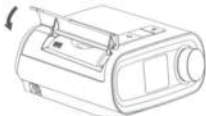

Close Accessory Door

Note: After powering on, up to 30 seconds may elapse before the modem is ready to make a call.

Important! The modem will automatically call within 5 minutes of installation if the blower is off and a manual call has not been successfully completed.

Making a Call

Normally, the modem will automatically make a call and upload therapy data at predetermined times set by your provider. The modem will not make a call while the blower is on and therapy device is delivering therapy.

If instructed to do so by your provider, you can manually make a call by navigating to the My Provider menu on the therapy device and selecting "Upload." The device will then display the animated Sending screen, then display that the upload was completed.



My Provider
Screen



Upload Screen



Animated Sending
Screen



Upload Complete
Screen

Note: When located near a clock radio, speaker, or other electronic device, an occasional buzzing noise may be heard. Relocate the therapy device and modem away from the electronic device to reduce or eliminate this noise.

Modem On/Off

If necessary, you may turn the modem off temporarily by navigating to the My Setup menu and selecting "Modem Off." The modem will not be able to make any calls while off, and an airplane icon will appear on the upload screen. When you are ready to turn the modem back on, navigate to the My Setup menu and select "Modem On."



My Setup Screen



Modem Off Icon



Upload Screen
when Modem is Off



Modem On Icon