



## Ruby Style Chinstrap — Large

Order# **CS007L**

1 Chinstrap

Distributed by

Sunset Healthcare Solutions  
180 N Michigan Ave Ste 2000  
Chicago, IL 60601  
Phone 877-578-6738  
[www.SunsetHCS.com](http://www.SunsetHCS.com)

MADE IN CHINA



LOT 022720148054



### ATTACHMENT INSTRUCTIONS

1. Place the chinstrap on your head prior to putting on your mask and headgear.
2. Holding the rounded chin-cup securely to your chin, stretch the straps up and over your head.
3. The chinstrap should be placed on the head so that the large section with the brand tag is at the back of the neck with the tag facing out. The thinner strap should be on top.
4. After putting on your chinstrap, you may need to readjust your mask and headgear to ensure a proper seal.

### CLEANING INSTRUCTIONS

- Hand wash only with a mild detergent once a week
- Lay flat to dry
- DO NOT MACHINE WASH, DRY OR IRON

