
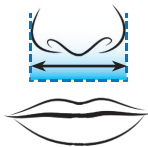


**A. Measure width of nose:**

1"  
1.25"  
1.5"

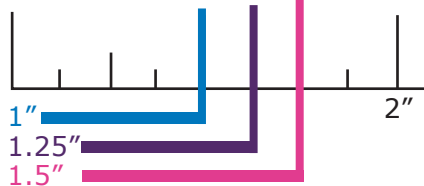

**Nasal Mask Measurement**


- Using the ruler above measure the width of your nose.
- Using your nose measurement, find your mask size in the chart below.

| Nose Size | *Advance | Advance Small | Elan® |
|-----------|----------|---------------|-------|
| Small     |          | ●             | ●     |
| Regular   | ●        |               | ●     |
| Large     | ●        |               | ●     |

\*SleepWeaver Advance is one size fits most. If your nose measurement falls under 1.25" try SleepWeaver Advance Small first.

**SleepWeaver 3D Neck Sizing Guide**


Not actual size  
Use cloth tape measure

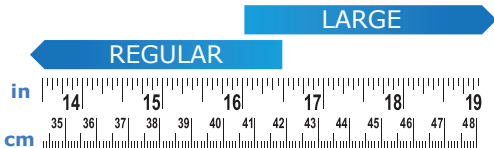

**NOTE: If your measurement falls between sizes on ANY mask, choose the SMALLER mask size.**


**A. Measure width of nose:**

1"  
1.25"  
1.5"

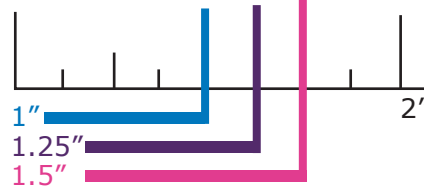

**Full Face Mask Measurements**


- Find measurement A. using top ruler.
- Find measurement B. using side ruler.
- Using both measurements, find your mask size in the chart below.

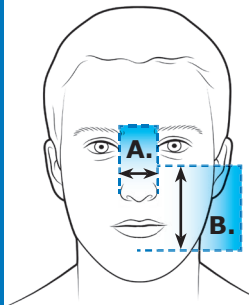

| Anew® Size | A. Measurement | B. Measurement |
|------------|----------------|----------------|
| Small      | 1" to 1.25"    | 3.25" to 3.5"  |
| Regular    | 1.25" to 1.5"  | 3.5" to 3.75"  |
| Large      | 1.5" & Up      | 3.75" & Up     |


**B. Measure from bridge of nose to center of chin:**

3.25"  
3.5"  
3.75"

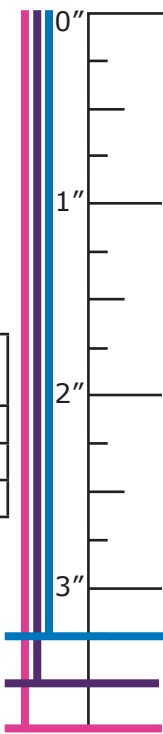


**A. Measure width of nose:**

25mm  
32mm  
38mm

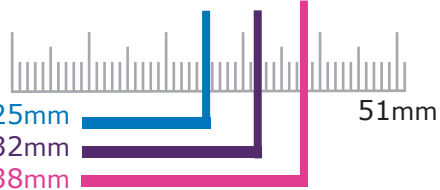

51mm

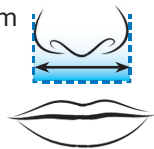

### Nasal Mask Measurement


- Using the ruler above measure the width of your nose.
- Using your nose measurement, find your mask size in the chart below.

| Nose Size      | *Advance | Advance Small | Elan® |
|----------------|----------|---------------|-------|
| <b>Small</b>   |          | ●             | ●     |
| <b>Regular</b> | ●        |               | ●     |
| <b>Large</b>   | ●        |               | ●     |

\*SleepWeaver Advance is one size fits most. If your nose measurement falls under 1.25" try SleepWeaver Advance Small first.

### SleepWeaver 3D Neck Sizing Guide


Not actual size  
Use cloth tape measure

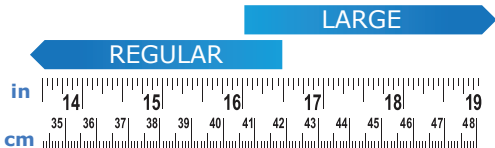

**NOTE: If your measurement falls between sizes on ANY mask, choose the SMALLER mask size.**

**A. Measure width of nose:**


25mm  
32mm  
38mm

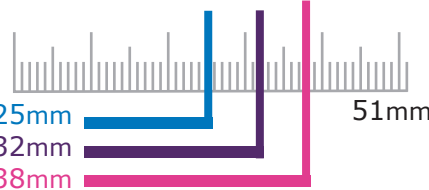

51mm

### Full Face Mask Measurements

- Find measurement A. using top ruler.
- Find measurement B. using side ruler.
- Using both measurements, find your mask size in the chart below.

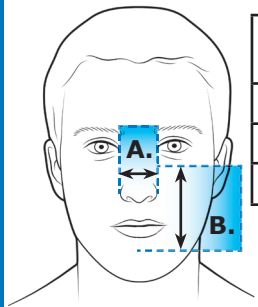

| Anew® Size     | A. Measurement       | B. Measurement       |
|----------------|----------------------|----------------------|
| <b>Small</b>   | <b>25 to 32mm</b>    | <b>83 to 89mm</b>    |
| <b>Regular</b> | <b>32 to 38mm</b>    | <b>89 to 95mm</b>    |
| <b>Large</b>   | <b>38mm &amp; Up</b> | <b>95mm &amp; Up</b> |

**B. Measure from bridge of nose to center of chin:**

83mm  
89mm  
95mm

