Co-Pilot 12 & Co-Pilot 24 Quick Start Guide

PLEASE NOTE

The Co-Pilot 12 and Co-Pilot 24 are intended to be used as an expansion unit for the Pilot 12 Plus and Pilot 24 Plus battery platforms, and will not function as a stand-alone CPAP Battery

Package Contents

| | |
|--|--|
| <p>(1) Co-Pilot 12 or Co-Pilot-24 add-on unit</p>  | <p>(2) Rubber Joiner(s)</p>  |
| <p>(1) DC Cable</p>  | <p>(1) Fabric Travel Pouch</p>  |

Connecting the Co-Pilot

Note: It is important to perform the following steps in order. If these steps are not performed in the order listed below the Co-Pilot may emit an error code. To clear this error code please disconnect all cords from the Co-Pilot and Pilot Plus and then power cycle the device and follow the instructions as they are written below.

Uninterruptible Power Supply (Backup Battery)

1. Secure the Co-Pilot and Pilot batteries together using the two rubber joiners included with the Co-Pilot. Please ensure that the Pilot battery (not included with Co-Pilot package) is in the LOWER slot and the Co-Pilot is in the UPPER slot and that the front of each battery is facing the same direction when joined together.

2. Connect the **OUTPUT** port on the Co-Pilot to the **CHARGE** jack on the Pilot Plus CPAP battery using the DC Cable that is included in the Co-Pilot package.

3. Connect the Pilot Plus to your PAP Device using the output cable that was included with the Pilot Plus.

4. Toggle power switch on Pilot Plus to ON and immediately afterward toggle power switch on Co-Pilot to ON.
5. Connect the AC adapter to the CHARGE port of the Pilot Plus.
Note: The Pilot-12 Plus can only be used with the AC adapter included with the battery. Use of other third-party AC adapters may result in the Pilot-12 Plus emitting an error code.

Note: The Pilot-24 Plus is designed to use the AC adapter that is included with the ResMed S9 or ResMed AirSense 10 platform of PAP devices.

6. The Co-Pilot and Pilot Plus system will now provide power to your PAP device. Refer to the instructions that came with your PAP device for instructions on how to begin therapy.

Standalone Battery (Travel Battery)

IMPORTANT: Use of heated humidification and/or heated tubing will significantly decrease the run-time of the Pilot Plus and Co-Pilot batteries. To maximize you run-time, please either disconnect your PAP device's humidifier or ensure that the heat setting is set to zero.

1. Secure the Co-Pilot and Pilot batteries together using the two rubber joiners included with the Co-Pilot. Please ensure that the Pilot battery (not included with Co-Pilot package) is in the lower slot and the Co-Pilot is in the upper slot and that the front of each battery is facing the same direction when joined together.

2. Connect the **OUTPUT** port on the Co-Pilot to the **CHARGE** jack on the Pilot Plus CPAP battery using the DC Cable that is included in the Co-Pilot package.

3. Connect Pilot Plus CPAP Battery to your CPAP Device using the correct output cable that came with your Pilot Plus battery.

4. Toggle power switch on Pilot Plus to ON and immediately afterward toggle power switch on Co-Pilot to ON.
5. The Co-Pilot and Pilot Plus system will now provide power to your PAP device. Refer to the instructions that came with your PAP device for instructions on how to begin therapy.

Charging the Co-Pilot as an Uninterruptible Power Supply (Backup Battery)

The Pilot Plus and Co-Pilot can both be charged at the same time using a single AC adapter. Connect the appropriate AC adapter to the **CHARGE** port of the Co-Pilot. **Ensure that the power switch of the Co-Pilot is toggled to ON.** The power switch on the Pilot Plus can be toggled to either **ON** or **OFF**.

Note: In this configuration the Pilot Plus will charge first, the Co-Pilot will not begin charging until the Pilot Plus has been fully charged.

Important: The power switch of the Co-Pilot must be **ON** for power to be transmitted to the Pilot Plus.

Note: The Pilot-12 Plus can only be used with the AC adapter included with the battery. Use of other third-party AC adapters may result in the Pilot-12 Plus emitting an error code.

Note: The Pilot-24 Plus is designed to use the AC adapter that is

included with the ResMed S9 or ResMed AirSense 10 platform of PAP devices. Use of other third-party AC adapters may result in the Pilot-24 Plus emitting an error code.

Charging the Co-Pilot as a Standalone Battery (Travel Battery)

For best results ensure that the power switch on the Co-Pilot is toggled to **OFF**. Connect the AC adapter to the **CHARGE** port on the Co-Pilot.

Note: The Pilot-12 Plus can only be used with the AC adapter included with the battery. Use of other third-party AC adapters may result in the Pilot-12 Plus emitting an error code.

Note: The Pilot-24 Plus is designed to use the AC adapter that is included with the ResMed S9 or ResMed AirSense 10 platform of PAP devices. Use of other third-party AC adapters may result in the Pilot-24 Plus emitting an error code.

General Troubleshooting

| LED indicator | Possible Cause | Solution |
|--|---|--|
| Normal Operation | | |
| Solid green LED | Greater than 90% capacity remaining | |
| Solid yellow LED | Remaining capacity is between 10% - 90% | |
| Solid red LED | Remaining capacity is less than 10% | |
| Green LED flashing | Battery is charging | |
| Error Codes | | |
| Orange and green LEDs alternate flashing | <ul style="list-style-type: none"> The Co-Pilot expansion unit was not correctly connected to the primary Pilot battery. Incorrect Pilot battery being used | <ul style="list-style-type: none"> The Co-Pilot expansion unit must be connected to a primary Pilot battery within the first 30 seconds of being switched on Ensure that the primary Pilot unit is the correct model for your Co-Pilot unit. The Co-Pilot-12 must be connected to a Pilot-12 and a Co-Pilot-24 must be connected to a Pilot-24 |
| Red LED flashing (rapidly) | <ul style="list-style-type: none"> Incorrect Pilot battery being used | <ul style="list-style-type: none"> Ensure that the primary Pilot unit is the correct model for your Co-Pilot unit. The Co-Pilot-12 must be connected to a Pilot-12 and a Co-Pilot-24 must be connected to a Pilot-24 |
| Red LED flashing (slowly) | <ul style="list-style-type: none"> The environmental temperature is too high for the battery to operate safely | <ul style="list-style-type: none"> Ensure that the battery is placed where there is no excess heat. Do not rest the battery on top of radiators or electronic devices that may emit heat. Do not attempt to use the battery in temperatures exceeding 40° C / 104° F |
| Red LED (alternates long and short flashing) | <ul style="list-style-type: none"> Incorrect Pilot battery being used | <ul style="list-style-type: none"> Ensure that the primary Pilot unit is the correct model for your Co-Pilot unit. The Co-Pilot-12 must be connected to a Pilot-12 and a Co-Pilot-24 must be connected to a Pilot-24 |

| | | |
|--|---|--|
| | <ul style="list-style-type: none"> • PAP device connected to battery is incompatible • DC output cable is damaged | <ul style="list-style-type: none"> • Ensure that the PAP device you are using is compatible with the Pilot-12 or Pilot-24 you are using • Ensure that the cable connected the Pilot-12 or Pilot-24 to the PAP device is securely connected and not damaged |
| Green LED flashing (slowly) | <ul style="list-style-type: none"> • Incorrect Pilot battery being used • A/C adapter is damaged | <ul style="list-style-type: none"> • Ensure that the primary Pilot unit is the correct model for your Co-Pilot unit. The Co-Pilot-12 must be connected to a Pilot-12 and a Co-Pilot-24 must be connected to a Pilot-24 • Check the A/C adapter for damage. Do not attempt to open or service the A/C adapter yourself. |
| Red and green LEDs alternate flashing | <ul style="list-style-type: none"> • Battery has entered over-discharge protection to prevent damage to the cells. | <ul style="list-style-type: none"> • Contact Medistrom technical support |
| Red LED (three short rapid flashes) | <ul style="list-style-type: none"> • PAP device connected to battery is incompatible | <ul style="list-style-type: none"> • Ensure that the PAP device you are using is compatible with the Co-Pilot-12 or Co-Pilot-24 you are using |
| Red LED (two short flashes followed by one long flash) | <ul style="list-style-type: none"> • PAP device connected to battery has abnormal power draw | <ul style="list-style-type: none"> • Disconnect and reconnect the Co-Pilot from both the primary Pilot battery and the PAP device. Power cycle the Co-Pilot by toggling the power switch off and then on. If the Co-Pilot emits this error code after these steps contact Medistrom technical support |

If errors persist after taking the recommended action contact Medistrom technical support toll-free at **1-888-407-4791** or via email at support@choiceonemedical.com. Do not open the device or attempt to service the device yourself.

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