# ComfortGel Blue

## Instructions for Use

### Intended Use

The ComfortGel Blue nasal mask is intended to provide an interface for application of CPAP or bi-level therapy to patients. The mask is for single patient use in the home or multi-patient use in the hospital/ institutional environment. The mask is to be used on patients (>66lbs/30kg) for whom CPAP or bi-level therapy has been prescribed.

### Notes:

- An exhalation port is built into this mask so a separate exhalation port is not required.
- This mask is not made with natural rubber latex or DEHP.

**⚠ Caution:** U.S. Federal law restricts this device to sale by or on the order of a physician.

### Symbols

Warning or Caution

Note

Tip

Not Made With Natural Rubber Latex

System One Resistance Control

### Warnings

- This mask is not suitable for providing life support ventilation.
- This mask is designed for use with CPAP or bi-level systems recommended by your health care professional or respiratory therapist. Do not wear this mask unless the CPAP or bi-level system is turned on and operating properly. **Do not block or try to seal the exhalation port. Explanation of the Warning:** CPAP systems are intended to be used with special masks with connectors which have vent holes to allow continuous flow of air out of the mask. When the CPAP machine is turned on and functioning properly, new air from the CPAP machine flushes the exhaled air out through the attached mask exhalation port. However, when the CPAP machine is not operating, enough fresh air will not be provided through the mask, and exhaled air may be rebreathed. This warning applies to most models of CPAP systems. Rebreathing of exhaled air for longer than several minutes can, in some circumstances, lead to suffocation.
- If oxygen is used with the device, the oxygen flow must be turned off when the device is not operating. **Explanation of the Warning:** When the device is not in operation, and the oxygen flow is left on, oxygen delivered into the ventilator tubing may accumulate within the device enclosure. Oxygen accumulated in the device enclosure will create a risk of fire.
- Oxygen supports combustion. Oxygen should not be used while smoking or in the presence of an open flame.
- At a fixed flow rate of supplemental oxygen flow, the inhaled oxygen concentration will vary, depending on the pressure settings, patient breathing pattern, mask selection, and the leak rate. This warning applies to most types of CPAP and bi-level machines.
- Some users may experience skin redness, irritation, or discomfort. If this happens, discontinue use and contact your healthcare professional.
- At low CPAP or EPAP pressures the flow through the exhalation port may be inadequate to clear all exhaled gas from the tubing. Some rebreathing may occur.
- If an additional exhalation device is added to the patient circuit, you may need to adjust the pressure

level to compensate for the additional leak of the exhalation device.

- The gel mask is not customizable. Do not attempt to heat.
- Hand wash prior to first use. Inspect the mask for damage or wear (cracking, crazing, tears or cushion damage resulting in gel exposure, etc). Discard and replace any component as necessary.
- Use of a nasal or full face mask may cause tooth, gum or jaw soreness or aggravate an existing dental condition. Consult your physician or dentist if symptoms occur.

---

### Before Use Read and Understand the Instructions

- Hand wash mask.
- Inspect all parts for damage; replace any parts that are visibly deteriorated (cracking, crazing, tears, etc).
- Clean your face.
- Verify that the mask and headgear are the correct size.

---

### Cleaning Instructions

Hand wash the mask before first use and daily. The headgear should be hand washed weekly, or as needed. The headgear does not need to be removed for daily cleaning of the mask. To disassemble the mask for cleaning, please refer to the Cushion Removal later in this document.

1. Hand wash the mask and headgear in warm water with a mild liquid dish washing detergent.

**⚠ Caution:** Do not use bleach, alcohol, cleaning solutions containing bleach or alcohol, or cleaning solutions containing conditioners or moisturizers.

**⚠ Caution:** Any deviation from these instructions may impact the performance of the product.

2. Rinse thoroughly with water and air dry out of direct sunlight. Make sure the mask is dry before use. Lay the headgear flat or line dry. Do not place the headgear into the dryer.

**⚠ Warning:** Inspect the mask for damage or wear (cracking, crazing, tears, cushion damage resulting in gel exposure, etc). Discard and replace any components as necessary.

### Institutional Disinfection

For multi-patient use in the hospital/institutional environment, use the Disinfection Guide to reprocess the mask between patients. These instructions can be obtained by visiting us online at [www.philips.com/respironics](http://www.philips.com/respironics) or by contacting Philips Respironics Customer Service at 1-800-345-6443 (USA or Canada) or at 1-724-387-4000.


---

### Philips Respironics System One Resistance Control

Your mask when combined with a Philips Respironics System One device, provides optimal resistance compensation. This mask value is X1 and should be set by your provider.

#### ⊖ Notes:

- Compare the mask to the device. See your device manual if the values do not match.
- System One is not compatible with masks that use a separate exhalation device.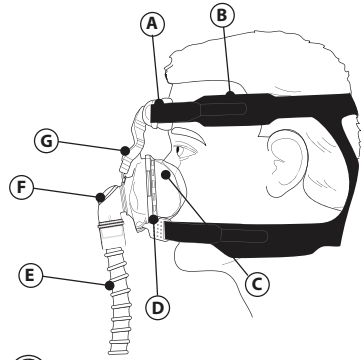

▶ Figure 1

A = Forehead Pad

B = Headgear Tab

C = Gel Cushion with Comfort Flap

D = Quick Clip and Faceplate Socket

E = Tubing

F = Elbow with Exhalation  
(Do not block or remove)

G = StabilitySelector

## Taking Off the Headgear

Undo the EZPeel tabs on the headgear and pull them through the slots in the forehead support and through the slots on the Quick Clips.

To reattach the headgear, thread the top headgear straps through the slots in the forehead support and through the slots on the Quick Clips. Fold the EZPeel tabs to latch onto the headgear straps.

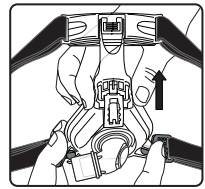
Figure 2

## Achieving the Right Fit

1. Position the stability selector to its uppermost position (Figure 2). Hold the mask cushion over your nose.

**Note:** You may want to disconnect a quick clip before putting on the headgear.

2. Slide the headgear over your head (Figure 3). Reconnect the quick clip after you have the headgear in position.
3. Adjust the headgear evenly using the top (Figure 4) and bottom (Figure 5) headgear straps. The mask should rest comfortably against your face.

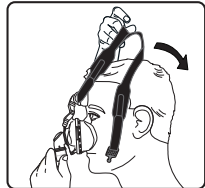
Figure 3

**Warning:** Do not overtighten the headgear!

4. Connect the flexible tubing from your therapy device to the mask elbow. Turn on the airflow.
5. Minimal leaks are acceptable. Small adjustments are all that are needed to eliminate excessive leaks. We suggest the following adjustments:
6. Re-seat the mask by pulling the mask directly away from your face and set it back into place gently (Figure 6).
7. For leaks around the eyes, use the stability selector and adjust downward one click at a time (Figure 7).
8. For leaks around the mouth, a slight adjustment to the lower headgear straps will minimize the leak, do not overtighten.
9. Make any final adjustments lying down.

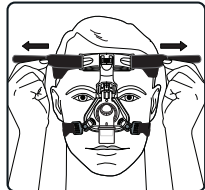
Figure 4

## Comfort Tips

- Re-seat the mask by pulling it directly away from your face and gently setting it back into place.
- Use the stability selector to adjust for leaks or pressure. Some leakage is normal.
- The headgear should fit comfortably and the mask should not dig into your skin. If your skin bulges around the mask or if you see red marks on your face due to tightness, loosen the headgear.

**Note:** Do not attempt to separate the StabilitySelector from the faceplate rail.

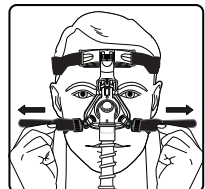
Figure 5

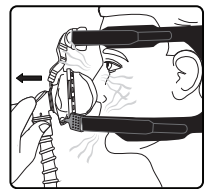
Figure 6

## Cushion Removal and Attachment

1. Remove the gel cushion by detaching the comfort flap with its integrated retaining ring from the faceplate. Do this by unhooking the side and bottom clips of the ring from their connection points on the faceplate and separating the pieces.


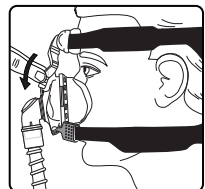
Figure 7

2. Clean or discard the gel cushion as necessary.
3. To attach, place the clean or new gel cushion into the faceplate. Then place the comfort flap over the gel cushion and push the retaining ring into the faceplate until it clicks into place.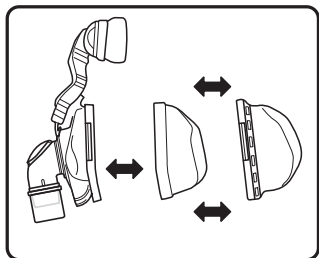


## Specifications

### **⚠ Warning:**

The technical specifications of the mask are provided for your healthcare professional to determine if it is compatible with your CPAP or bi-level therapy device. If used outside these specifications, or if used with incompatible devices, the mask may be uncomfortable, the seal of the mask may not be effective, optimum therapy may not be achieved, and leak, or variation in the rate of leak, may affect device function.

## Pressure Flow Curve

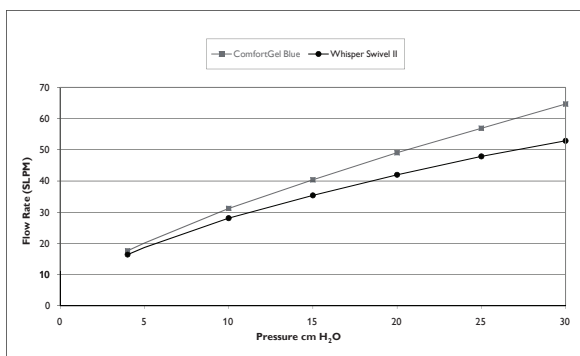## Resistance

Drop in Pressure at

	50 SLPM	100 SLPM
All sizes	0.1 cm H <sub>2</sub> O	0.7 cm H <sub>2</sub> O

## Deadspace Volume

Petite	76 mL
Small	97 mL
Medium	99 mL
Large	118 mL

## Sound Levels

A-weighted Sound Power Level 28 dBA

A-weighted Sound Pressure Level at 1 m 20 dBA

## Disposal

Dispose of in accordance with the local regulations

## Storage Conditions

Temperature: -4° to 140° F (-20° to +60° C)

Relative Humidity: 15% to 95% non-condensing

---

## LIMITED WARRANTY

Respironics, Inc. warrants that its mask systems (including mask frame and cushion) (the "Product") shall be free from defects of workmanship and materials for a period of ninety (90) days from the date of purchase (the "Warranty Period").

If the Product fails under normal conditions of use during the Warranty Period and the Product is returned to Respironics within the Warranty Period, Respironics will replace the Product. This warranty is non-transferable and only applies to the original owner of the Product. The foregoing replacement remedy will be the sole remedy for breach of the foregoing warranty. This warranty does not cover damage caused by accident, misuse, abuse, negligence, alteration, failure to use or maintain the Product under conditions of normal use and in accordance with the terms of the product literature, and other defects not related to materials or workmanship. This warranty does not apply to any Product that may have been repaired or altered by anyone other than Respironics.

Respironics disclaims all liability for economic loss, loss of profits, overhead, or indirect, consequential, special or incidental damages which may be claimed to arise from any sale or use of the Product. Some states do not allow the exclusion or limitation of incidental or consequential damages, so the above limitation or exclusion may not apply to you.

THIS WARRANTY IS GIVEN IN LIEU OF ALL OTHER EXPRESS WARRANTIES. IN ADDITION, ANY IMPLIED WARRANTY, INCLUDING ANY WARRANTY OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE, IS SPECIFICALLY DISCLAIMED. SOME STATES DO NOT ALLOW DISCLAIMERS OF IMPLIED WARRANTIES, SO THE ABOVE LIMITATION MAY NOT APPLY TO YOU. THIS WARRANTY GIVES YOU SPECIFIC LEGAL RIGHTS, AND YOU MAY ALSO HAVE OTHER RIGHTS, WHICH VARY FROM STATE TO STATE.

To exercise your rights under this limited warranty, contact your local authorized Respironics, Inc. dealer or Respironics, Inc. at 1001 Murry Ridge Lane, Murrysville, Pennsylvania 15668, 1-800-345-6443 (USA and Canada only) or 1-724-387-4000.

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