

**SETTING UP YOUR UNIT**

**NOTE:** Refer to instruction manual for more detailed instructions.

**STEP 1**



Remove the IntelliPAP DV55, the humidifier cradle, and the blue water chamber from the largest section of the carrying case.

**STEP 2**



Remove the tubing and the power cord from the zipper pockets found on either side of the carrying case.

**STEP 3**



Remove blue water chamber from humidifier cradle (A). Tilt upright so the front of the chamber is pointing toward the ground (B). Pour water into the water chamber, which is marked with maximum capacity water lines (C). It is recommended that you use distilled water so there is not mineral deposit build up on the inside of the humidifier.



**STEP 4**



Slide the water chamber back into the humidifier cradle.

**STEP 5**



Place IntelliPAP on top of heated humidifier cradle (A). Slide the blue water chamber out slightly before attaching IntelliPAP. Be sure to align the front corners first so the IntelliPAP and humidifier lock into place (you'll hear a click, indicating it's in the proper position). All sides should be flush (B).



**STEP 6**



Plug power cord into the back of IntelliPAP.

 **RespsShop**  
better sleep, better life.  
[Click Here To Buy](#)

**STEP 7**



Attach tubing to air supply port on the back of the humidifier. Push rubber seal downward until it meets the plastic. **NOTE:** The rubber air supply port plug should be sealing the port on the back of the CPAP, not the port on the bottom of the unit.

**STEP 8**



Attach free end of tubing to interface/mask port.

## CLINICAL SETUP MENU

Follow these instructions to access the device settings and prepare the device for patient use. All settings have pre-set values as shown below.

**FIRST** - Make sure the device is plugged in and turned off. Simultaneously press the **DOWN** arrow button and the **DELAY** button.



Use the **LEFT** and **RIGHT** arrow buttons to navigate through each category of the menu.



**SECOND** - While holding the **DOWN** arrow button and the **DELAY** button, press the **ON/OFF** button. This will turn on the device and take you to the "Clinical Menu". You can now access the Clinical Setup Menu. (reference chart below)



Use the **UP** and **DOWN** arrow buttons to adjust settings for the individual patient.



### CLINICAL SETUP MENU CHART

Menu Item	Description	Factory Default
<b>Clinical Menu</b>		
Language Selection	Select device language. Range: English, French, German, Spanish, Italian, Dutch, Danish, Finnish, Norwegian, Swedish.	English
IPAP	Set to IPAP pressure prescribed by doctor. Range: EPAP-25 cmH <sub>2</sub> O, increments of 0.5. The pressure delta between IPAP and EPAP is also displayed for reference.	15.0 mH <sub>2</sub> O
EPAP	Set to EPAP pressure prescribed by doctor. Range: 3-IPAP cmH <sub>2</sub> O, increments of 0.5. The pressure delta between IPAP and EPAP is also displayed for reference.	8.0 cmH <sub>2</sub> O
Delay Time	Select period of time to stay at delay pressure. This delay is patient activated. Range: 0-45 minutes, increments of 5	20 min
Delay Pressure	If prescribed, set to a level that is comfortable for the patient to fall asleep. During a delay the pressure starts at this level and rises to the set CPAP/IPAP/EPAP pressure over the delay time. Range: 3 cmH <sub>2</sub> O to EPAP (or CPAP in CPAP mode), increments of 0.5	4.0 cmH <sub>2</sub> O
IPAP Rounding	Pressure waveform profile (if in bilevel operating modes). Range: 0 - 5, increments of 1.	3
EPAP Rounding	Pressure waveform profile (if in bilevel operating modes). Range: 0 - 5, increments of 1.	3
Pressure Unit	The device operating pressure may be displayed in one of three pressure units: cmH <sub>2</sub> O, hPa or mbar.	cmH <sub>2</sub> O
Compliance Meter	Cumulative hours of "while breathing" time. Clear compliance meter and SmartCode data by pressing ▼ button for eight seconds with compliance meter shown (value will go to zero).	N/A
Enable Menu	Press the ▼ button to enter. Reference the Enable Submenu section for details.	N/A
Lockout Menu	Press the ▼ button to enter. Reference the Lockout Submenu section for details.	N/A
<b>Enable Submenu</b>		
Auto-OFF	This option controls the Auto-OFF feature, which automatically stops the flow of air after taking off the mask. Range: Enabled, Disabled	Disabled
Auto-ON	This option controls the Auto-ON feature, which automatically starts the flow of air after breathing once or twice into the mask. Range: Enabled, Disabled	Enabled
Notifications	Setting to enable mask/filter check notification. Range: Enabled, Disabled	Enabled
Exit Submenu	Press the ▲ button to exit and return to Clinical Menu.	N/A
<b>Lockout Submenu</b>		
Delay Time Lockout	Used to allow patient access to CPAP delay time setting. Range: Locked, Unlocked	Unlocked
Exit Submenu	Press the ▲ button to exit and return to Clinical Menu.	N/A