

AirFit™ F30i



Don't sacrifice freedom for performance

Now you don't have to choose between what your patients want and what they need. Introducing ResMed AirFit™ F30i, the full face mask with a top-of-head tube connection that's designed to deliver a reliable seal with the freedom to sleep in any position. AirFit F30i was preferred by 3 out of 4 CPAP users to DreamWear™ full face for its overall performance in a direct comparison study.*



ResMed.com/AirFitF30i

AirFit F30i highlights

Quick-release

elbow allows for easy disconnection. It stays attached to the tubing to ensure mask parts don't get lost or stuck.

Flexible conduit frame

allows pressurized air to flow through the elbow and into the frame and cushion.

UltraCompact full face

cushion design is curved for stability and improved seal performance.

Top-of-head tube design

gives patients the freedom to sleep in any position.

Magnetic clips

make it easy to securely connect the bottom of the headgear to the frame.



Complete mask systems (includes mask frame, full face cushion and headgear)

Product name	Model number
AirFit F30i System – S/SML	63330
AirFit F30i System – S/STD	63331
AirFit F30i System – M/STD	63332
AirFit F30i System – W/STD	63333
AirFit F30i System – M/LGE	63334

Full face cushion

Product name	Model number
AirFit F30i Cushion SML	63350
AirFit F30i Cushion MED	63351
AirFit F30i Cushion SML – WIDE	63352
AirFit F30i Cushion WIDE	63353

Headgear

Product name	Model number
AirFit F30i Headgear – STD	63372

Frame system (includes mask frame and full face cushion)

Product name	Model number
AirFit F30i Frame System – S/SML	63355
AirFit F30i Frame System – M/SML	63356
AirFit F30i Frame System – SW/SML	63357
AirFit F30i Frame System – W/SML	63358
AirFit F30i Frame System – S/STD	63359
AirFit F30i Frame System – M/STD	63360
AirFit F30i Frame System – SW/STD	63361
AirFit F30i Frame System – W/STD	63362
AirFit F30i Frame System – S/LGE	63363
AirFit F30i Frame System – M/LGE	63364
AirFit F30i Frame System – SW/LGE	63365
AirFit F30i Frame System – W/LGE	63366

Conduit mask frame (does not include full face cushion)

Product name	Model number
AirFit F30i Conduit – SML	63368
AirFit F30i Conduit – STD	63369
AirFit F30i Conduit – LGE	63370


A better full face mask experience

In a recent study, full face mask users found that AirFit F30i was significantly better* than DreamWear full face.¹ They also found it provided them with a more reliable seal, was easier to use,* and was more comfortable to wear and breathe in than DreamWear full face.¹




Complete mask system

- 63330
- 63331
- 63332
- 63333
- 63334

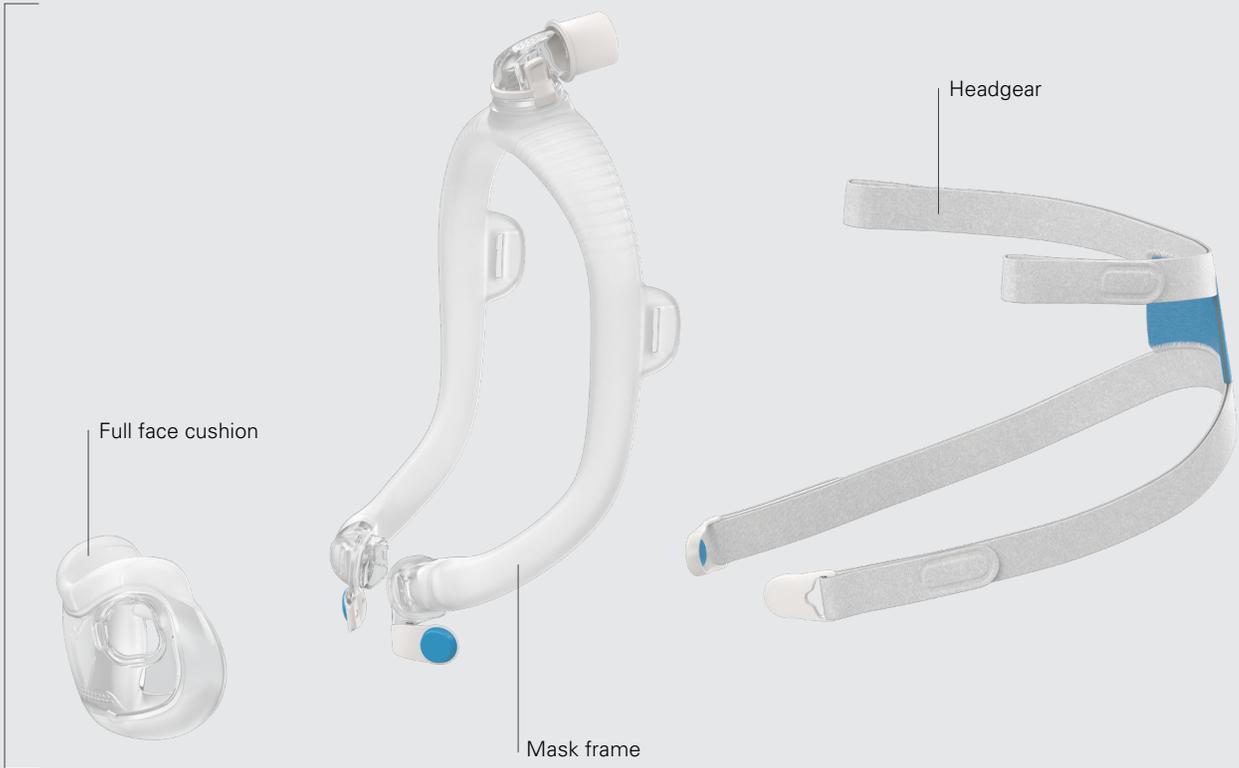

AirFit™ F30i
Full face mask

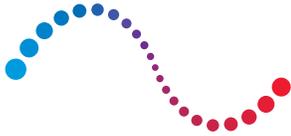

ResMed

AirFit™ F30i

Fitting guide for AirFit F30i



- With one hand holding the bottom of the cushion and the other holding the conduit frame, position the mask in front of the face.



- Place the cushion under the nose and against the face. With the ResMed logo and headgear facing up, pull the headgear over the head.



- Bring the lower headgear straps under the ears and attach the magnetic clips to the frame connectors.



- Undo the fastening tabs on the upper headgear straps and pull evenly.



- Repeat with the lower headgear straps.



- Squeeze the side buttons on the elbow and detach from the frame. Connect the air tubing from the device to the elbow, then reattach it to the frame.



For people with long hair, we recommend feeding the hair through the back of the headgear, as pictured. If tied in a ponytail, it should sit high enough to fit through the back of the headgear.

Refer to the user guide for further instructions and complete labeling information including indications, contraindications, warnings and precautions.

* Calculations based on CPAP users in the study who had a mask preference. CPAP users rated objective criteria for AirFit F30i and DreamWear full face on a Likert Scale of 0-10. Statistically significant differences ($p < 0.05$) in median scores were found between the two masks. Easy to use is defined as: easy to fit, adjust and remove.

1 ResMed guided external clinical study of 48 evaluable current full face CPAP mask users from Oct. 24 to Nov. 20, 2018 in Sydney, AU and San Diego, CA, who trialed AirFit F30i and Phillips™ Respiroics DreamWear full face mask at home. Distributed by ResMed Corp, 9001 Spectrum Center Boulevard, San Diego, CA 92123 USA +1 858 836 5000 or 1 800 424 0737 (toll free). See ResMed.com for other ResMed locations worldwide. AirFit is a trademark and/or registered trademark of the ResMed family of companies. DreamWear and Phillips are trademarks of Koninklijke Philips N.V, registered in the US and other countries. Specifications may change without notice. For patent and other intellectual property information, see ResMed.com/ip. © 2019 ResMed. 10111403/2 2019-11