

Heat Moisture Exchanger For Humidification

Your Z1 includes a complimentary Heat Moisture Exchanger(HME) to provide humidification during therapy. HME technology has been in use for over 50 years and is now available for your Z1.

The hygienic, disposable HME works by capturing heat and moisture as you exhale and provides it back to you as you breathe. The HME does not use any electricity or complex moving parts and avoids the problems encountered with electrical humidification systems.

This HME should be discarded after seven days of consecutive use after initial installation or when it fails to provide adequate humidification, whichever comes first. Additional HMEs may be purchased from your retail medical supplier.

Installation

To install a Heat Moisture Exchanger for therapeutic use, connect the large diameter end of the HME to your mask and the smaller diameter end to your CPAP Tube. Be sure both connections are fully seated and secure. Your HME is now ready for use.

When you have finished therapy, please be sure to dispose of the used HME.



Manufactured for Breas Medical, Inc

Reorder No. HD60-2100

Mfg. No. MC-775NF

BREAS