

Less mask. **More you.**



## Mirage™ FX. A transformation in nasal masks.

Make room for the only nasal mask you need. The perfect balance of lightweight comfort, premium performance and simplicity, the Mirage™ FX will transform your nasal mask business.

### **Instant patient appeal**

With its compact frame, gentle cushion and ultra soft headgear, your patients will love the openness and comfort of the Mirage FX.

### **One mask fits all**

One single cushion and frame size accommodates nearly all your patients. And, for patients who require a slightly wider fit, the Mirage FX is available in a Wide size. For your female patients who want a personalized option with a smaller headgear size, try the Mirage FX for Her.

### **Effortless and effective**

Intuitive adjustment and contouring headgear make it unbelievably easy to achieve a perfect fit and secure seal on the first try.

### **Better business efficiencies**


Reduce the time you spend on refitting and swapouts. Additionally, with fewer product codes and four simple components, inventory management is dramatically simplified.

**Mirage™ FX**  
NASAL MASK

[ResMed.com](http://ResMed.com)

## Technology highlight

- New “butterfly wing” forehead support flexes to automatically adjust during fitting
- Streamlined design enables simple adjustment of seal on nasal bridge
- The soft, contoured headgear is all that touches the forehead—no silicone or plastic contact

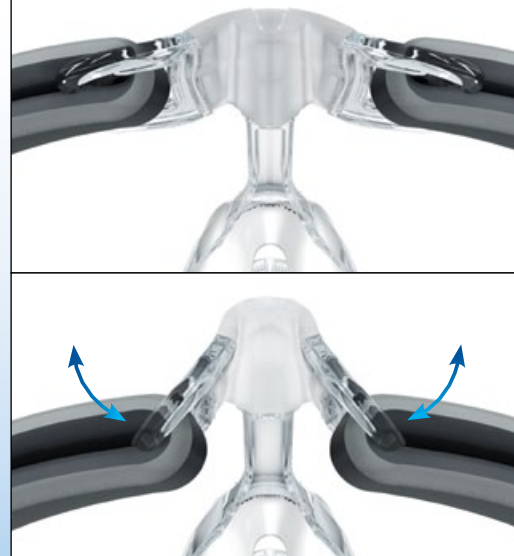

**Flexible, “butterfly wing” forehead support**  
allows for a dynamic range of fit and adjustment; the plastic does not contact the skin

**Lightweight, durable frame**  
is strong enough to stabilize the mask on the face, and soft and flexible enough to accommodate user movement during sleep

**Diffused vent design**  
disperses air gently and quietly away from the bed partner

**Squeeze-tab elbow**  
makes it easy to attach and detach the elbow from the frame

**New 3D-contoured SoftEdge™ headgear**  
springs to life, ready to fit and wear

**SoftEdge design**  
is kind to the face, reducing marks and discomfort

**Dual-wall, Spring Air™ cushion**  
provides even pressure distribution with extra softness at the nasal bridge, reducing common discomfort in this sensitive area

**Headgear loops**  
simplify attachment and detachment of headgear; retain optimal headgear settings



# Components Guide

Give your patients a nasal mask that helps them accept therapy more easily and adopt it long-term. Make the **Mirage FX** the go-to choice for your nasal mask patients.

**PRODUCT CODES**

**Complete system**



**62103 Mirage FX**



**62128 Mirage FX for Her**



**62118 Mirage FX Wide**



**Frame system (no headgear)**  
**62112 Mirage FX**  
**62112 Mirage FX for Her**  
**62126 Mirage FX Wide**



**Headgear**  
**62110**  
**62129 For Her**



**Cushion**  
**62111**  
**62125 Wide**

**REIMBURSEMENT CODES**

HCPCS	Code Descriptor
A7034	Nasal mask used with positive airway pressure device, 1 per 3 months
A7032	Replacement cushion for nasal mask, 2 per month
A7035	Headgear used with positive airway pressure device, 1 per 6 months

 For details, refer to the **Mirage FX, Mirage FX Wide or Mirage FX for Her Nasal Mask User Guide.**



## Fitting Guide



- Prior to fitting the mask, remove the elbow from the mask by pressing the squeeze-tabs.



- Connect the elbow to the device air tubing.



- With both lower headgear straps released, hold the mask against the face and pull the headgear over the head.



- Bringing the lower straps below the ears, loop the headgear into the lower hooks on the mask frame.



- Unfasten the straps, pull the upper headgear straps evenly until they are comfortable and firm, and reattach the straps.



- Repeat step 5 with the lower headgear straps.



- Attach the combined elbow assembly and air tubing to the mask by pressing the squeeze-tabs and pushing into mask frame.
- Turn on the airflow device.

Pair the Mirage FX with ResMed's S9™ for an exceptionally quiet, comfortable system that fits your patients' lifestyles.



**Mirage™ FX**  
NASAL MASK

ResMed.com