



MAKERS OF RESPIRATORY VALVES SINCE 1938



Click Here To Buy

HANS RUDOLPH, inc.

8325 Cole Parkway, Shawnee, KS 66227 USA

PH: (913) 422-7788
ISO 9001

USA & Canada: (800) 456-6695
ISO 13485

FAX: (913) 422-3337

hri@rudolphkc.com
www.rudolphkc.com


**SIZE DOES MATTER WHEN WEARING THE HANS RUDOLPH V2 MASKS
PLEASE USE OUR NEW MASK SIZING GAUGE
HANS RUDOLPH MASKS RUN LARGE
WE HAVE 5 SIZES AVAILABLE**

**USING THE NEW HANS RUDOLPH MASK SIZING GAUGE (MSG)
(Gauge is a reference to aid in sizing and not an exact measurement)**

- 1 Deepest point of the depression between the nose and forehead (sellion)
- 2 Measurement location indicating mask size

Instructions

1. Place top of gauge at location noted
2. Close mouth
3. Tilt head back slightly to tighten skin under chin
4. Size indication is the point where the chin EXITS the gauge
5. If between sizes, we recommend you use the smaller size.

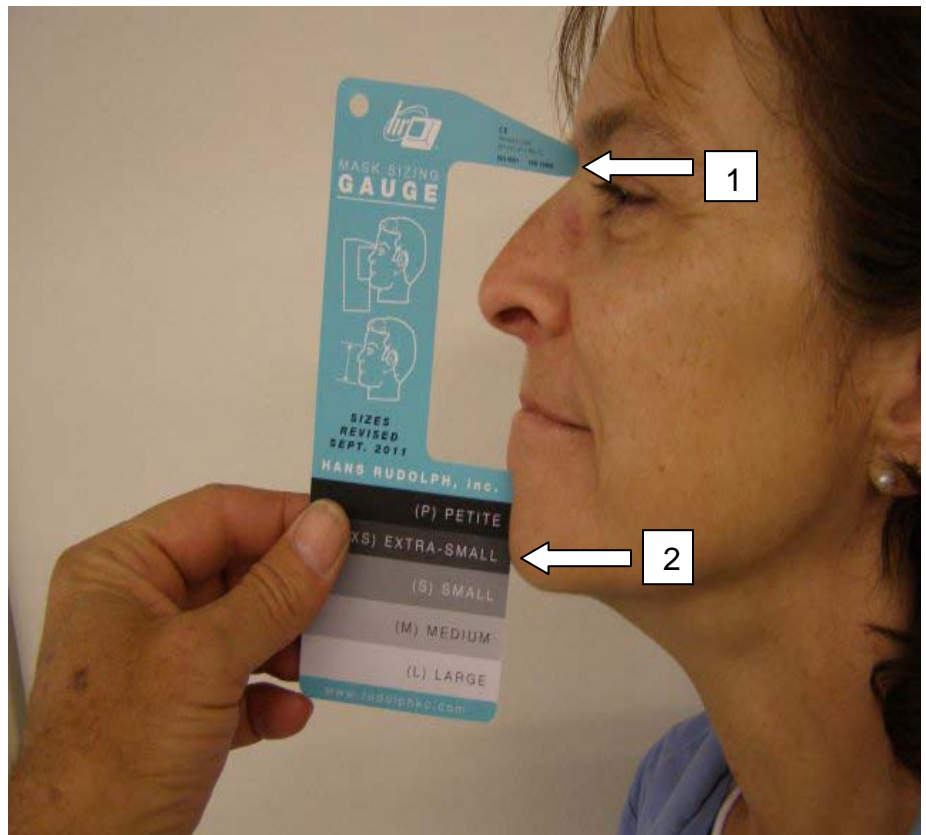

This patient measurement would recommend an XS size V2 mask

MSG Reorder No. 691143