

QUICK START GUIDE

⚠ IMPORTANT!

- When using the Pilot-12 Lite or Pilot-24 Lite as a portable PAP battery, please disconnect the heated humidifier and heated hose from your PAP device as this will significantly decrease the battery run time!
- Fully charge the battery before storage and a minimum every 6 months while stored!
- DO NOT expose the battery to temperatures above 40°C / 104°F or below -10°C / 14°F. Exposing the battery to very low temperatures can cause damage to the lithium ion cells and exposing the battery to very high temperatures can be a fire hazard!


For technical support and questions about this product please email to support@medistrom.com or contact us by phone:

North America: 1-888-407-4791

Australia: 1300-146-358

Europe: +44 800 060 8576

PILOT LITE CONTROL PANEL

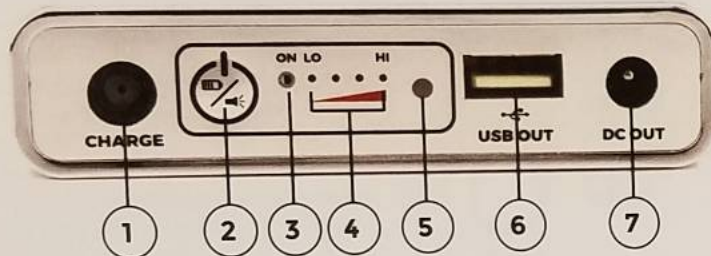


- | | |
|--|---------------------------|
| 1. Charge Port | 5. LED Flashlight |
| 2. Multi Function Button | 6. USB Port (5V 2.5A Out) |
| 3. Blue "ON" Indicator Light | 7. DC Out Port |
| 4. Green Charge Level Indicator Lights | |

SETTING UP YOUR PILOT-12 LITE

STEP 1



Plug the original Phillips DreamStation or Phillips System One S60 power adapter into the Charge Port of the Pilot-12 Lite and plug the other end into the mains power outlet. (Please fully charge your Pilot-12 Lite before using for the first time).

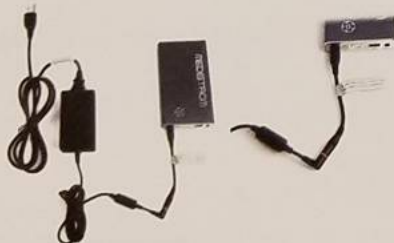

NOTE: If you are using the following compatible 12V PAP device AC Power Adapters, please use the appropriate Short Charging Adapter cables:

- DeVilbiss IntelliPAP I (not included, SKU P12MXDPLK)
- HDM Z1 (not included, SKU P12MXDPLK)

STEP 2



Plug the L-shaped yellow tip of the corresponding DC Output cable for your PAP device into the DC Out port of the Pilot-12 Lite. Then connect the other end into your compatible 12V PAP device.



For help with disabling or disconnecting your heated humidifier and heated tubing please visit www.medistrom.com/support.html



SETTING UP YOUR PILOT-24 LITE

STEP 1



Plug the original ResMed AirSense 10 AC power adapter into the Charge Port of the Pilot-24 Lite and plug the other end into the mains power outlet. (Please fully charge your Pilot-24 Lite before using for the first time).



NOTE: If you are using the following compatible 24V PAP device AC Power Adapters, please use the appropriate Short Charging Adapter cables:

- ResMed S9 series (not included, SKU P24R09PLK)
- Apex Medical iCH series (not included, SKU P24AMIPLK)

STEP 2



Plug the L-shaped yellow tip of the corresponding DC Output cable for your PAP device into the DC Out port of the Pilot-24 Lite. Then connect the other end into your compatible 24V PAP device.



For help with disabling or disconnecting your heated humidifier and heated tubing please visit www.medistrom.com/support.html