

# Somnetics Transcend Series: Single Battery Connection

## REQUIRED COMPONENTS


- Freedom™ CPAP Battery Kit
- 150W Sine Wave Power Inverter (inverter)



Due to the initial power draw required to start the device **the connection instructions below must be followed in exact sequence** in order to prevent “tripping” the internal circuitry of the Freedom™ Battery. If the battery does shut down simply disconnect all components, press the charge level indicator button to reset the battery and follow the instructions in exact sequence.

## CONNECTION INSTRUCTIONS

- Plug the DC input cord into the DC OUT jack of the battery.
- Plug the inverter into the DC input cord.
- Plug the AC power cord of the PAP device into the inverter.
- Plug the AC power cord into the PAP device.
- Turn the PAP device on and it should begin functioning.
- The digital display screen will show the charge level of the battery. You may press the charge level indicator button to illuminate this screen. It will stay illuminated for a few seconds at a time.
- When not in use disconnect all devices from the battery thus allowing the battery to completely shut down.

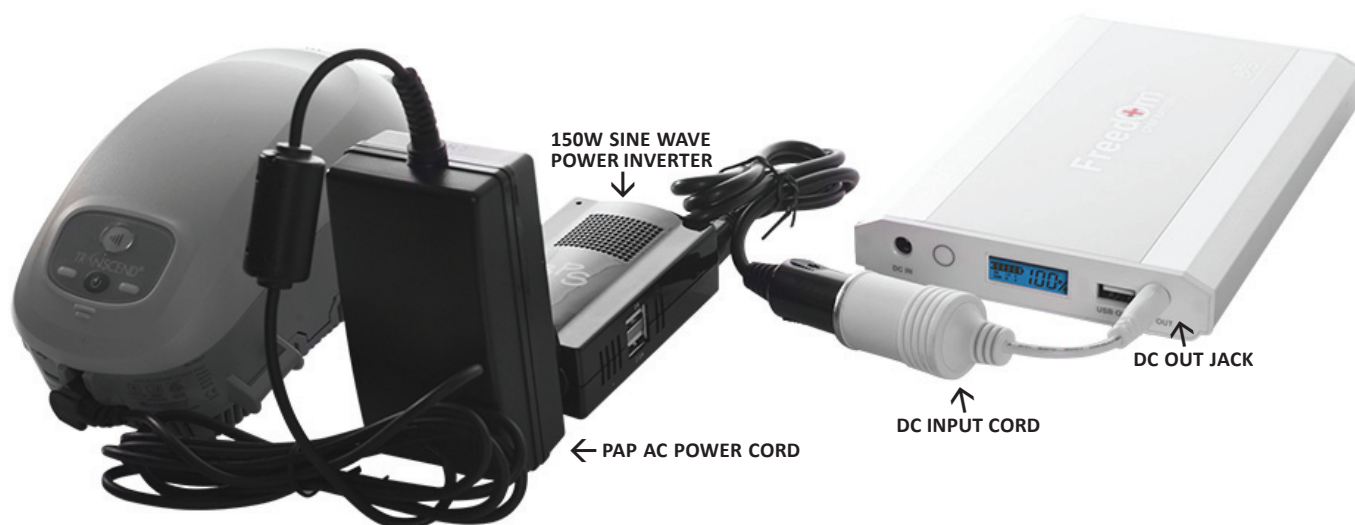

The inverter can also be used to charge the battery in a vehicle (vehicle does not need to be running) and to power/charge other small electronic devices with the battery. The inverter is not proprietary to the Freedom™ CPAP Battery and can be used with other external batteries.

## OPTIONAL FREEDOM™ CPAP BATTERY CHARGING ACCESSORIES

- Auto Charge DC Cable {BPS DC.CABLE}: Used to charge the battery in a vehicle (vehicle must be running).
- SunPower 40W Solar Panel {BPS SP40}: Used to charge the battery using the power of the sun.

# Somnetics Transcend Series: Dual Battery Connection

## REQUIRED COMPONENTS

- Freedom™ CPAP Battery Kit
- Freedom™ CPAP Battery (additional battery only)
- 150W Sine Wave Power Inverter (inverter)

Due to the initial power draw required to start the device **the connection instructions below must be followed in exact sequence** in order to prevent “tripping” the internal circuitry of the Freedom™ Battery. If the battery does shut down simply disconnect all components, press the charge level indicator button to reset the battery and follow the instructions in exact sequence.

## CONNECTION INSTRUCTIONS

- Plug the DC power pigtail cord into the DC OUT jacks of both batteries.
- Plug the DC input cord into the DC power pigtail cord.
- Plug the inverter into the DC input cord.
- Plug the AC power cord into the PAP device.
- Turn the PAP device on and it should begin functioning.
- The digital display screen will show the charge level of the battery. You may press the charge level indicator button to illuminate this screen. It will stay illuminated for a few seconds at a time.
- When not in use disconnect all devices from the battery thus allowing the battery to completely shut down.



The inverter can also be used to charge the battery in a vehicle (vehicle does not need to be running) and to power/charge other small electronic devices with the battery. The inverter is not proprietary to the Freedom™ CPAP Battery and can be used with other external batteries.

## OPTIONAL FREEDOM™ CPAP BATTERY CHARGING ACCESSORIES

- Auto Charge DC Cable {BPS DC.CABLE}: Used to charge the battery in a vehicle (vehicle must be running).
- SunPower 40W Solar Panel {BPS SP40}: Used to charge the battery using the power of the sun.