

Slim Style CPAP Tubing Instructions

Installation & Removal

Be sure to follow your CPAP device manufacturer's instruction for the proper installation and removal of the CPAP tubing.



Maintenance & Cleaning

Daily

Remove the CPAP tube from your device and mask. Hang it in a clean, dry place out of direct sunlight. Reattach to your device and mask before next use.

Weekly

Remove the CPAP tube from your device and mask. Wash the CPAP tube in warm water using mild soap. Rinse thoroughly, hang, and allow to dry.

NOTE: Soaking the CPAP Tube in water for longer than 10 minutes may shorten the life of the tube.

Every 90 days

The CPAP tube should be replaced every 90 days or earlier if it becomes discolored, cracked or damaged.

CAUTION: Do not use bleach, chlorine, alcohol, or aromatic-based solutions, moisturizing or antibacterial soaps or scented oils to clean the CPAP tube. Never attempt to clean the CPAP tube with any of these solutions. These solutions may create a shock hazard, cause damage and reduce the life of the CPAP tube.

CAUTION: Prolonged exposure to sunlight may cause the CPAP Tube to discolor and degrade.

Replacement CPAP tubes may be purchased directly from Human Design Medical at (855) 436-8724 [(855) HDM-USA4] or by visiting www.hdmusa.com.

Manufactured for Human Design Medical, LLC

Reorder No. HD60-2010

Mfg. No. AG36825

