

# Slim Style CPAP Tubing Instructions

## Installation & Removal

Be sure to follow your CPAP device manufacturer's instruction for the proper installation and removal of the CPAP tubing.

## Maintenance & Cleaning

### Daily

Remove the CPAP tube from your device and mask. Hang it in a clean, dry place out of direct sunlight. Reattach to your device and mask before next use.

### Weekly

Remove the CPAP tube from your device and mask. Wash the CPAP tube in warm water using mild soap. Rinse thoroughly, hang, and allow to dry.

**NOTE:** Soaking the CPAP Tube in water for longer than 10 minutes may shorten the life of the tube.

### Every 90 days

The CPAP tube should be replaced every 90 days or earlier if it becomes discolored, cracked or damaged.

**CAUTION:** Do not use bleach, chlorine, alcohol, or aromatic-based solutions, moisturizing or antibacterial soaps or scented oils to clean the CPAP tube. Never attempt to clean the CPAP tube with any of these solutions. These solutions may create a shock hazard, cause damage and reduce the life of the CPAP tube.

**CAUTION:** Prolonged exposure to sunlight may cause the CPAP Tube to discolor and degrade.

Replacement CPAP tubes may be purchased directly from Human Design Medical at (855) 436-8724 [(855) HDM-USA4] or by visiting [www.hdmusa.com](http://www.hdmusa.com).

Manufactured for Human Design Medical, LLC

Reorder No. HD60-2010

Mfg. No. AG36825

 **Respsshop**  
better sleep, better life.  
**Click Here To Buy**

