

**Click Here To Buy** Installing Your Transcend/Z1 Adapter


1. Remove the hose from your Transcend or Z1 device.



2. Install the Transcend/Z1 SoClean adapter onto the CPAP hose.



3. Connect the SoClean injection hose to the Transcend/Z1 SoClean adapter.

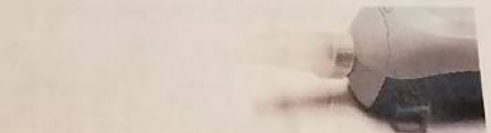

4. Place mask into SoClean chamber or SoClean 2 Go sanitizing bag and run sanitizing cycle.



5. Once the sanitizing cycle is complete and you are ready to use your Transcend or Z1 CPAP please remove the Transcend/Z1 SoClean adapter from your CPAP hose and reconnect your CPAP hose to the CPAP device.

