



Fitting guide for AirFit N20



- 1
- Start by ensuring the magnets on the lower straps are unclipped.
 - If they are clipped, detach them by twisting and pulling them away from the frame.



- 2
- Pull the headgear over the head, making sure the logo is facing outward.



- 3
- Position the lower straps so they both sit below the ears.
 - Bring each of the magnets up to meet their corresponding clips on the frame.



- 4
- Adjust the fastening tabs on the upper straps until the mask feels even and stable.
 - Reattach the fastening tabs.



- 5
- Repeat step 4 with the lower straps.
 - Reattach the fastening tabs.



- 6
- Connect the device's air tube to the elbow, then click the elbow into the front of the mask.
 - Start the device.



Refer to the user guide for further instructions and complete labeling information including indications, contraindications, warnings and precautions.

¹ ResMed AirFit N20 internal international fitting study of new and existing CPAP patients, conducted Nov 2015.

Distributed by ResMed Corp, 9001 Spectrum Center Boulevard, San Diego, CA 92123 USA +1 858 836 5000 or 1 800 424 0737 (toll free). See ResMed.com for other ResMed locations worldwide. AirFit and InfinitySeal are trademarks and/or registered trademarks of the ResMed family of companies. Specifications may change without notice. © 2016 ResMed Ltd. 1019410/2 2016-10