


# ResMed



## Disassembly for AirFit™ P10



• Holding the top of the mask tubing, squeeze and pull the pillows away from the mask frame.

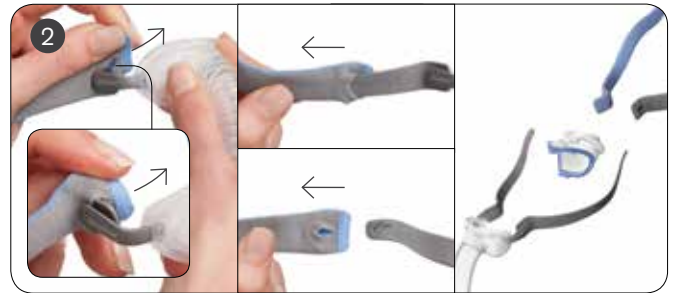

• Remove the headgear from the stabilizers by holding the fabric end of each strap and pulling out to reveal the buttonhole, then sliding over the node and off the end of each stabilizer. Your mask is now disassembled.


## Reassembly for AirFit P10



• With the gray side facing out, insert each end of the headgear strap over the stabilizers through the buttonholes. Secure in place by wrapping each strap end up and around the nodes. Holding the split straps, pull the headgear from the stabilizer to ensure it's tightly secure around the nodes. Ensure the headgear strap isn't twisted.



• Attach the pillows to the mask frame by inserting the shorter clip into the top slot, and the longer clip into the bottom slot on the mask frame. Your mask is now reassembled.

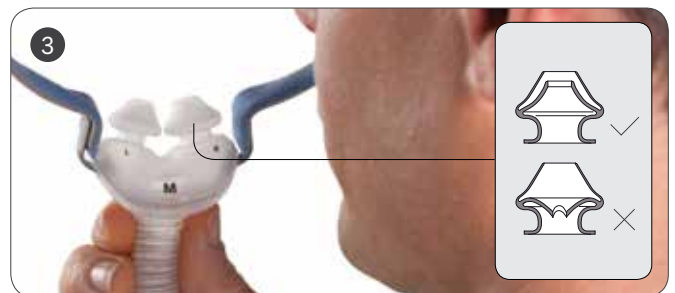

• Ensure the inner walls of the nasal pillows are not compressed or collapsed. If they are, squeeze each nasal pillow so they return to the correct shape.

**Note:** For complete user instructions, refer to the AirFit P10 User Guide.

**Rx Only**

**ResMed Ltd**  
1 Elizabeth Macarthur Drive,  
Bella Vista, NSW 2153 Australia  
MANUFACTURER

**DISTRIBUTED BY**  
**ResMed Corp**  
9001 Spectrum Center Boulevard, San Diego, CA 92123 USA

AirFit is a trademark of ResMed Ltd. Specifications may change without notice.  
© 2013 ResMed Ltd. 1016812/1 2013-10

[www.resmed.com](http://www.resmed.com)

**AirFit™** P10  
Nasal Pillows System