



PHILIPS

RESPIRONICS

Amara View

Minimal contact


Full face mask



Respshop
better sleep, better life.

[Click Here To Buy](#)

High marks for **performance**,
no marks on the bridge of
your nose

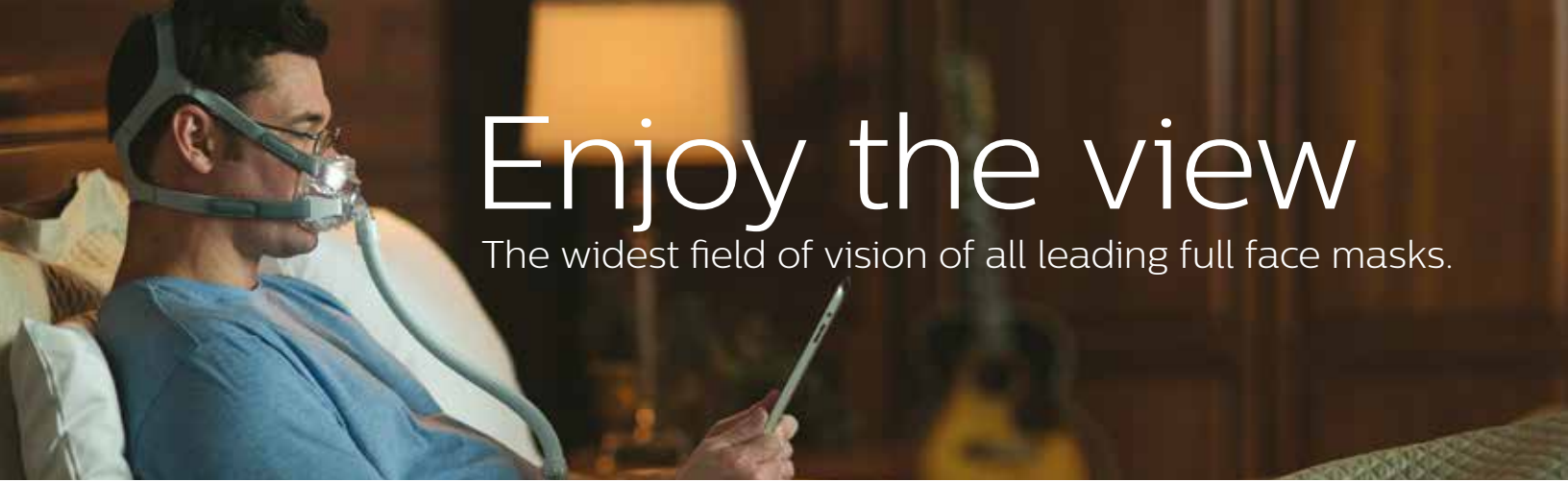


Enjoy the view

The widest field of vision of all leading full face masks.

An idea whose time has come. The Amara View's incredible under-the-nose design makes it the only full face mask that won't cause red marks on the bridge of your nose when compared to the market leaders. Amara View is also the smallest, lightest[†] and has the widest field of vision of all leading full face masks. It's easy to wear glasses, read, watch TV, and use a computer or tablet before falling asleep.*

Features

- Innovative design prevents red marks, discomfort or irritation on the nose bridge*
- The smallest and lightest[†] of all leading full face masks
- Widest field of vision of all leading full face masks*
- Enables wearing glasses, reading, and watching TV
- Easy to use with one modular frame for all cushion sizes
- No bulky cushion and frame in front of your face and eyes

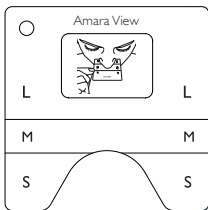

Unlike most full face masks, Amara View's cushion size is determined by measuring the depth of the nose using the simple sizing gauge.



Amara View covers less of a patient's face than all leading full face masks*

With its amazingly innovative design, only Philips Respironics can offer the smallest and lightest choice with the widest field of vision of all leading full face masks.

*Internal testing; data on file

† Without quick release tube

Get started

To learn more about how Amara View can start making a difference for your patients and your business, contact your Philips Respironics representative for more information.

| Item | Part number | HCPCS code |
|---|-------------|--------------|
| Small Amara View mask with headgear | 1090622 | A7030, A7035 |
| Medium Amara View mask with headgear | 1090623 | A7030, A7035 |
| Large Amara View mask with headgear | 1090624 | A7030, A7035 |
| Small Amara View mask without headgear | 1090632 | A7030 |
| Medium Amara View mask without headgear | 1090633 | A7030 |
| Large Amara View mask without headgear | 1090634 | A7030 |

| Replacement parts | Part number | HCPCS code |
|--------------------------------|-------------|------------|
| Amara View quick-release tube | 1090689 | N/A |
| Small Amara View cushion | 1090692 | A7031 |
| Medium Amara View cushion | 1090693 | A7031 |
| Large Amara View cushion | 1090694 | A7031 |
| Amara View headgear, large | 1090696 | A7035 |
| Amara View headgear, standard | 1090697 | A7035 |
| Amara View headgear clip, 2 pk | 1090698 | N/A |
| Amara View sizing gauge | 1090699 | N/A |

The ANSWER

Transform your business the easier way

The ANSWER, was designed to respond to the unprecedented challenges facing DME providers today. This easy, 3-phase process impacts every touch point in the patient journey, from set-up, to compliance, through ongoing resupply. The Amara View full face mask, a part of the ANSWER, is our latest innovation in mask technology developed to work together with our therapy systems and resupply services to transform your business the easier way.

Amara View fitting guide

Achieving the right fit with Amara View is easy; just follow the instructions below to get started. Please read OIS for further fitting instructions.

1



Loosen headgear to a large setting, then grip and twist the headgear clips away from the frame to disconnect.

2



Hold the mask against your face. Place the mask under your nose and over the cushion nostril opening. Push the mask cushion up.

(Note: Do not place the mask over your nose or place your nose in the cushion nostril opening.)

3



Pull the mask over your head.

4



Press the headgear clips into place to connect to mask.

5



Pull back the tabs on the headgear to evenly adjust the top and bottom straps. Pull back the tab to adjust the crown strap.

6



Connect the flexible tubing (included with your CPAP or bi-level device) to the Amara View quick release tube.

Learn more

Find out more about why Amara View is the new choice in full face masks
Visit www.philips.com/amaraview



©2015 Koninklijke Philips N.V. All rights reserved.
Specifications are subject to change without notice.
Trademarks are the property of Koninklijke Philips N.V.
(Royal Philips) or their respective owners.

www.philips.com/amaraview

Caution: US federal law restricts these devices to
sale by or on the order of a physician.

RRDPGH MG 3/10/15 MCI 4106634 PN 1122200
1010 Murry Ridge Lane, Murrysville, PA 15668
800-345-6443 • 724-387-4000