

CoughAssist 70 Series Circuit, Patient Interface

This CoughAssist 70 Series Circuit is compatible with the following Philips Respiration devices: CoughAssist T70 and CoughAssist E70

This CoughAssist 70 Series Circuit includes:

- Disposable circuit tubing
- Disposable bacteria filter
- Face mask
- Adapter, 22mm x 22mm (22mm x 15mm included with infant mask)


Please refer to the manufacturer's instructions regarding appropriate use of all applicable accessories.

Warnings

- Before initiating device therapy, confirm the proper connection of the circuit, and verify device operation. Patient, device, and circuit must be monitored on a regular basis per established standards of care.
- Do not pull or stretch the circuit tubing. This could result in circuit leaks.
- Do not attempt to sterilize the patient circuit. Always use a new bacteria filter when using the device on a new patient.
- This disposable circuit is for single patient use only.
- For children and those who are pregnant or breast feeding: This product may contain chemicals (phthalates) which may cause adverse health effects. Healthcare professionals should discuss the implications of using this product.

Assembly

Attach the appropriate patient interface to the flexible tubing. Patient interface options include a face mask and adapter. A sample breathing circuit assembly is shown.

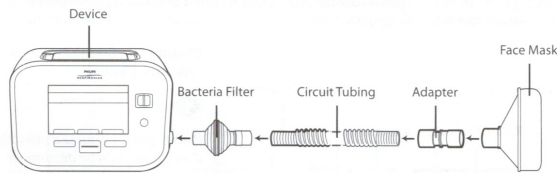

CoughAssist 70 series Circuit, Patient Interface



Notes: The length of time a circuit is used should be determined by the appropriate personnel, based on accepted infection control procedures.

A low-resistance main flow bacteria filter is recommended. The filter should be placed between the device and the circuit tubing.

Refer to your device manual for instructions on how to attach the circuit to the device.

Patient interface options include a face mask that is available either for infant, toddler, or adult sizes (small, medium, or large).

Reorder Numbers

REF	1090830 - CoughAssist 70 Series, Circuit, Infant, 6ft
REF	1098403 - CoughAssist 70 Series, Circuit, Infant, 9ft
REF	1090831 - CoughAssist 70 Series, Circuit, Toddler, 6ft
REF	1098404 - CoughAssist 70 Series, Circuit, Toddler, 9ft
REF	1090832 - CoughAssist 70 Series, Circuit, Adult, Small, 6ft
REF	1098405 - CoughAssist 70 Series, Circuit, Adult, Small, 9ft
REF	1090833 - CoughAssist 70 Series, Circuit, Adult, Medium, 6ft
REF	1098407 - CoughAssist 70 Series, Circuit, Adult, Medium, 9ft
REF	1090834 - CoughAssist 70 Series, Circuit, Adult, Large, 6ft
REF	1098408 - CoughAssist 70 Series, Circuit, Adult, Large, 9ft

Distributed by:
Respiration Inc.
1001 Murry Ridge Lane
Murrysville, PA 15668 USA

Respiration Deutschland
Gewerbstrasse 17
82211 Herrsching, Germany



1099090 R01
CV 02/27/2012

Respiration
better sleep, better life.
Click Here To Buy