FLEXIFIT™ 406 PETITE NASAL MASK PATIENT INSTRUCTIONS FOR USE

THE FLEXIFIT™ 406 PETITE NASAL MASK IS INTENDED FOR MULTIPLE PATIENT OR SINGLE PATIENT ADULT USE BY INDIVIDUALS WHO HAVE BEEN DIAGNOSED BY A PHYSICIAN AS REQUIRING CPAP OR BI-LEVEL VENTILATOR TREATMENT.

FITTING YOUR MASK

- Remove plastic cover from Silicone Seal.
- Holding your mask in one hand, spread the Stretchgear™ Headgear (H) with your free hand just enough to fit comfortably over your head.
- Lightly hold the mask against your face, covering the nose, then slowly pull the Headgear over your head.
- Attach the Headgear to the Glider™ Strap (E).
- Pull the lower straps as low as possible down back of neck. Gently tighten straps to remove slack. Connect to CPAP/Bi-Level machine set at prescribed pressure.
 - If necessary, tighten the lower straps to eliminate any leaks (avoid over-tightening the straps).
 - When you need to remove your mask, unhook the Stretchgear™ Headgear from the Glider™ Strap.

FITTING TIPS

- Forehead pads should rest gently on the forehead without top straps being pulled too tight.
- If there are any leaks after gently tightening the top and lower straps, pull the mask slightly out from the face while straps are still attached. This will allow the Silicone Seal to inflate with air. Put the mask back onto the face.

CLEANING YOUR MASK

To ensure continuing safety and comfort for your FlexiFit™ 406 Petite Nasal Mask:

- BEFORE FIRST USE:**
 - Hand-wash the Stretchgear™ Headgear (H) in pure soap dissolved in lukewarm water. Do not soak for longer than 10 minutes, then rinse in fresh water and allow to dry out of direct sunlight.
- BEFORE EACH USE:**
 - Inspect the mask for deterioration. Do not use if damaged.
- AFTER EACH USE:**
 - Disassemble the Silicone Seal (C), Foam Cushion (D) and the Headgear (H) from the Mask Base (A).
 - Wash the Silicone Seal and Mask Base in pure soap dissolved in lukewarm water. Do not soak for longer than 10 minutes, then rinse in fresh water.
 - Wipe the Foam Cushion with a damp cloth. (Do not immerse the Foam Cushion in water)
 - Leave all parts to dry out of direct sunlight before reassembling.
- EVERY 7 DAYS:**
 - Hand-wash the Stretchgear™ Headgear (H) in pure soap dissolved in lukewarm water. Do not soak for longer than 10 minutes, then rinse in fresh water and allow to dry out of direct sunlight.

CAUTION

- Do not soak the mask for longer than 10 minutes.
- Do not wash the mask in the dishwasher.
- Do not clean the mask with products containing alcohol, anti-bacterial agents, antiseptic, bleach, chlorine or moisturizer.
- Do not store the mask in direct sunlight.

The above actions may deteriorate or damage the mask and shorten its life. If your FlexiFit™ 406 Petite Nasal Mask weakens or cracks, discontinue use and seek replacement immediately.

ASSEMBLING YOUR MASK

Your FlexiFit™ 406 Petite Nasal Mask is fully assembled and ready to use. However, when you need to clean your mask follow these instructions for reassembly.

- Place Silicone Seal (C) face down onto flat surface. With thick side of Foam Cushion (D) facing down, place into Silicone Seal.
- Starting with the slot at the apex of the Silicone Seal, assemble onto Mask Base (A). Push firmly together.
- Lay the Stretchgear™ Headgear (H) flat, with the loop side down. Position the assembled mask over the Headgear and attach the four straps into the corresponding slots in the Forehead Rest (F) and the Glider™ Strap (E). This can be done without undoing the VELCRO® brand tabs*, by sliding the Headgear into the slots.

*VELCRO® is a registered trademark of Velcro Industries BV.

OXYGEN/PRESSURE PORT CONNECTOR

If pressure readings and/or additional oxygen are required, an Oxygen/Pressure Port Connector is available (Code 900HC452).

- Connect the large end of the Connector (T) to the Swivel (G) on the mask and the small end of the Connector to the main breathing tube (R).
- Open the Port Cap(s) (P) and place the Pressure and/or Oxygen Tube (S) firmly over the Port(s) (Q).

(The two ports are identical; either port may be used for oxygen or pressure measurement. Only remove the Port Cap from ports that are being used.)

- Note:** At a fixed flow rate of supplemental oxygen flow, the inhaled oxygen concentration will vary, depending on the pressure settings, patient's breathing pattern, mask selection, and the leak rate.
- Warning:** If oxygen is used with CPAP/Bi-Level machine, the oxygen flow must be turned off when the CPAP/Bi-Level machine is not operating.
- Explanation of Warning:** When the CPAP/Bi-Level device is not in operation, and the oxygen flow is left on, oxygen delivered into the ventilator tubing may accumulate within the CPAP/Bi-Level machine enclosure. Oxygen accumulated in the CPAP/Bi-Level machine enclosure will create a risk of fire.
- Warning:** Do not smoke while using or in the vicinity of someone using oxygen with this mask.
- Warning:** Do not use Polyvinyl Chloride (PVC) Pressure and/or Oxygen tubing. Use silicone tubing instead.
- Explanation of Warning:** Contact with PVC can cause premature cracking of ports.

WARRANTY AND DISPOSAL

WARRANTY STATEMENT

Fisher & Paykel Healthcare warrants that the mask (excluding foams and material diffusers), when used in accordance with its instructions for use, shall be free from defects in workmanship and materials and will perform in accordance with Fisher & Paykel Healthcare's official published product specifications for a period of 90 days from the date of purchase by the end user. This warranty is subject to the limitations and exceptions set out in detail in <http://www.fphcare.com/osa/productwarranty.asp>

DISPOSAL

This device does not contain any hazardous substances and may be disposed of with general waste.

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