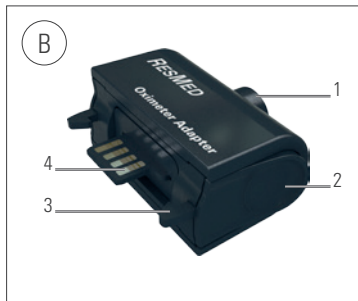
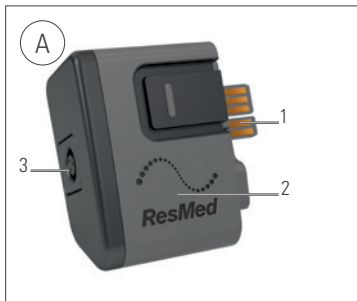
ResMed

Oximeter adapter

User guide

English | Deutsch | Français | Italiano | Español | Português | Nederlands
Svenska | Dansk | Norsk | Suomi | Česky | Eesti | 简体中文 | 繁體中文

Thank you for choosing a ResMed oximeter adapter. The oximeter adapter connects a NONIN Xpod™ oximeter to your device. Oximetry data (oxygen saturation and pulse rate) is recorded on the SD card inserted into the device.

The Air10 oximeter adapter shown in illustration A is compatible with AirSense™ 10 / AirCurve™ 10 devices. The oximeter adapter shown in illustration B is compatible with S9 devices.

Read the entire guide before using the oximeter adapter.

Air10 oximeter adapter

Refer to illustration A:

1. Adapter connector
2. Release button
3. Oximeter port.

AirSense 10 / AirCurve 10

Refer to illustration C:

1. SD Card cover
2. Air filter cover
3. Adapter cover
4. Adapter port.

S9 oximeter adapter

Refer to illustration B:

1. Oximeter port
2. Release buttons
3. Adapter clip
4. Adapter connector.

S9

Refer to illustration D:

1. DC power inlet
2. Adapter clip socket
3. Adapter port
4. SD card slot
5. Alarm (Bilevel alarmed devices only).

Associated accessories

- Xpod oximeter
- Xpod fixation clip
- Finger pulse sensor
- Oximetry accessories pack (including Xpod oximeter, Xpod fixation clip and 3m oximeter soft sensor reusable).

Save the packaging

Keep all the packaging the oximeter adapter came in. You will need to return the oximeter adapter, this User Guide and the SD card to your care provider.

Setup

Refer to illustration E or F:

1. Unplug your device from the power outlet.
2. Make sure an SD card is inserted into the SD card slot on your device.
3. Connect the oximeter adapter to your device.
4. Align the Xpod oximeter connector with the oximeter adapter port and connect them.

Not pictured:

5. Connect the Xpod oximeter to the finger pulse sensor.
For instructions refer to the User Guide supplied with the finger pulse sensor.
6. Connect the finger pulse sensor to your finger.
7. Plug your device back into the power outlet.
The AirSense 10 / AirCurve 10 device will briefly display the oximeter software code, eg, SX123-4567.
The S9 device will briefly display Oximeter enabled.

You are now ready to start therapy. For more information, refer to your device User Guide.

Removing and returning the oximeter adapter and SD card

1. Remove the sensor from your finger.
2. Unplug your device from the power outlet.
3. Press the release button/s on the side of the oximeter adapter and pull gently to remove it from the device.
4. Pull the Xpod oximeter out of the oximeter port on the oximeter adapter.
5. Remove the finger sensor cable from the Xpod oximeter.
6. Open the SD card cover and push in the SD card to release it.
7. Place the SD card in the protective folder and send it back to your care provider together with the oximeter adapter and this User Guide.

Cleaning and maintenance

If you need to, wipe the outside of the oximeter adapter with a clean, dry cloth. For instructions on how to clean the oximeter and its sensor, refer to the User Guide supplied with those accessories.

Troubleshooting

If you have any problems, refer to the following troubleshooting topics. If you are not able to fix the problem, contact your care provider or refer to www.resmed.com. Do not try to open the oximeter adapter.

Problem/possible cause	Solution
AirSense 10 / AirCurve 10: Check your finger sensor or press OK to continue without oximetry	
S9: Check the oxi sensor or press dial to accept no oximetry data logging	
The finger pulse sensor is not attached to your finger	Make sure the finger pulse sensor is properly attached according to the instructions supplied with that accessory.
The sensor cable is not connected to the Xpod oximeter at the start of therapy.	Disconnect and reconnect the Xpod oximeter connector and sensor cable according to the instructions supplied with these accessories.
AirSense 10 / AirCurve 10: Check oximeter connection or press OK to continue without oximetry	
S9: Connect the oxi sensor or press dial to accept no oximetry data logging	
The oximeter adapter is not connected properly or data is not transmitting correctly.	Remove your oximeter adapter from the device according to the instructions. Reattach your oximeter adapter and plug your device back into the power outlet.
S9: Insert your SD Card or press dial to accept no oximetry data logging	
The device cannot detect an SD card and no oximetry data can be logged or stored.	Make sure that an SD card is correctly inserted in the SD card slot of your device.

Technical specifications

Operating temperature	+41°F to +95°F (+5°C to +35°C)
Operating humidity	10 to 95% non-condensing
Operating altitude	Refer to your device User Guide for details.
Storage and transport temperature	-4°F to 140°F (-20°C to +60°C)
Storage and transport humidity	
Air10	5 to 95% non-condensing
S9	10 to 95% non-condensing
Design life	
Air10	1825 connections
S9	1825 connections
Gross dimensions	
Air10 (W x D x H)	2.1" x 1.2" x 2.0" (54 mm x 30 mm x 51 mm)
S9 (L x W x H)	1.74" x 2.12" x 0.91" (44.35 mm x 54.10 mm x 23.30 mm)
Weight	
Air10	0.95 oz (27 g)
S9	0.77 oz (22.1 g)

Oximeter compatibility

Air10

XPod 3012, XPod 3012LP

S9

Xpod 3012

Electromagnetic compatibility

Refer to your device User Guide for details.

Note: The manufacturer reserves the right to change these specifications without notice.

Symbols

The following symbols may appear on the oximeter adapter or packaging:

 Caution.  Read instructions before use.  Manufacturer.

 European Authorized Representative.  Batch code.  Catalog number.  Serial number. **IP21** Protection against insertion of fingers and

against vertically dripping water. **IP22** Protected against finger sized objects and against dripping water when tilted up to 15 degrees from specified orientation.

 Humidity limitation.  Temperature limitation.  China pollution control logo.  Canadian Standards Association.  Type BF equipment.

Notes:

- IP21 rating applies only to S9 oximeter adapter.
- IP22 rating applies only to Air10 oximeter adapter.

Environmental information

This device should be disposed of separately, not as unsorted municipal waste. To dispose of your device, you should use appropriate collection, reuse and recycling systems available in your region. The use of these collection, reuse and recycling systems is designed to reduce pressure on natural resources and prevent hazardous substances from damaging the environment.

If you need information on these disposal systems, please contact your local waste administration. The crossed-bin symbol invites you to use these disposal systems. If you require information on collection and disposal of your ResMed device please contact your ResMed office, local distributor or go to www.resmed.com/environment.

WARNING

Do not open or modify the oximeter adapter. The oximeter adapter is not designed to be serviced. There are no serviceable parts inside.

CAUTION

- In the US, Federal law restricts this device to sale by or on the order of a physician.
- Only use the oximeter adapter as specified in this guide.
- S9 oximeter adapters are specifically designed for use with ResMed S9 devices.
- Air10 oximeter adapters are specifically designed for use with ResMed AirSense 10 / AirCurve 10 devices.
- Do not use liquids to clean any part of the oximeter adapter.

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Air10, AirSense, AirCurve, ResScan and S9 are trademarks of ResMed Ltd. S9 is registered in U.S. Patent and
Trademark Office. © 2014 ResMed Ltd. 378400-ENG/1 2014-07
ResMed Ltd
ResMed Corp 9001 Spectrum Center Boulevard San Diego CA 92123 USA
EC REP ResMed (UK) Ltd 96 Jubilee Ave Milton Park Abingdon