

RESMED

 **ResPshop**
better sleep, better life.
Click Here To Buy

Quattro Air

The lighter way to better sleep



The Quattro Air is the lightest full face mask on the market. With only four parts, it is 45% lighter and is even more comfortable than the Mirage Quattro—our leading full face mask. Because it builds on the proven reliability and performance of the Quattro Series, you can be confident it will deliver a comfortable therapy experience.



Quattro™ Air
FULL FACE MASK

ResMed.com/QuattroAirVideos



Quattro Air

The Quattro Air delivers the proven performance and reliability of the Mirage Quattro™ in a lighter, more comfortable mask that is simpler to set up and use.

Ideal for your patients

With its lightweight design and only four parts, the Quattro Air appeals to new and existing patients. Because it's incredibly easy to use, clean and assemble on a regular basis, they can focus on getting a good night's sleep.

Ideal for your business

While your patients enjoy comfortable therapy, you can enjoy the benefits of long-term compliance: easier setups, reduced callbacks, less time spent on troubleshooting and more replenishment business.



The Quattro Air for Her features a stylish headgear and fit range specifically for women



SoftEdge™ headgear design features premium rolled-edge fabric that improves comfort and reduces facial marks

Circular diffuse vent gently directs air away for less patient and bed partner disturbance

One-piece elbow, swivel and anti-asphyxia valve with squeeze-tabs make attaching and detaching quicker and easier

Swivel reduces tube drag, giving patients the freedom to move during sleep without compromising seal



Flex-wing forehead support enables auto-adjustments and eliminates forehead pads for greater comfort and lighter feel

Improved comfort and less chance for skin irritation at the sensitive nasal bridge area

Enhanced dual-wall Spring Air™ cushion is the next-generation design that delivers a high-performance seal with even more comfort at nasal bridge area; clipless cushion-to-frame attachment is easy to assemble

Headgear loops simplify attachment for improved usability

Quattro Air and Quattro Air for Her components



Elbow with swivel

Frame

Cushion
62736 (XS)
62737 (S)
62738 (M)
62739 (L)

Frame system

Includes frame, cushion, and elbow with swivel.

- 62752 (XS)
- 62753 (S)
- 62754 (M)
- 62755 (L)



Quattro Air headgear
62757 Small
62756 Standard



Quattro Air for Her headgear
62758 Small
62759 Standard

PRODUCT CODES

Complete System

Quattro Air Full Face Mask

Style	Part Number
Quattro Air mask system Small	62701
Includes frame, small cushion, elbow with swivel, and standard blue headgear	
Quattro Air mask system Medium	62702
Includes frame, medium cushion, elbow with swivel, and standard blue headgear	
Quattro Air mask system Large	62703
Includes frame, large cushion, elbow with swivel, and standard blue headgear	
Quattro Air for Her mask system X-Small	62740
Includes frame, X-small cushion, elbow with swivel, and small pink headgear	
Quattro Air for Her mask system Small	62741
Includes frame, small cushion, elbow with swivel, and small pink headgear	
Quattro Air for Her mask system Medium	62742
Includes frame, medium cushion, elbow with swivel, and small pink headgear	

Reimbursement Codes

HCPCS	Code Descriptor
A7030	Full face mask used with positive airway pressure device, 1 per 3 months
A7031	Replacement cushion for full face mask, 1 per month
A7035	Headgear used with positive airway pressure device, 1 per 6 months

Quattro™ Air
FULL FACE MASK

[ResMed.com/QuattroAirVideos](https://www.resmed.com/QuattroAirVideos)



Encourage your patients to join Wake Up to Sleep—ResMed's sleep apnea support community—at WakeUpToSleep.com/Join.

Quattro Air



- Remove the elbow from the mask by pressing the squeeze tabs and pulling the elbow away.
- Pull headgear over your head with both lower headgear straps released.



- Adjust both the straps of the upper headgear evenly until the mask is stable.



- Bring the lower straps below your ears and loop the headgear into the lower hooks on the frame.



- Adjust the lower headgear straps evenly until the mask is stable.
- The fitting should be firm but not uncomfortable.
- Do not over tighten.



- Connect the air tubing from your device to the elbow.
- Attach the elbow to the mask by pressing the squeeze tabs and pushing the elbow into the mask.
- Turn on your device so it blows air.



- Undo the fastening tabs on the upper headgear straps.
- Reposition both of the straps evenly, only enough until a seal is achieved, then reattach the fastening tabs.
- Your mask is now ready to use.



Scan here to see fitting and care videos

www.resmed.com/QuattroAirVideos



S9 Series
More. Comfort.

Combine a Quattro Air mask with ResMed's S9 AutoSet™ therapy device for a comfortable, quiet and lifestyle-friendly system.

Quattro™ Air
FULL FACE MASK

ResMed.com/QuattroAirVideos