RESMED

Respshop
better sleep, better life.
Click Here To Buy

S9™ USB Adapter

DATA TRANSFER ACCESSORY

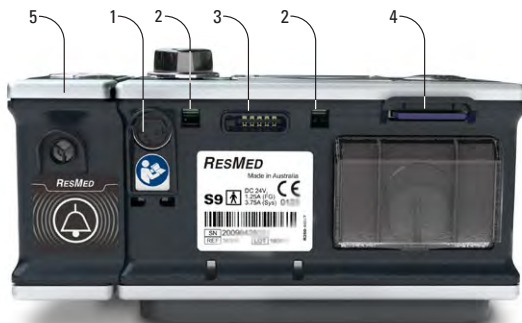
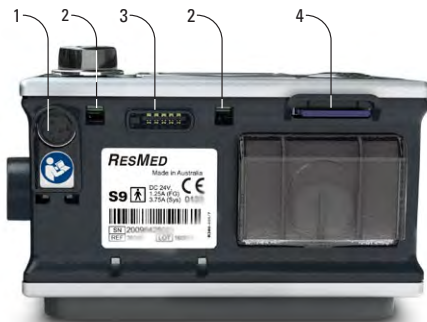
User Guide

English • Deutsch • Français • Italiano • Español • Português • Nederlands •
Svenska • Dansk • Norsk • Suomi • 简体中文 • 繁體中文

A

B

C

D

Thank you for choosing the ResMed S9 USB adapter. Used with ResMed S9 devices, the S9 USB adapter transmits patient therapy data between the device and a personal computer (PC) for compliance monitoring.

Please read the entire manual before using this device.

S9 USB adapter at a glance

The S9 USB adapter comprises the following elements:

- S9 USB adapter
- USB cable.

S9 USB adapter

Refer to illustration A.

1. USB cable connector port
2. Release button
3. Adapter clip
4. Adapter connector

Rear panel of S9 device

Refer to illustration B.

1. DC power inlet
2. Adapter clip socket
3. Module/Adapter port
4. SD card slot
5. Alarm (Bilevel alarmed devices only)

Setup

1. Ensure the PC intended for use has ResMed's patient management software installed (EasyCare, or ResScan v3.8 or greater).
2. Disconnect the S9 device from the power outlet.
3. Align the S9 USB adapter clips with their corresponding sockets and push the adapter onto the S9 device [see illustrations C and D]. Ensure the adapter clips click into place.
4. Connect one end of the USB cable to the USB adapter and the other end to the PC.

5. Reconnect the S9 device to the power outlet.

Patient data can now be downloaded from the S9 device to the PC via EasyCare or ResScan. Refer to the EasyCare or ResScan online help for further details. Once the data has been downloaded, the USB adapter can be removed.

Removing the USB adapter

1. Disconnect the S9 device from the power outlet.
2. Press and hold in the release buttons located on each side of the adapter while removing it from the S9 device.

Cleaning and maintenance

If necessary, wipe the outside of the S9 USB adapter with a clean, dry cloth.

Save the packaging

Please keep this user guide and all the original packaging for your S9 USB adapter.

Technical specifications

Operating temperature	+41°F to +95°F (+5°C to +35°C)
Operating humidity	10 to 95% non-condensing
Storage and transport temperature	-4°F to 140°F (-20°C to +60°C)
Storage and transport humidity	10 to 95% non-condensing
Gross dimensions (L x W x H)	1.65" x 2.12" x 0.91" (42.15 mm x 54.10 mm x 23.30 mm)
Weight	0.68 oz (19.5 g)
Electromagnetic compatibility	Refer to the S9 Information Guide for details

Note: The manufacturer reserves the right to change these specifications without notice.

Symbols which appear on the device

The following symbols may appear on the S9 USB adapter or packaging:

 Caution;  Read instructions before use;  IP21 Protection against insertion of fingers and against vertically dripping water;  Manufacturer;  LOT Batch Code;  EC REP European Authorised Representative;  RoHS European RoHS;  REF Catalogue Number;  SN Serial number;  CSA Canadian Standards Association;  UL Listed.

Note: IP21 rating of adapter applies without USB cable attached.

Environmental information

WEEE 2002/96/EC is a European Directive that requires the proper disposal of electrical and electronic equipment. These devices should be disposed of separately, not as unsorted municipal waste. To dispose of the device, you should use appropriate collection, reuse and recycling systems available in your region. The use of these collection, reuse and recycling systems is designed to reduce pressure on natural resources and prevent hazardous substances from damaging the environment.

If you need information on these disposal systems, please contact your local waste administration. The crossed-bin symbol invites you to use these disposal systems. If you require information on collection and disposal of your ResMed device, please contact your ResMed office, local distributor or go to www.resmed.com/environment.

WARNINGS

- The S9 device should be disconnected from the power outlet when attaching or detaching the S9 USB adapter.
- Beware of electrocution. Do not immerse the device or power cord in water.
- Always unplug the device before cleaning and make sure it is dry before plugging it back in.

CAUTION

- In the US, Federal law restricts this device to sale by or on the order of a physician.
- Only use the S9 USB adapter as specified in this guide.
- S9 USB adapters are specifically designed for use with ResMed S9 devices.
- Avoid exposing the S9 USB adapter to moisture or extreme temperatures.
- Do not use liquids to clean any part of the device.
- Do not open the S9 USB adapter. There are no user serviceable part inside. Repairs and internal servicing should only be performed by an authorized service agent.

Limited warranty

Please refer to the ResMed S9 Information Guide.

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© 2012 ResMed Ltd.
S9 and ResScan are trademarks of ResMed Ltd and S9 is registered in U.S. Patent and Trademark Office.
ResMed Ltd 1 Elizabeth Macarthur Drive Bella Vista NSW 2153 Australia
ResMed Corp 9001 Spectrum Center Boulevard San Diego CA 92123 USA
ResMed (UK) Ltd 96 Milton Park Abingdon Oxfordshire OX14 4RY UK