

## SimplyGo Mini Airline Travel Addendum

**PHILIPS**  
**RESPIRONICS**

The SimplyGo Mini device can be used when traveling by air. Before you leave, make sure you pack the following:

- Fully charged battery (and extra batteries for a long trip)
- AC power supply and connector cord
- DC power supply
- Carrying bag

Also, be sure to take the telephone numbers of your home care provider and physician in case of an emergency.

Every airline has policies for travel with portable oxygen concentrators such as your SimplyGo Mini device. Some airlines do not permit their use while flying on their aircraft. When you make your reservation, be sure to ask the airline for their policy regarding use of your SimplyGo Mini during flight. For updated information about air travel with your device, visit our web site: [www.philips.com](http://www.philips.com).


When flying in the U.S. and abroad, commercial airlines may have unusual requirements for oxygen concentrators. Before you fly, contact all of your airlines on each of your connecting flights for permission to carry your SimplyGo Mini system on board. Be sure to ask about any documentation you may need to bring with you.

Make sure your battery is fully charged and that you have enough battery power for the duration of your flight. On the day of your flight, plan to arrive early. Airport security officers may need extra time to clear your luggage and oxygen system. They may also ask you to remove the SimplyGo Mini device from the carrying case for inspection.



Complies with  
RTCA/DO-160G  
section 21, category M

The manufacturer of this POC has determined this device conforms to all applicable FAA requirements for POC carriage and use on board aircraft.

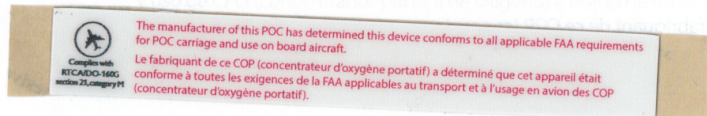


Respironics Inc.  
1001 Murry Ridge Lane  
Murrysville, PA 15668 USA



English

## SimplyGo Mini Airline Label Placement

**PHILIPS**  
**RESPIRONICS**

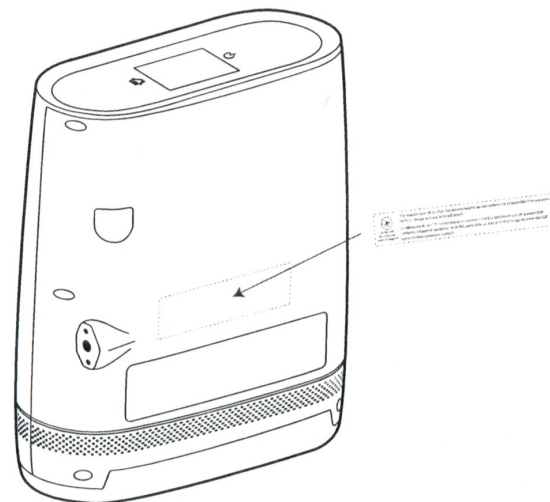

Respironics Inc.  
1001 Murry Ridge Lane  
Murrysville, PA 15668 USA



1127818 R01  
JH 06/15/16