
SleepStyle™ 600

CPAP SERIES

 **Respshop**
better sleep, better life.
[Click Here To Buy](#)

**Thermo
Smart™**

Operating Manual

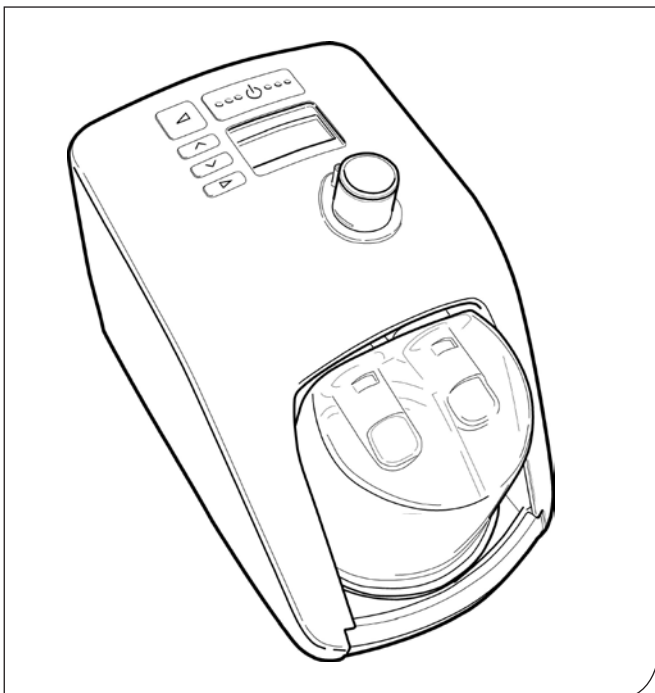Fisher & Paykel
HEALTHCARE

SECTION

English A

Français B

Español C

This product is covered by one or more of the following patents or patent applications in the corresponding countries listed below.

HC604 USA: US6598604, US6398197, US7111624, US6953354, US7086399, US7616871, Australia: AU761713, AU756477, AU2003204474, AU2003265025, AU2004266551, AU2009201906, Japan: JP4436758, Europe: EP1369141, EP1542757, EP1656173, EP1055431 (GB, IT), Hong Kong: HK1074178

HC608 USA: US6598604, US6398197, US7111624, US6953354, US7086399, US7616871, Australia: AU761713, AU756477, AU2003204474, AU2003265025, AU2004266551, AU2009201906, Japan: JP4436758, Europe: EP1369141, EP1542757, EP1656173, EP1055431 (GB, IT), Hong Kong: HK1074178

SleepStyle™ 600

CPAP SERIES

The SleepStyle™ 600 CPAP Series has integrated heated humidification and a heated breathing tube. It is designed for use in the home or sleep laboratory, for the treatment of Obstructive Sleep Apnea.

For further assistance, please contact your local Fisher & Paykel Healthcare office – see back cover for details.

Please keep this manual for future reference.

TABLE OF CONTENTS

1. Symbol Definitions	A - 2
2. Warnings, Cautions and Contraindications	A - 2
3. Description of the Device	A - 3
3.1 Package Contents	A - 3
3.2 Important Parts of the Device	A - 3
3.3 Accessories	A - 3
3.4 ThermoSmart™ Technology	A - 4
4. Intended Use	A - 4
5. Oxygen Use Warnings	A - 4
6. Cleaning and Maintenance	A - 5
7. Setup Instructions	A - 5
8. Controls and Display	A - 6
8.1 Display Description and Functions	A - 6
8.2 Patient Menu Summary	A - 7
8.3 Patient Menu - Additional Settings Summary	A - 7
9. Frequently Asked Questions	A - 8
10. Operating Conditions	A - 8
11. Storage and Transport Conditions	A - 8
12. Troubleshooting	A - 8
13. Warranty	A - 8
14. Device and Consumables Disposal Instructions	A - 8
15. Product Specifications	A - 9
16. Appendix	A - 9

PLEASE READ ALL INSTRUCTIONS BEFORE INITIAL USE

Caution: USA Federal Law restricts this device for sale by or on the order of a physician.

1. SYMBOL DEFINITIONS

Type BF
Applied Part

ATTENTION
Consult
accompanying
documents

IPX1
Drip Proof

Alternating Current

Class II
Double Insulated

Power
On/Off

CE 0123

93/42/EEC
Class IIb

2. WARNINGS, CAUTIONS AND CONTRAINDICATIONS

NOTES

- This manual refers to the SleepStyle™ 600 CPAP Series unit as “the device”.
- If required to use CPAP by a referring physician, you should use your device every time you sleep. Should your device stop working for any reason, contact your healthcare provider immediately.
- The user of this system shall have sole responsibility and liability for any injury to persons or damage to property resulting from operation of the device which is not in accordance with the operating instructions supplied.
- The device should only be used with ISO 17510-2 compliant masks, connectors and delivery tubes recommended by Fisher & Paykel Healthcare, or your healthcare provider.
- Do not use the device when the room temperature exceeds 95 °F (35 °C) as this may lead to an elevated airflow temperature which may give rise to an irritation of the airways.
- Refer all repairs and maintenance to Fisher & Paykel Healthcare.

WARNINGS

To avoid electric shock from your device, do not:

- Operate the device if the power cord or plug is damaged.
- Operate the device if it has been dropped in water.
- Plug the device into the power socket if it is wet.
- Clean the device while connected to the power socket.
- Store or use the device where it can tilt, fall or be pulled into water. If water has entered the unit enclosure, disconnect the power cord and discontinue use. Seek advice from Fisher & Paykel Healthcare.

To avoid choking or inhalation of a foreign body:

- Never place any object into any opening of the tube.
- Ensure the air filter is fitted during device use.

To ensure optimal therapy, do not:

- Operate the device if dropped or damaged.
- Operate the device if it is not working properly.
- Adjust the pressure. Pressure must only be adjusted by a healthcare professional.
- Operate the device if the breathing tube has been damaged with holes, tears or kinks.
- Block the exhaust flow on the mask.
- Use the mask if the unit is turned off or not operating properly.
- Use an insulating sleeve or any similar accessories which are not approved by Fisher & Paykel Healthcare. These may affect the humidification performance.
- Try to remove the heated breathing tube without pushing the sleeve up first. This will avoid damage to the connection port.

To avoid burns, do not:

- Fill the chamber with boiling water.
- Touch the exposed heater plate or chamber base.
- Position the breathing tube in such a way that it can come into prolonged contact with skin.

To avoid the risk of fire while using oxygen, do not:

- Turn oxygen flow on when the device is not operating; this can lead to accumulation of oxygen within the device.
- Locate the device in a position where ventilation around the device is restricted.
- Use oxygen while smoking or in the presence of an open flame.
- Use any materials which will burn in air or ignite easily at high oxygen concentration.

- Keep any source of ignition near the product. To avoid ignition, it is preferable to keep all sources of ignition out of the room where supplemental oxygen is being used.
- Keep oxygen regulators, cylinder valves, tubing, connections and all other oxygen equipment near oil, grease or greasy substances. Spontaneous and violent ignition may occur if these substances come into contact with oxygen under pressure.
- Operate the device without the cover over the serial port adaptor.

Other:

- Place the device on a level surface lower than head height to prevent water entering the tubing. If water does enter the tubing, drain excess condensate. Water in the tubing may result in aspiration.
- The device is intended to be used with CPAP masks and connectors that have exhaust flow holes to allow continuous flow of air out of the mask. When the device is turned on and operating properly new air flushes exhaled air out of the mask through the exhaust flow holes. At low CPAP pressures and in the event of power failure or machine malfunction, remove the mask immediately, as flow through the mask may be insufficient to clear all exhaled gas and CO₂ re-breathing may occur which can be hazardous.
- Failure to select the correct altitude level (for any given location) will have an adverse effect on delivered pressure.
- This device is not intended for life support.
- Avoid unnecessary removal of the power cord from the rear of the device. If removal is necessary then hold the connector and remove. Avoid pulling on the power cord (refer to Section 3.2).

CAUTIONS

To prevent water damage to your device:

- Remove the humidification chamber from the device before filling.
- Empty water from the chamber before transporting the device. If the device is required to be handled with water in the chamber, avoid tilting the device to prevent water entering its enclosure.

Other:

- To prevent damage to your PC, only operate the device if it is connected to a PC via an isolated serial port adapter (900HC236).
- To prevent airway irritation, do not use the device when room temperature exceeds 95 °F (35 °C).

CONTRAINDICATIONS

- Research indicates the following pre-existing conditions may contraindicate the use of positive pressure for some patients: pneumothorax, bullous lung disease, pneumocephalus, cerebrospinal fluid leak, recent cranial surgery or trauma, abnormalities of the cribriform plate, pathologically low blood pressure or in patients whose upper airways are bypassed.

PRECAUTIONS

- The safety and effectiveness of positive pressure has not been established in patients with respiratory failure or COPD.


ADVERSE EFFECTS

- Nosebleeds, ear and sinus discomfort may occur from the use of positive pressure therapy.


Please contact your physician if you have any questions concerning your therapy.

3. DESCRIPTION OF THE DEVICE

3.1 Package Contents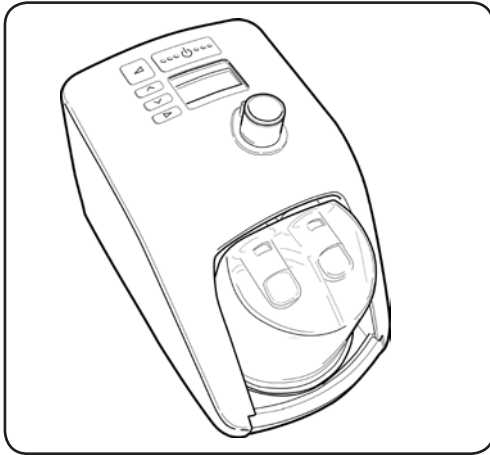


1 x SleepStyle™ 600 CPAP Series

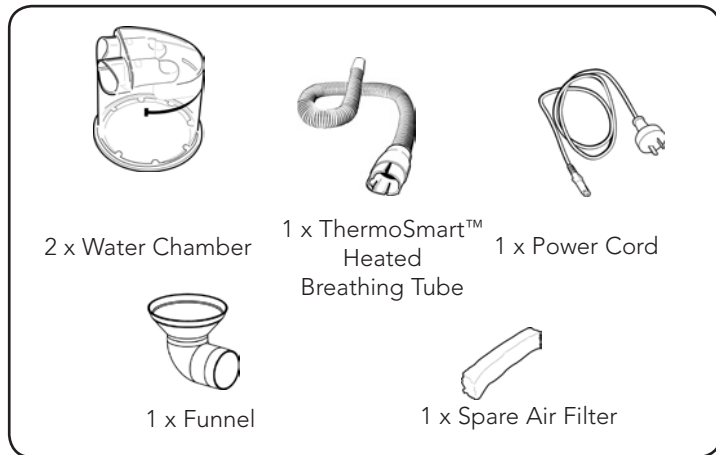Accessories

3.2 Important Parts of the Device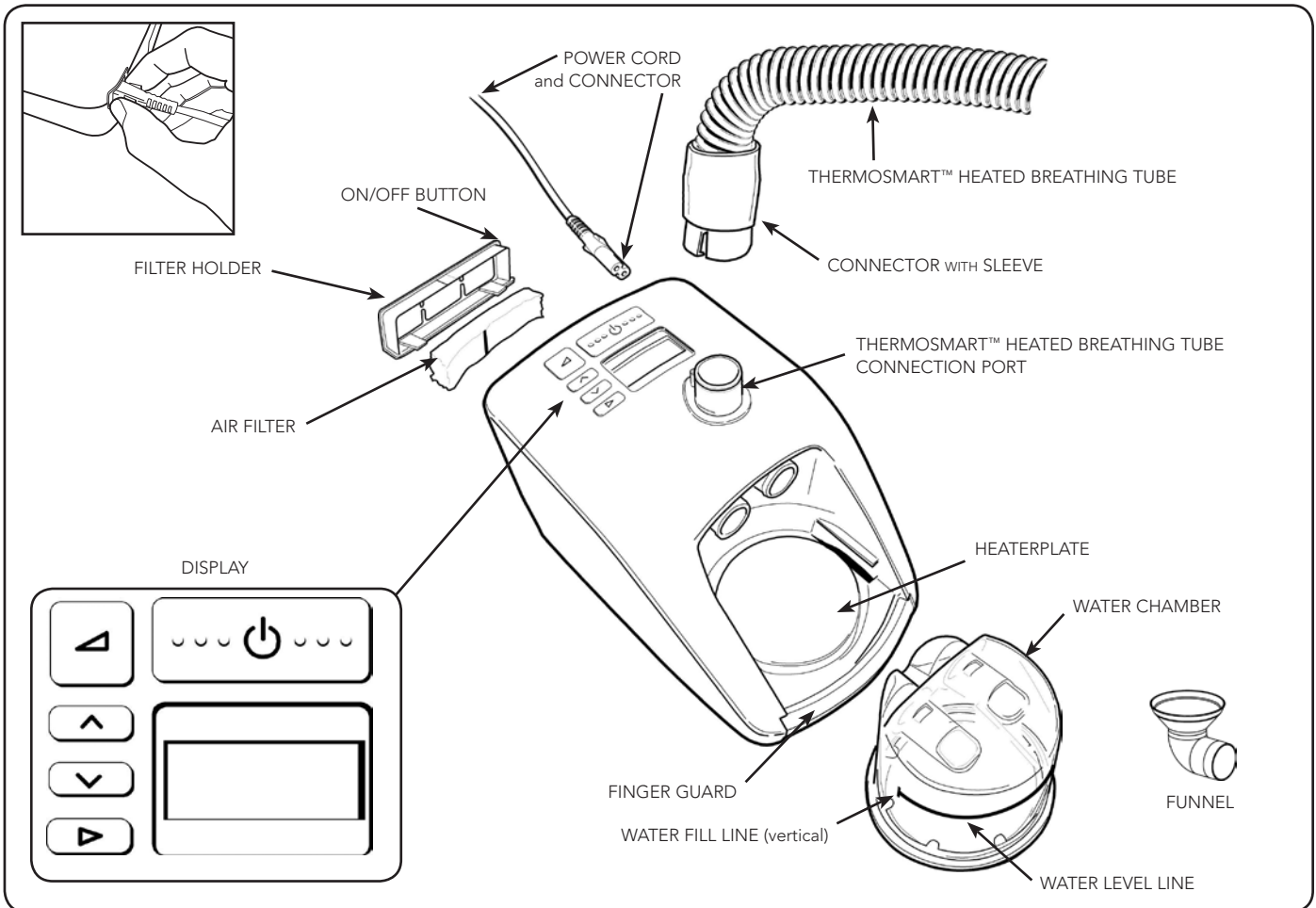

3.3 Accessories

HC365	Standard Humidification Chamber	900HC240	Filter (2-Pack)
HC360*	Extended Life Humidification Chamber	900HC604	Outlet Cleaning Kit HC600
900HC522	ThermoSmart™ Heated Breathing Tube to fit 22 mm (.86") conical connector	900HC627	Elbow Funnel (10-Pack)

*Not available in all countries

3.4 ThermoSmart™ Technology

This device has been designed for optimal patient comfort and offers the unique heated breathing tube technology ThermoSmart™. ThermoSmart™ Technology allows for the delivery of optimal (higher and customized) levels of humidity to the patient, independent of ambient temperature.

This device can be adjusted on two levels to achieve optimal comfort:

ThermoSmart™ Humidity setting: this setting regulates the temperature of the heater plate, adjusting the humidity levels for the comfort of the patient. Adjustment can be made from 1 to 4 with increments of 0.5. The device is preset to 2.5 which is suitable for most patients.

- If you experience dryness or find the air too cool: increase the setting.
- If you find there is too much moisture in the tube or the air feels too hot: decrease the setting.
- If you do not achieve optimal comfort with this setting further adjustments can be made by altering your ThermoSmart™ Climate setting.

ThermoSmart™ Climate setting: this setting provides further adjustment options by optimizing the delivered humidity. Adjustment can be made from 1 to 3 with increments of 1. The device is preset to 2 which is suitable for most patients. Note, if the climate setting is adjusted, the humidity setting will revert back to the default setting.

- ThermoSmart™ Climate setting 3 is recommended for oral CPAP delivery.
- If you feel the range of settings offered by the ThermoSmart™ Humidity Setting are too dry or too cool: Increase the ThermoSmart™ Climate setting.
- If you feel the range of settings offered by the ThermoSmart™ Humidity Setting are too warm or too humid: Reduce the ThermoSmart™ Climate setting.

It is recommended to keep the (heated breathing) tube on top of bedding for the best humidity performance. To operate ThermoSmart™ Humidity and Climate settings, refer to section 8 "Controls and Display" of this manual.

4. INTENDED USE

The device is used to assist with patient breathing while sleeping for the purpose of treating Obstructive Sleep Apnea (OSA). This is done by the delivery of Continuous Positive Airway Pressure (CPAP) in order to prevent airway obstruction. The addition of heated respiratory humidification to the device relieves the drying and irritating effects on the patient airways which usually arises from use of a CPAP system. The device is for use on adult patients at home or in the sleep lab. Please refer to the SleepStyle™ 600 Series product range listed in the appendix section of this manual.

5. OXYGEN USE WARNINGS

If oxygen is required, it is recommended that supplemental oxygen be administered at the mask. Please see instructions specific to your mask type.

NOTE:



- At a fixed flow rate of supplemental oxygen, the inhaled oxygen concentration will vary, depending on the pressure settings, patient's breathing pattern, mask selection and leak rate.

Before using oxygen with the device please see oxygen warnings in Section 2.

6. CLEANING AND MAINTENANCE

PLEASE NOTE: The device should be cleaned as required.









- Switch off the device and unplug from the power socket. Wait for the water to cool, then remove chamber.
- Wipe the exterior of the device with a clean, damp (not wet) cloth and mild dishwashing detergent. Do not use harsh abrasives or solvents, as these may damage the device.
- DAILY**
Clean the chamber, heated breathing tube and heated breathing tube connection port.
 - Remove the heated breathing tube from the device by pulling the sleeve upwards to unlock the connector. Then remove the connector.
 - Wash the heated breathing tube with soapy water, rinse and then hang-up the heated breathing tube with its ends pointing to the floor to dry.
 - Remove the chamber by pushing down the finger guard and pulling out the chamber.
 - Pour out and discard the remaining water.
 - For standard (HC365) chambers, clean and wash with soapy water, then rinse and dry.
 - The extended life (HC360) chamber requires disassembly before cleaning. Push the base off the chamber top as shown in Fig. 1. Ensure the round seal at the base is not loose or damaged. Wash the chamber top and base in a mild dishwashing detergent. Rinse thoroughly and dry. This chamber may also be cleaned in a domestic dishwashing machine.
 - Thoroughly wipe the inside of the heated breathing tube connection port with a clean damp, low lint cloth.
- WEEKLY**
Thoroughly clean the chamber.
 - Soak the inside of the chamber for 10 minutes in a solution of one part white vinegar to two parts water and rinse well with distilled water.
- MAINTENANCE**
 - Replace the air filter when it becomes significantly discoloured, at least once every three months or after 1000 hours machine run time. To do this, remove the filter holder from the back of the device as shown in Fig. 2. Ensure the vertical black line of the new filter is facing towards the device, and fit both back to the device.
 - Replace the chamber if it is cracked, discoloured or scratched.
 - Replace the heated breathing tube after 4000 hours usage or once per year. A damaged tube needs to be replaced immediately.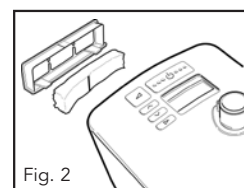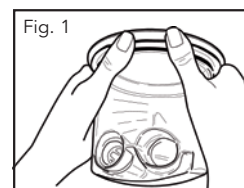

7. SETUP INSTRUCTIONS

When the chamber is fitted and filled, the device must remain upright, so that water cannot enter the enclosure.


- Remove your device from its packaging.
- Place the device on a low shelf or on the floor beside your bed, the unit needs to be placed below head height.
- Remove water chamber from the packaging.
- Remove the blue caps and discard them (Fig. 1).
- Fill the chamber up to the water filling line with distilled water only (Fig. 2).
Never fill the chamber through the heated breathing tube connector port. When moving your device, ensure that the water chamber is empty. Machine failure due to water damage is not covered by warranty.
- To attach the water chamber to the device, press down the finger guard and slide the chamber on (Figs. 3 and 4).
- When the chamber is fitted correctly, the finger guard will click into place.
- One end of the heated breathing tube has a grey plastic connector. Lift the sleeve and slide connector on to the device. Push the sleeve down to lock (Figs. 5, 6 and 7).
- Connect the other end of the heated breathing tube to the mask.
- Plug the device power cord into your household power socket.
- Switch on the device by pressing the on/off button (Fig. 8). The current ThermoSmart™ Humidity setting is displayed.
- For changing the setting refer to section 8 "Controls and Display".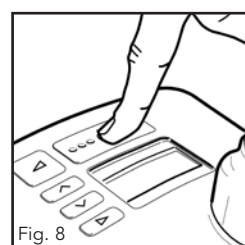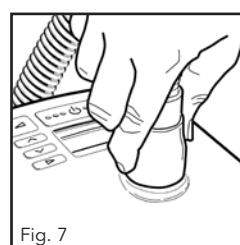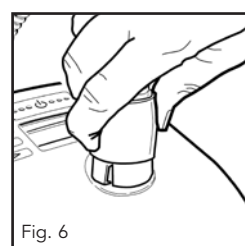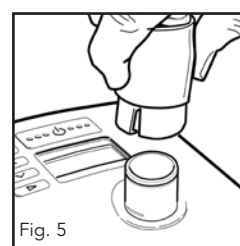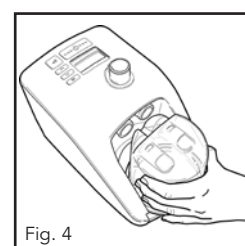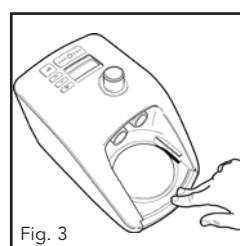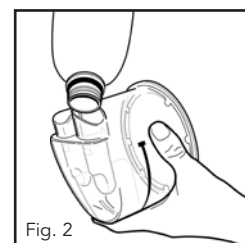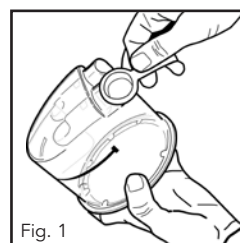

Your device is now ready for use.

8. CONTROLS AND DISPLAY

8.1 Display Description and Functions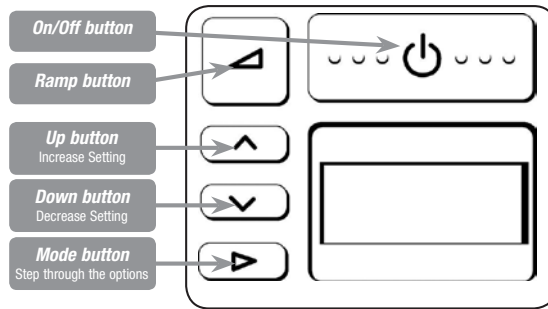

KEY	FUNCTION & DESCRIPTION	OPERATION
	ON/OFF Switches the device ON and OFF	TO SWITCH "ON" <ul style="list-style-type: none"> Press button down briefly The device will default to display the ThermoSmart™ Humidity setting TO SWITCH "OFF" <ul style="list-style-type: none"> Remove your mask Press button briefly
	RAMP Reduces pressure to a lower level to help you fall asleep. The pressure will gradually return to full operating pressure over 20 minutes	TO SWITCH "ON" <ul style="list-style-type: none"> Press (NB: to reset ramp during a cycle the device must be switched off first) TO SWITCH "OFF" <ul style="list-style-type: none"> Press
	INCREASE/DECREASE Adjusts settings up or down	<ul style="list-style-type: none"> Press and button to raise or lower settings
	MODE To step through options	<ul style="list-style-type: none"> Press sequentially to step through and view options available


PATIENT MENU	DISPLAY	OPERATION
For viewing CPAP settings, humidity, compliance data, altitude and pressure settings		TO VIEW ITEMS Press key to sequentially view: <ul style="list-style-type: none"> ThermoSmart™ Humidity setting (default display) For adjusting ThermoSmart™ Humidity setting levels to minimize upper airway side effects from CPAP use Press and button to raise or lower settings
	hr	<ul style="list-style-type: none"> Total run hours: displays the total number of hours the device has been switched on (includes hours without mask use)
	hr/⌂	<ul style="list-style-type: none"> Average compliance hours per night: displays average number of hours the device has been used at night with the mask on
		<ul style="list-style-type: none"> Check sum: displays information for the medical practitioner or the dealer to check compliance
		<ul style="list-style-type: none"> Altitude setting: displays altitude setting. Press or to change altitude
cmH ₂ O	<ul style="list-style-type: none"> Pressure setting: displays pressure setting 	

PATIENT MENU - Additional Settings	DISPLAY	OPERATION
To access additional settings: From the Default setting "THERMOSMART™ HUMIDITY" Press the key for 3 seconds		<ul style="list-style-type: none"> ThermoSmart™ Climate setting* To change the delivered temperature: Press for warmer more humid air delivery. Press for cooler and drier air delivery.* Refer to Section 3.4
		<ul style="list-style-type: none"> Altitude Select "m" for metres or "ft" for feet using and keys
	3000	<ul style="list-style-type: none"> Altitude setting - Set the altitude level by pressing and keys - Press to exit menu
	cmH ₂ O	<ul style="list-style-type: none"> Actual Pressure (Displays actual pressure) - Display reverts to default if the menu is not exited. Press button in order to exit menu (only available in HC608)


Warning: Failure to select correct altitude setting (for any given location) will have an adverse effect on delivered pressure.

- * When the ThermoSmart™ Climate setting is altered the Humidity Setting will revert back to default setting 2.5

8.2 Patient Menu Summary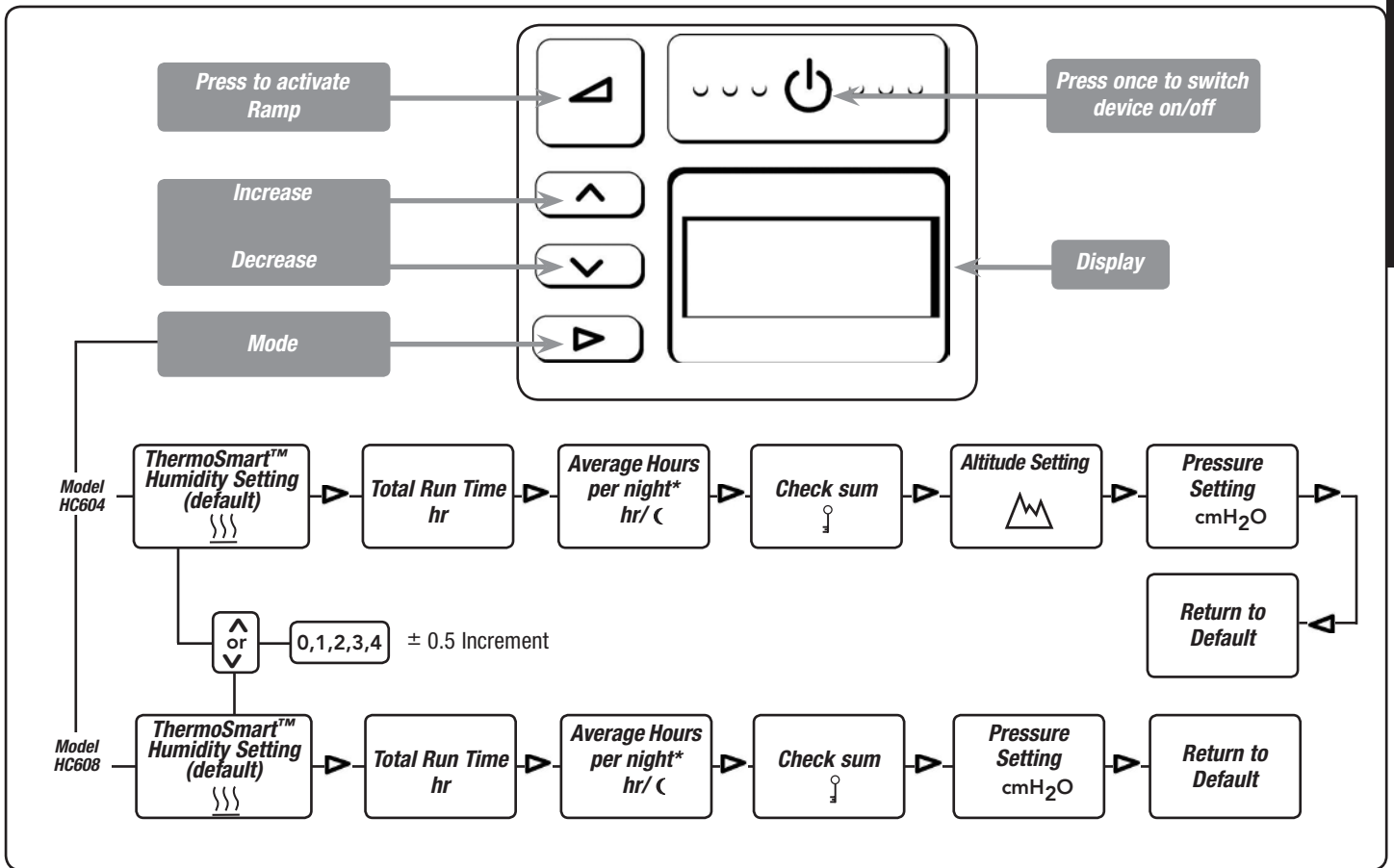

8.3 Patient Menu - Additional Settings Summary

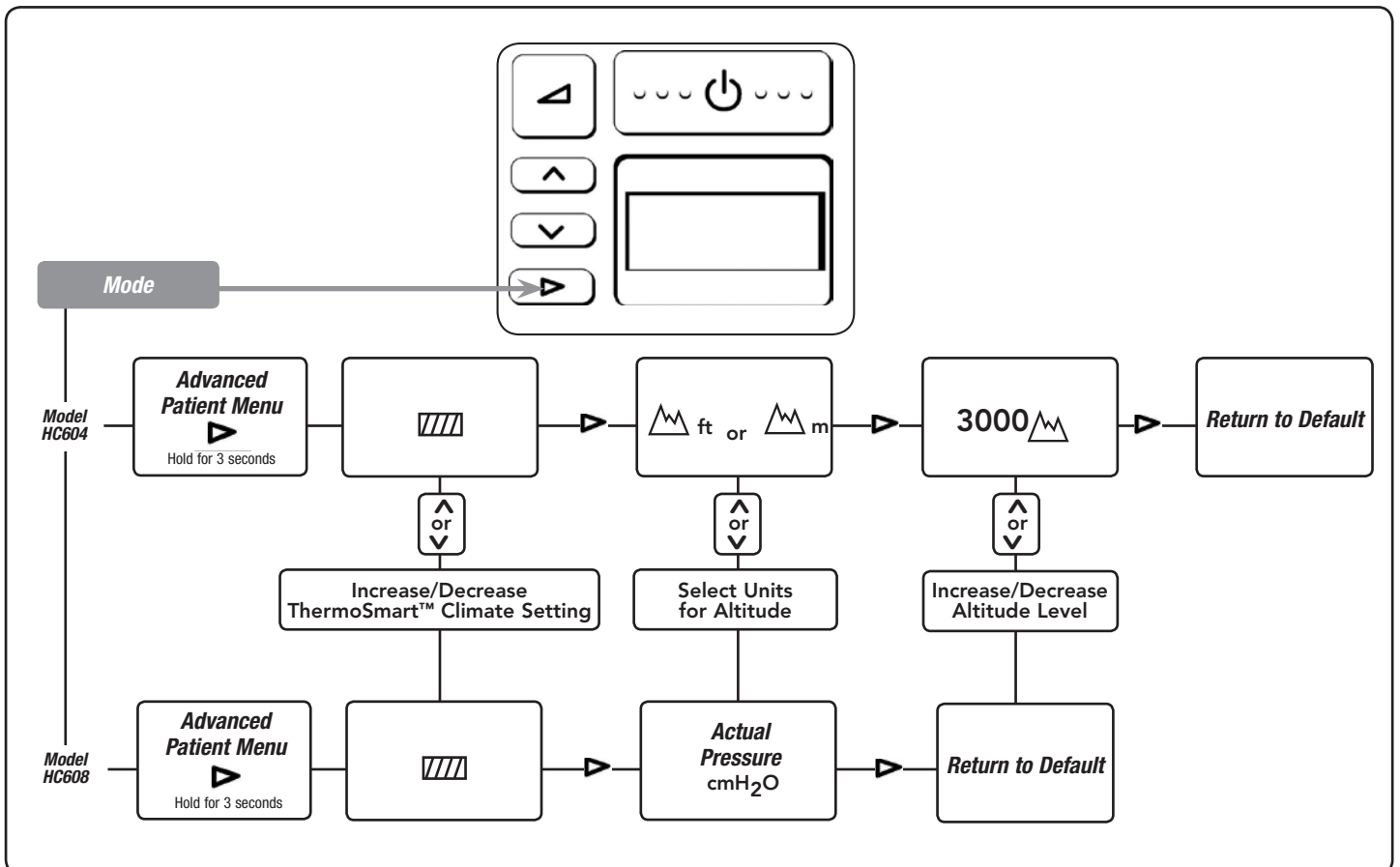- **Warning:** Failure to select correct altitude setting (for any given location) will have an adverse effect on delivered pressure.

9. FREQUENTLY ASKED QUESTIONS

Do I have to use distilled water with my device?

The use of distilled water will maximize the life of the water chamber. Water from the faucet will often contain minerals which can damage the chamber, causing pitting in the base, even if the water is passed through a filter.

Can I use my device in other countries?

Yes. Simply use the appropriate electrical socket adaptor and the device will automatically adapt to any voltage supply.

Note: When moving your device, ensure that the water chamber is empty. Machine failure due to water damage is not covered by warranty.

Can I use supplemental oxygen with my device?

Yes, oxygen can be administered at the mask. Turn the device on before turning on the oxygen. Ensure that the oxygen source is turned off before turning off the device to avoid oxygen accumulating in the machine.

When do I replace my water chamber?

Replace the chamber if the plastic walls of the chamber become cracked or cloudy or the chamber base becomes pitted.

(Note: If the chamber leaks at all, it should be replaced immediately).

Can I use my device without a heated breathing tube?

Yes you can, but you will not gain the optimum humidity performance the device offers.

The device will then not display the heated breathing tube symbol and show the following display:

What happens to my device during power failure?

Remove the mask immediately. Upon restoration of the power supply, the device will restart in the same operation mode with the same settings as before the power failure. Refer to Section 2 for additional information.

10. OPERATING CONDITIONS

AMBIENT TEMPERATURE: 41 - 95 °F (5 - 35 °C)

HUMIDITY: 10 - 95% Relative Humidity (RH)

ALTITUDE: 0 - 9000 ft (0 - 3000 m)

Above 1500 m (4500 ft) the maximum operating pressure will be reduced.

11. STORAGE AND TRANSPORT CONDITIONS

The device should be stored and transported in environmental conditions of: 14 to 140 °F (-10 to 60 °C).

12. TROUBLESHOOTING

If you feel that your device is not operating correctly, consult your healthcare provider. Do not attempt to repair the device yourself.

13. WARRANTY

Please contact your Fisher & Paykel Healthcare Provider regarding warranty.

14. DEVICE AND CONSUMABLES DISPOSAL INSTRUCTIONS

- Unit Disposal Instructions

This unit contains electronics. Please do not discard as regular rubbish. Dispose according to local guidelines for disposing of electronics.

- Consumable Disposal Instructions

Place the mask, breathing tube and water chamber in a rubbish bag at the end of use and discard with normal rubbish.

15. PRODUCT SPECIFICATIONS

DIMENSIONS:	10.7" x 6.7" x 6.9" (273 mm x 170 mm x 175 mm)
WEIGHT:	4.6 lbs (2.1 kg) unit only 7.5 lbs (3.4 kg) packaged in bag incl. accessories
ELECTRICAL RATINGS	
Supply Frequency:	50 - 60 Hz
Supply Voltage and Current:	100 - 115 V (1.5 A, max 1.7 A), 220 - 240 V (1.0 A, max 2.0 A) Note: These values represent average current.

DC to AC Inverter requirements:	115 V - 200 W/300 W surge 230 V - 300 W/500 W surge
--	--

Heater Plate:	max. 85 W
Heater Plate Temperature:	max. 149 °F (65 °C)
Heated breathing tube Power Setting:	max. 30 W

PERFORMANCE

Pressure Range:	4 to 20 cmH ₂ O
Altitude Range:	0 to 9000 ft, or 0 to 3000 m

Maximum Flow Rates

CPAP Pressure Setting (cmH ₂ O)	4	8	12	16	20
Measured pressure at the patient connection port (cmH ₂ O)	3	7	11	15	19
Maximum flow rate (L/Min) at mask connection port	60	78	80	81	83

Static Pressure Stability (long term)

-0.05 to 0.18 cmH₂O pressure difference, measured at the mask connection port, at the pressure setting of 10 cmH₂O

Dynamic Pressure Stability (short term)

CPAP Pressure Setting (cmH ₂ O)	4	8	12	16	20
Pressure Difference (cmH ₂ O) at mask connection port	.73	.74	.74	.81	.79

Warm-up time:	30 min
Humidity delivered:	30 mg/L*
Gas Temperature:	max. 98.87 °F (37.15 °C)

*measured at 22 °C ambient temperature: 10 cmH₂O at bias flow.

STANDARDS COMPLIANCE:

Designed to conform to the requirements of:	IEC 60601-1 UL60601-1 CSA C22.2/No. 601.1 AS3200.1.0 EN60601-1 ISO 17510-1
---	---

The device complies with the electromagnetic compatibility requirements of IEC 60601-1-2. In certain circumstances, the device may affect or be affected by nearby equipment due to the effects of electromagnetic interference. If this should happen, try moving your unit or the location of the device causing interference, or alternatively consult your healthcare provider.


16. APPENDIX

The following table lists the main difference in features across the SleepStyle™ 600 Product Range.

MODEL	Fully Integrated CPAP with Heated Humidification	Patient Compliance Monitoring (LCD & Download)	Auto Altitude Adjustment/ Leak Compensation
HC604	✓	✓	-
HC608	✓	✓	✓

NOTE:

- All models are fully integrated with ThermoSmart™ Technology.
- Some models are not available for sale in all countries.

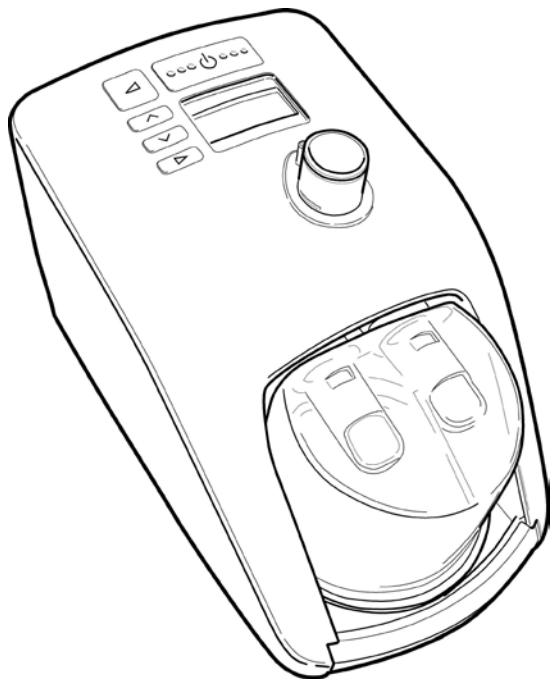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