# iQ® Nasal Mask

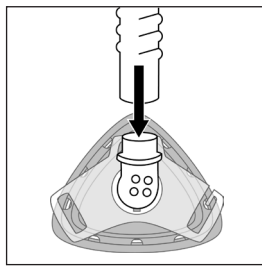
Figure 1


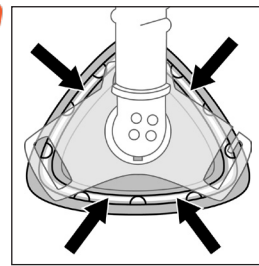
Figure 2

## VENT HOLE PRESSURE/FLOW CHARACTERISTICS (Approx.)
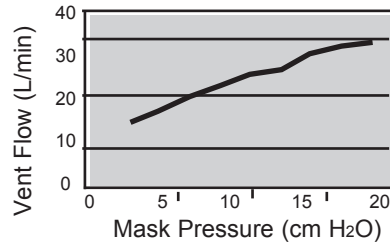
### Sleepnet Corporation

5 Merrill Industrial Drive  
Hampton, NH 03842 USA

Tel: (603) 758-6600

Fax: (603) 758-6699

Toll Free: 1-800-742-3646 (USA AND CANADA ONLY)

www.sleepnetmasks.com

EU Rep: Medical Product Services GmbH

Borngasse 20, 35619 Braunfels Germany

info@mps-gmbh.eu

90160-W Rev. A

## iQ® Nasal Mask

CONTENTS: iQ® Nasal Mask, Tubing Assembly

Rx ONLY LATEX FREE MADE IN U.S.A.

The Sleepnet Corporation iQ® Nasal Mask is intended to be used with positive airway pressure devices, such as CPAP, operating at or above 3 cm H<sub>2</sub>O for the treatment of adult obstructive sleep apnea.

### GENERAL INFORMATION - WARNINGS AND CAUTIONS

**CAUTION:** U.S. Federal Law restricts this device for sale by or on the order of a physician.

**CAUTION:** Do not expose the iQ® Nasal Mask to temperatures above 122°F (50°C).

**CAUTION:** This mask may be cleaned and used repeatedly on the same person. Do not sterilize. USA ONLY: For multiple patients use, high level disinfection protocol must be followed.

**WARNING:** At low baseline pressure, the flow through the vent holes may be inadequate to clear all exhaled gas from the tubing. Some re-breathing may occur.

**WARNING:** This mask should be used with the positive airway pressure device recommended by the manufacturer, your physician, or a respiratory therapist. The mask should not be used unless the positive airway pressure device is turned on and operating properly. The mask's vent holes should never be blocked. The vent holes allow a continuous flow of air out of the mask. When the positive airway pressure device is turned on and functioning properly, new air from the device flushes the exhaled air out through the mask's vent holes. However, when the positive airway pressure device is not operating, exhaled air may be re-breathed. Re-breathing of exhaled air can, in some circumstances, lead to suffocation. This warning applies to most models of nasal positive airway pressure devices and masks.

**WARNING:** If oxygen is used with this CPAP mask, the oxygen flow must be turned off when the CPAP machine is not operating. Explanation of the warning: When the CPAP device is not in operation, and the oxygen flow is left on, oxygen delivered into the ventilator tubing may accumulate within the CPAP machine enclosure. Oxygen accumulated in the CPAP machine enclosure will create a risk of fire.

**WARNING:** No smoking or open flames such as candles may be used when oxygen is in use.

**WARNING:** Do not use the iQ® Nasal Mask with baseline pressure less than 3 cm H<sub>2</sub>O or greater than 20 cm H<sub>2</sub>O.

**WARNING:** Consult manufacturer for proper O<sub>2</sub> enrichment tubing.

### **GENERAL INFORMATION**

The iQ® Nasal Mask has a 3 month warranty for manufacturing defects. If a manufacturing defect is suspected, contact your supplier. The mask has an expected service life of 6 to 12 months. Please dispose of the mask and accessories in accordance with local and regional guidelines. The iQ® Nasal Mask and its accessories are Latex Free.

### **CLEANING AND MAINTENANCE - SINGLE PATIENT USE**

Cleaning is recommended on a daily basis. Wash the mask and components in warm water using a mild detergent. Rinse the mask thoroughly after washing. Visually verify that the vent holes are clear and allow the mask to air dry. Washing your face prior to fitting the mask will remove excess facial oils and help extend the life of the mask.

NOTE: Inspect mask prior to each use. Replace the mask if parts are damaged or gel is exposed due to tear or punctures.

NOTE: The gel cushion, although silky to touch, will not tear, leak or detach from the mask under normal use and following recommended cleaning and maintenance protocols. However, the gel cushion is not indestructible. It will tear, cut and rip if subjected to mishandling or abuse. Do not clean in a dishwashing machine. The warranty does not cover these types of product failures. Therefore, please handle your new iQ® Nasal Mask with care.

### **TECHNICAL INFORMATION**

Resistance to air flow at 50 L/min: 1.10 cm H<sub>2</sub>O; at 100 L/min: 2.52 cm H<sub>2</sub>O

A-weighted Sound Pressure Level at 1 m distance: 29.9 db (A); A-weighted Sound Power Level: 37.9 db (A)

### ***iQ® Mask Assembly***

---

1. Attach the tubing assembly to the iQ Nasal Mask (Figure 1).
2. Connect your positive airway pressure device to the swivel connector on the tubing. Turn unit on and verify airflow through the mask.
3. Connect Headgear (not included) as per instructions.  
Note: Mask is configured for 3 point headgear. To use 2 point headgear, you must rotate the headgear connector 180° then press firmly against the shell to snap into place.
4. Gently bend pliable ring embedded inside mask shell to change mask shape, create a comfortable fit, and reduce leaks (Figure 2).

 **iQ®** is a trademark of Sleepnet Corporation.  
Visit our website at [www.sleepnetmasks.com](http://www.sleepnetmasks.com).

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EU Rep: Medical Product Services GmbH  
 Borngasse 20, 35619 Braunfels Germany  
[info@mps-gmbh.eu](mailto:info@mps-gmbh.eu)

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iQ®, Sleepnet Corporation'ın ticari markasıdır.